

Etac Hi-Loo & Supporter Manual

etac®


74949.19 23-01-17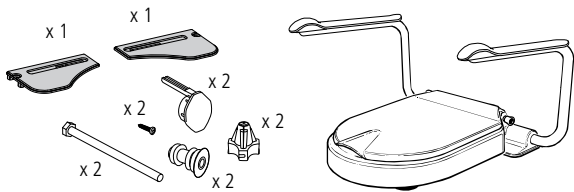


8030 1116-2 / 8030 1117-2 / 8030 1118-2 / *8400 1291 / *8400 1292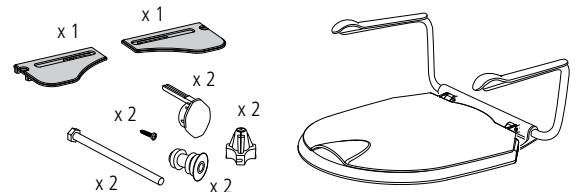


8030 3012-2 / *8400 1293

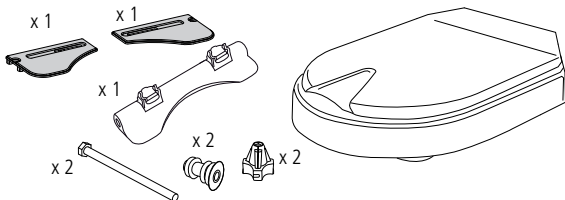
8030 1114-2 / 8030 1115-2

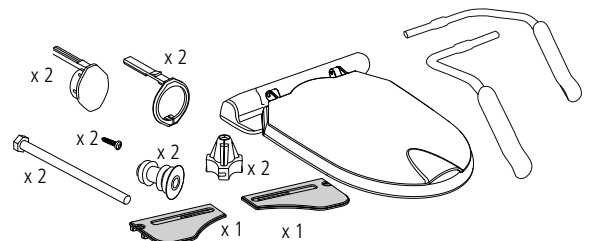
8030 3112-2 / *8400 5101

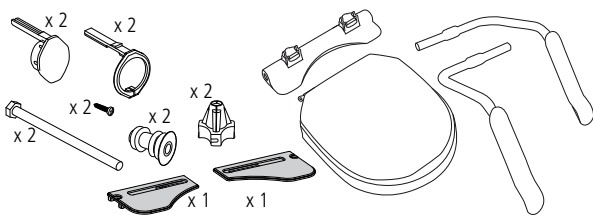
8030 1316-2 / 8030 1317-2 / 8030 1318-2 / *8400 5099 / *8400 5100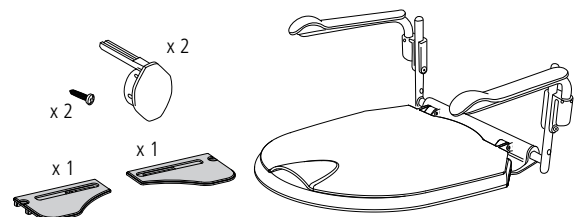


8030 3018-2 / *8400 1294

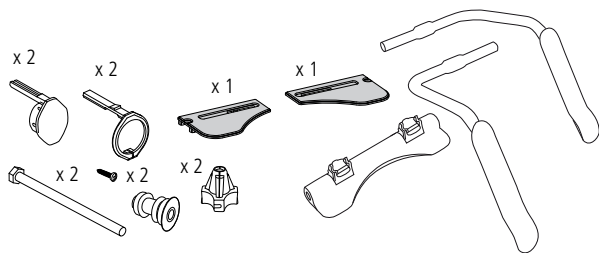8400 1299 / 8400 1322

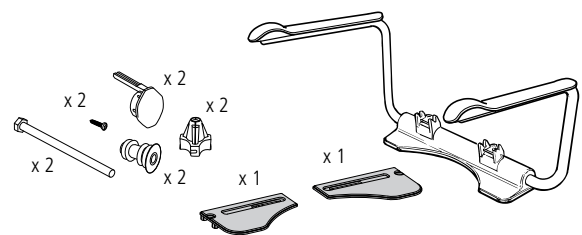
8400 1297 / 8400 1298



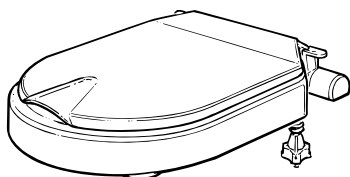
8030 1106 / 8030 1107

| **sv** *Reservdel, endast armstöd | **no** *Reservdel, kun armlene
 | **da** Reservdel, kun armlæn | **en** *Spare part, only armrest
 | **de** *Ersatzteil, nur Armlöhne | **fi** *Varaos, vain käsinoja
 | **fr** *Pièce détachée, repose-bras uniquement | **nl** *Reserveonderdeel, alleen armleuning
 | **is** *Varahlutur, eingöngu armhvíla | **it** *Ricambio, solo bracciolo
 | **es** *Piezas de recambio, solo reposabrazos
 | **pt** *Peça sobressalente, apenas apoio para os braços


A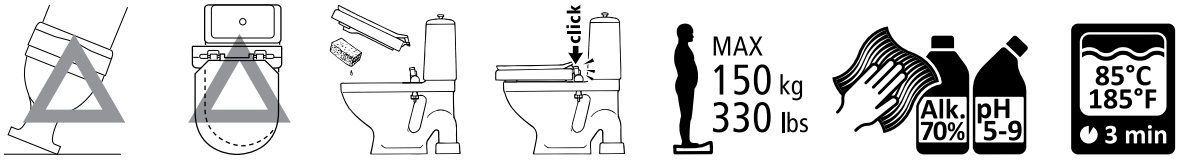


B

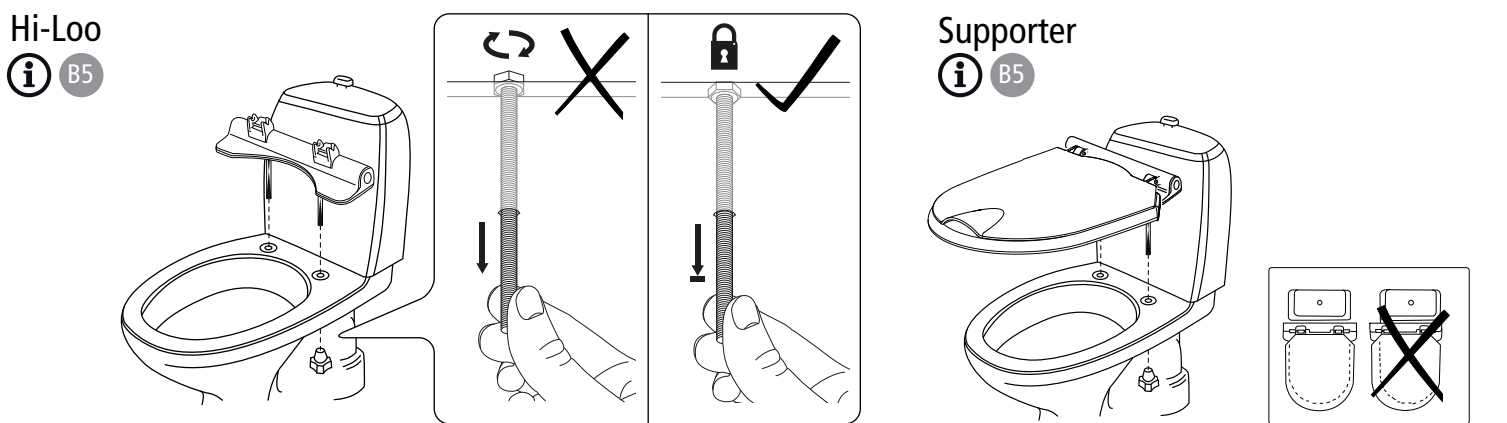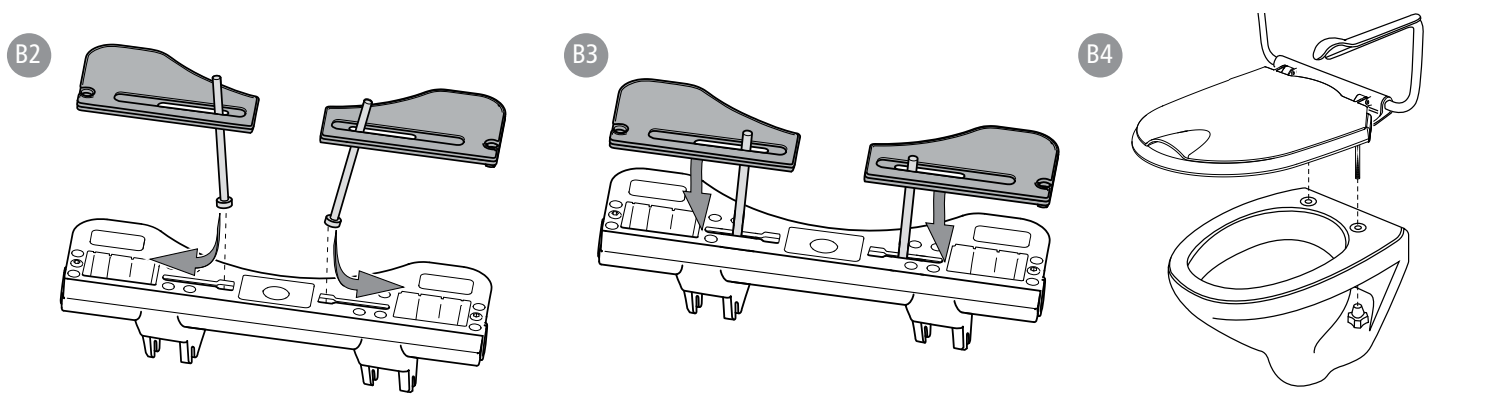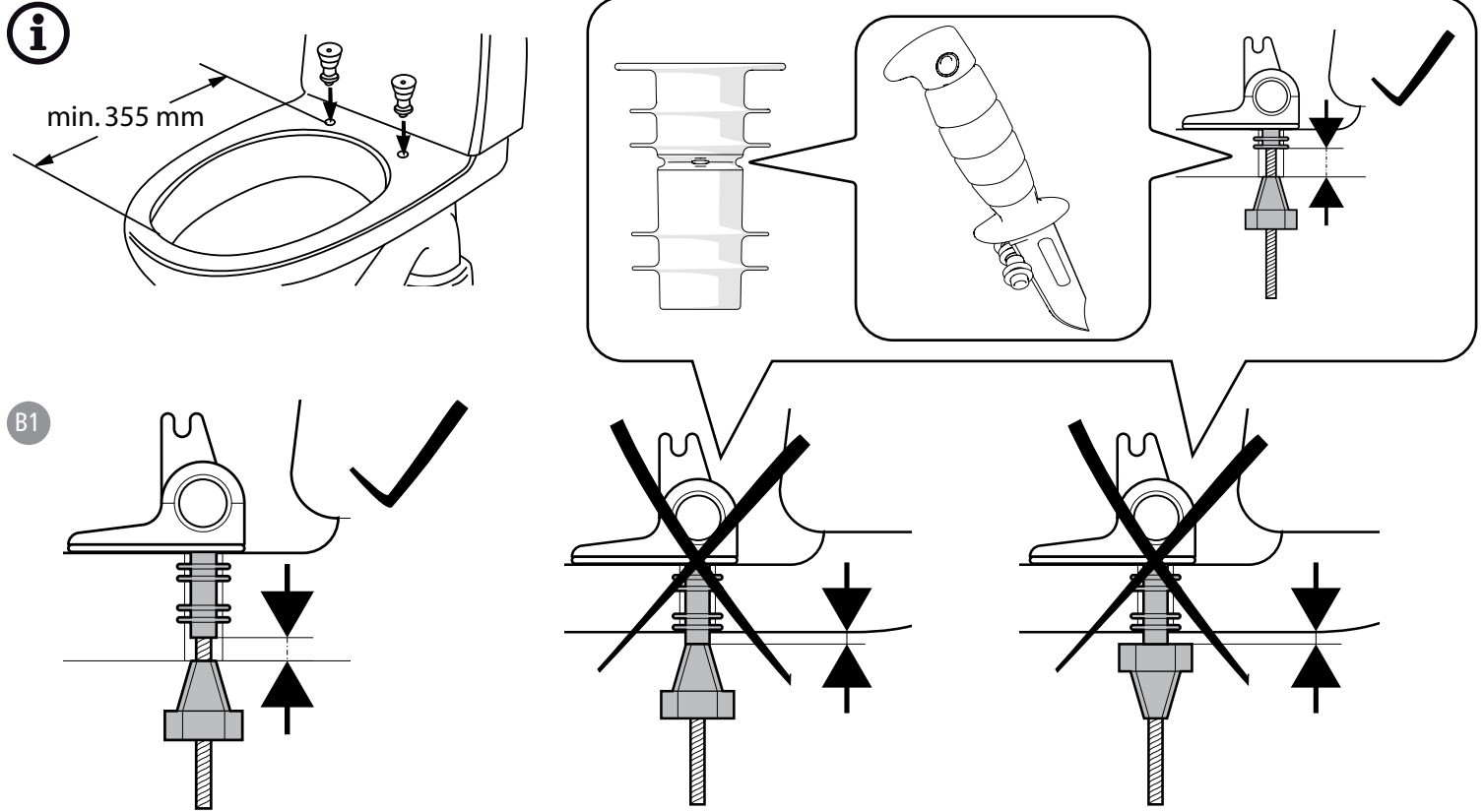
B6

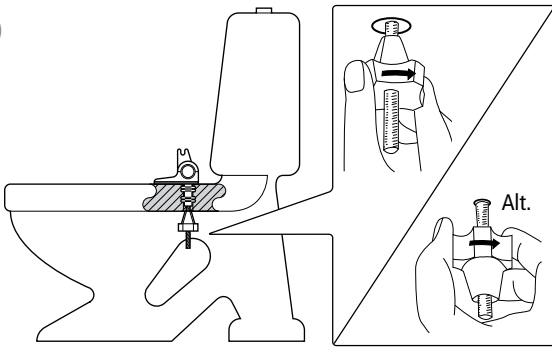
i 8030 1106 / 8030 1107

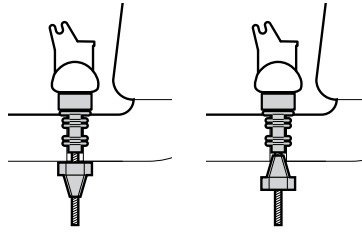
B7

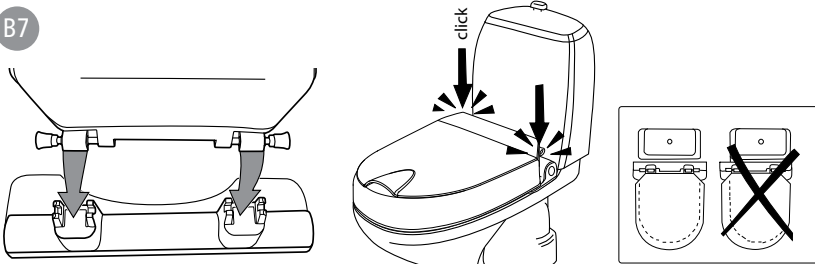
C

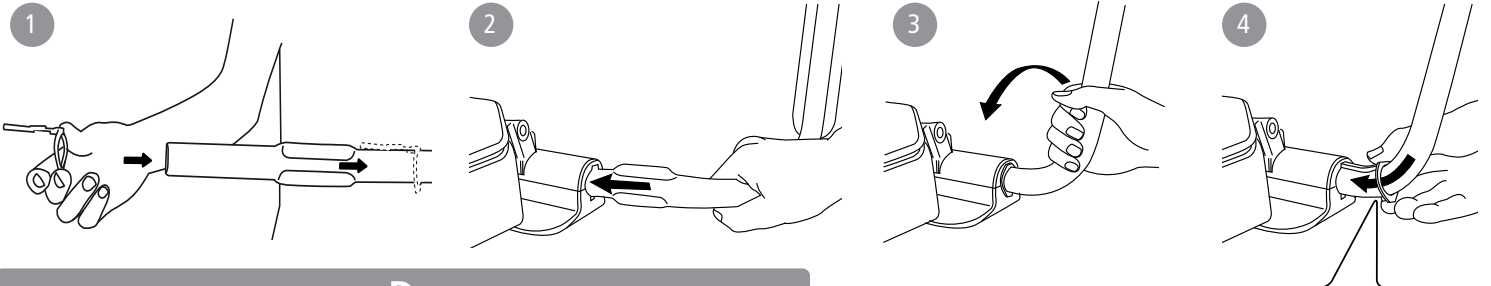
D

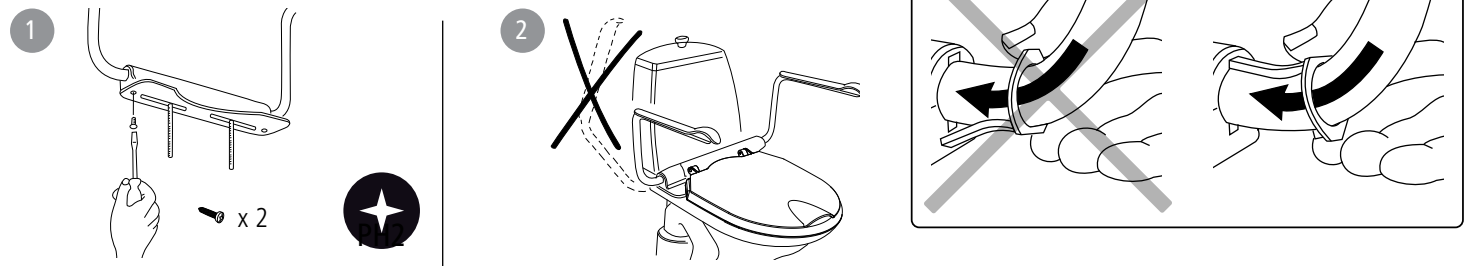E

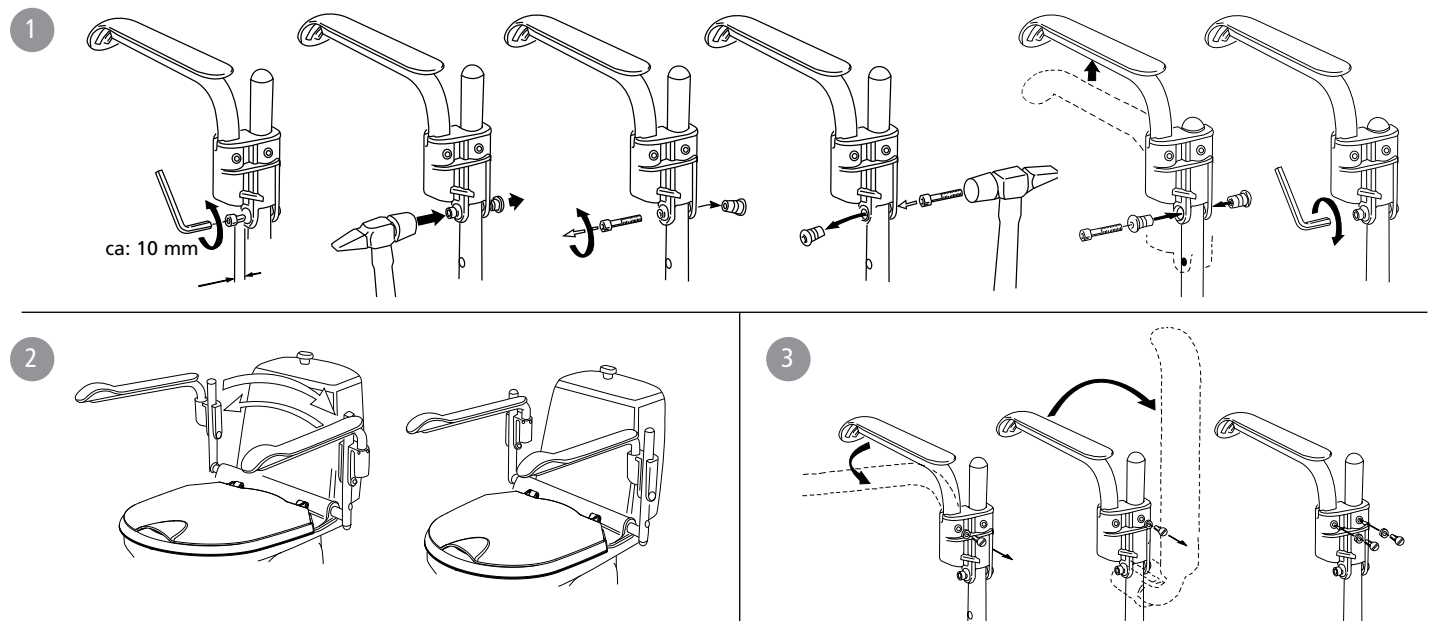F

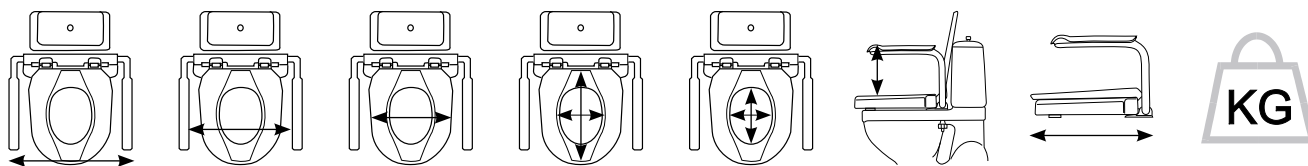
Supporter	600 mm	490 mm	365 mm	-	220/285 mm	250 mm	460 mm	4,8 kg
Supporter adj.	620/513 mm	512/336 mm	365 mm	-	220/285 mm	240/310/375 mm	460 mm	4,8 kg
Hi-Loo 6 cm	600 mm	490 mm	365 mm	210/360 mm	195/270 mm	230 mm	460 mm	4,9 kg (2,6 kg*)
Hi-Loo 10 cm	600 mm	490 mm	365 mm	210/360 mm	195/270 mm	230 mm	460 mm	5,8 kg (3,4 kg*)


Fe	Zn	Al	PP	PA	TPE	HDPE		
----	----	----	----	----	-----	------	--	--

G

G1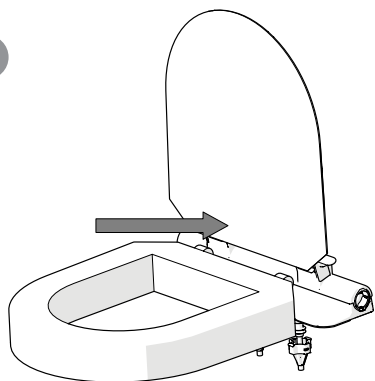

G2

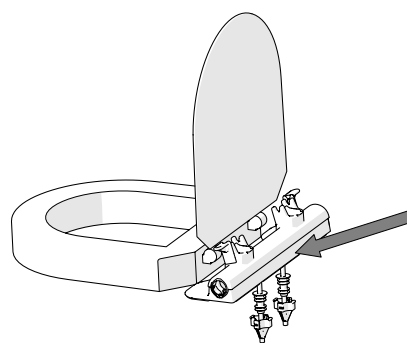sv no da en de fi fr nl is it es pt

sv

Svenska

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English

Thank you for choosing an Etac product. In order to avoid damage during assembly, handling and use, it is important to read this manual and save it for future reference. You can also find it at www.etac.com. You can select your language via the "International" and "Local websites" link. Here you will also find other product documentation, such as prescriber information, pre purchase guide, and reconditioning instructions.

Intended use

Hi-Loo is a range of ergonomic toilet seat raisers that facilitates sitting down and getting up from the toilet. Hi-Loo can be used with or without armrests. Armrests provide greater security when seated and support when you sit down/get up from the toilet. The armrests can be flipped up, but can also be fixed in the lowered position.

Supporter is a range of toilet armrests that facilitates sitting down and getting up from the toilet. The armrests provide greater security when seated and support when you sit down/get up from the toilet. The armrests can be flipped up, but can also be fixed in the lowered position.

The product is intended for use in home environments and in institutions.

The product must not be used in any other way than that stated above.

The product conforms to the requirements of the Medical Devices Regulation (EU) 2017/745.

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

Contra indications

There are no known contraindications.

Guarantee, service life

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

Service life 10 years. For complete information regarding the service life of the product, see www.etac.com.

The product's date of manufacture can be read from the barcode on the product. The number 11 is shown below the barcode in brackets. The number combination after these brackets is the date of manufacture.

Storage

The product should be stored indoors in a dry place at a temperature above 5°C. If the product has been stored for a long time (more than four months), its function must be checked by an expert before use.

Symbols [Figure A](#)

The toilet chair must be securely fixed in place / The product must be fitted at the centre of the toilet chair / The seat ring (Hi-Loo) can be removed when cleaning. Make sure it clicks in place completely when refitting / Maximum user weight 150 kg* /

Clean the product with a solvent-free detergent which has a pH of 5-9, or with a 70% disinfectant solution. Can be decontaminated at max. 85°C if so required.

*In order for the max. user weight 150 kg to be valid, both the porcelain attachment and the armrests must be approved for this weight (check the marking on the product label and the label on the armrest). Supporter adjustable, user weight 130 kg.

Installation..... [Figure B](#)

For wall hung porcelain, mount armrests before step 4.

Mounting armrest..... [Figure C](#)

Fixed armrest..... [Figure D](#)

Height adjustable armrests..... [Figure E](#)

1. Height adjustment / 2. Width adjustment /

3. Folding, fixing

Technical data..... [Figure F](#)

*Without armrest

Accessories

- Adjustable armrest for Supporter
- Seat unit for angled Hi-Loo
- Complete bracket kit
- Crossbar
- Screw and fixing kit special Supporter
- Adjustment kit

Position of labels..... [Figure G](#)

1. Product label (warnings, CE marking)

2. Article number label (serial number, date of manufacture)

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Etac Supply Center AB