Babywaage
Baby scale
Pèse-bébé

Cosy
8310.02

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1. General information

Thank you for choosing a Soehnle Professional product from Soehnle Industrial Solutions. This product is state of the art.

Please read the instructions carefully before use.

If you have any questions or if problems occur with your device that are not covered in the operating instructions, please contact your Soehnle Industrial Solutions service center or our customer service:

Phone: +49 7191 3453-220

fax: +49 7191 3453-211

E-mail: info@soehnle-professional.com

These operating instructions describe all standard functions of the 8310.02 baby scale.
Deviations are possible with special versions.

Scope of delivery:

- ▶ Babywaage 8310.02
- ▶ Gebrauchsanweisung
- ▶ 4x AA batteries

Further information and documentation can be found under:

<https://www.soehnle-professional.com/site/documents>

Please go to the customer center on our website www.soehnle-professional.com and select the baby scale 8310.02 under Downloads.

2. Technical Data

	Baby scale "Cosy" 8310.02
Max. capacity:	20 kg
Division:	5 g
Platform size:	280 x 600 mm / 11" x 23.62"
Construction:	ABS housing, four point load cell
Weighing units:	kg, lb
Calibration units:	kg
Application modes:	Weighing
Display:	LCD
Function keys:	ON/TARE, UNITS, RECALL
Operating temperature:	5°C to 35°C / 40°F to 95°F
Auto-Off:	after 3 minutes w/o operation
Low battery indicator:	< 4,5 V ± 0,2
Power supply:	4 x AA batteries
	AC adapter

3. Functions

Functions:

ON/TARE (OFF): Press this button to turn on the balance. During the weighing process, press the key again to clear the zero point and reset the balance to zero. In the „0.000“ state, press this key for 3 seconds and the balance will turn off.

HOLD: After stable weighing (0 in the upper left corner of the display), the current weight is automatically locked. After locking, adding or reducing the weight by more than 800 g, the hold function is enabled, the weight changes and displays the new weight.

UNITS: Select key for weight unit, press this key to select kilogram or pound. The last unit will be saved when the power is turned off.

MEM: When „0.000“ is displayed, press the RECALL key to retrieve the last weight. The „MEM“ symbol and the weight data appear in the lower right corner of the display. Press ON/TARE to display the determined weight. Press the RECALL button again to return to the weighing mode.
The minimum load for this is > 1.0 kg.

Operation: Insert batteries. Press „ON/TARE“ to turn on the balance. The balance will display 0.000 after the self check.

Weight measurement: Carefully place the baby on the scale. As soon as the scale is stable, the „0“ sign is displayed. After the baby has been lifted off the scale, the weight value automatically returns to zero. The scale automatically turns off after weighing if it is not used again within 1 minute.
When „0.000“ is displayed, press the „MEM“ key to display the weight of the last weight stored.

Notes:

1. Keep the balance clean to ensure normal use.
2. The scale should be placed on a flat, hard surface and not on a carpet.
Failure to do so will compromise accuracy.
3. Do not overload the scale (weigh more than 23 kg) as this may damage the scale.
4. Replace the batteries immediately when „Lo“ appears.
If you will not be using the scale for a long period of time, remove the batteries to prevent possible damage from leaking battery fluid.
5. When you disconnect the mains adapter from the power source or replace the batteries, the scale automatically resets to the lbs weighing unit. It is therefore necessary to set the scale to kg again. To do this, simply press the „Units“ key. The weight value will then appear in kg again. This procedure is not necessary if the scale is only switched off!

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All rights reserved.
Soehnle Industrial Solutions GmbH