



- Ø 1-3mm | Power: █
- Ø 2-4mm | Power: █ █
- Ø 3-5mm | Power: █ █ █
- Ø 5-8mm | Power: █ █ █ █

With better control of procedures the CryoPen XP sets new standards for superb outcomes.

The CryoPen XP dispenses a fine jet of nitrous oxide under high pressure which allows you to work with millimeter precision.

It offers the ideal solution for accurate, quick, effective, stress-free treatment of all skin imperfections.

Instrument

This instrument is well prepared for all treatment options. With the possibility to use both 8g and 16g cartridges and the complete range of applicators, CryoPen XP gives additional value in all situations.

Cartridges

8g N₂O cartridges: 1 box containing 24 cartridges
1 carton containing 12 boxes

16g N₂O cartridges: 1 box containing 6 cartridges
1 carton containing 12 boxes

Once punctured the pressure will remain without loss for many weeks. Shelf life 36 months. Storage at room temperature.

CryoPen XP set with 4 applicators

Blue ring applicator: Ø 1-3mm
flow time 200-230sec (8g) / 370-430sec (16g)

Grey ring applicator: Ø 2-4mm
flow time 145-175sec (8g) / 270-340sec (16g)

Green ring applicator: Ø 3-5mm
flow time 95-110sec (8g) / 175-215sec (16g)

Yellow ring applicator: Ø 5-8mm
flow time 60-70 (8g) / 110-130sec (16g)

What kind of lesions?

Acrochordon, actinic keratosis and cheilitis, capillary haemangioma, cutaneous leishmaniasis, discoid lupus erythematosus, filiform, flat, vulgar, plantar and anogenital warts, rosacea, lichen sclerosus, granuloma annulare, molluscum contagiosum, seborrheic keratosis and viral lesions.

About the procedure

No special preparation is required. Treatment is applied for 2 to 25 seconds on each lesion. In larger areas circular or crossing movement is required to include all tissue.