

MEDISAFE[®] FINETOUCH[®] II

Lancing device for capillary blood sampling



The puncture with a gentle touch

Gentle blood collection




Less pain¹

Needle design


Hollow needle specially cut for noticeable pain reduction.¹

5 adjustable penetration depths

- Adaptable to individual patient's skin type.
- Minimum penetration depth is 0.3 mm.
- Allows particularly gentle blood sampling.

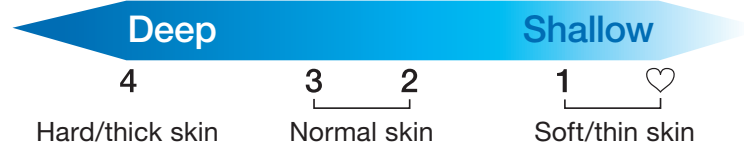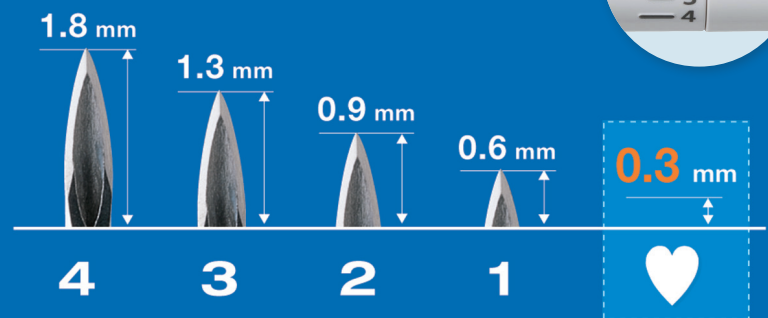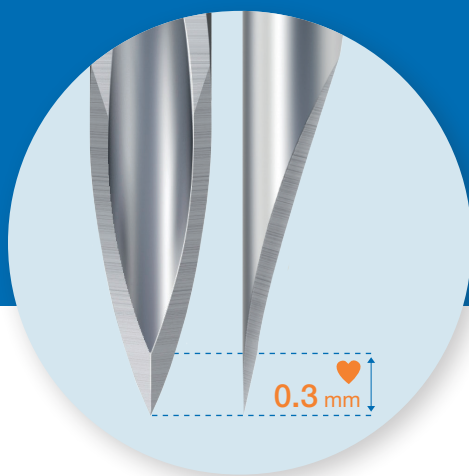


Protected use and hygienic handling

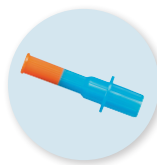

Protection

The lancet needle is surrounded by a plastic holder, before and after penetration, preventing needle stick injury.

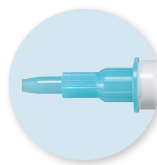


Prevention

The lancet holder protects the lancing device from skin contact and contamination with blood, thus preventing cross-contamination.



Safeguard

Multiple use of the lancet is not possible. The triggering mechanism will only be released once a new lancet has been attached.

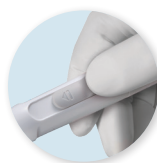

Easy to remove

Easy lancet removal with the ejector. Used lancet can be disposed simply and safely.



Easy to target

The slim lancet holder tip makes the desired puncture site particularly easy to target.



Easy to clean

Lancing device is easy to clean due to the smooth surface design.

The lancet is inserted into the lancing device in one simple move. Once you hear the “Click”, the device is ready for use.


1. Setting the penetration depth

Use the dial to set desired lancet penetration depth.



2. Inserting the lancet

Insert the lancet up to the stop (clear Click). Twist and remove protective orange cap.

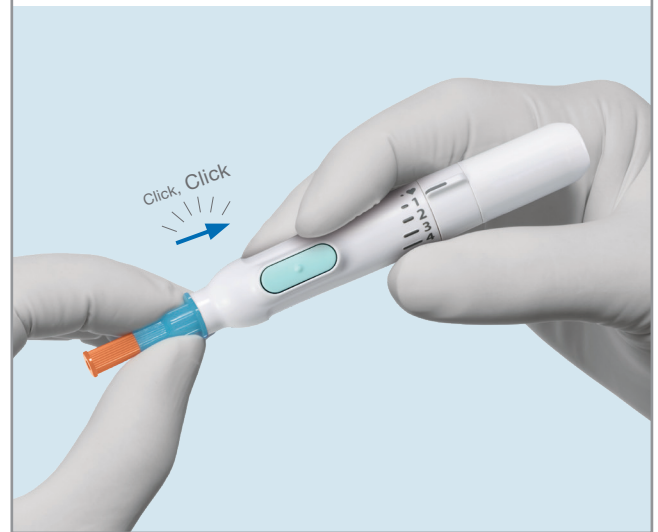

3. Blood sampling

Hold the lancet gently against the skin and press the push button.




4. Disposal



Remove used lancet by sliding the ejector and dispose of in designated sharps container.



Specifications

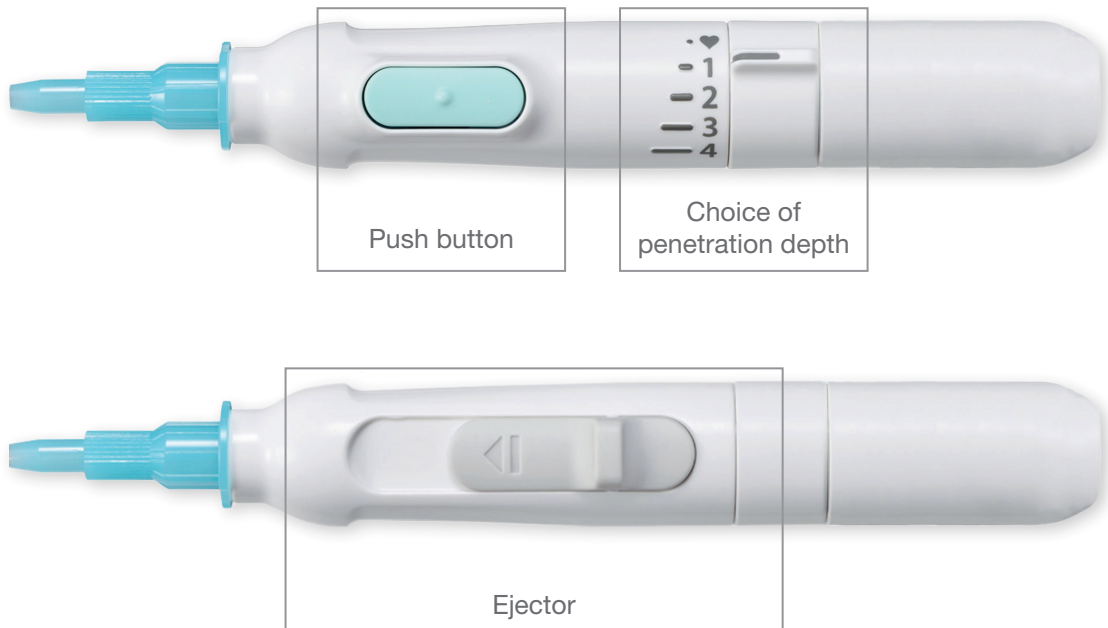MEDISAFE®
FINETOUCH® II



	Product Name	Code	Languages	Pieces
	MEDISAFE FINETOUCH II Lancing Device without Set Lever	MS*FT2S	EN, DE, FR, IT, NL, ES, PT, AR	1 / 10 / 100
	TERUMO FINETOUCH Lancet	Country related [△]	Country related [△]	25 / 250 / 2500 [△] 30 / 300 / 3000 [△]

Contains neither components made of natural rubber latex nor PVC.

[△] Please contact your local Terumo office.

1 Intra company data shows that puncture with MEDISAFE FINETOUCH II lancing device is experienced less painful compared to TERUMO FINETOUCH lancing device.

This document is an overview and not intended to be comprehensive. For more details or in case of questions we refer to the instructions for use delivered with the product or call your local Terumo office.

MEDISAFE FINETOUCH II lancing device is manufactured by Taidoc Technology Corporation. FINETOUCH lancet is manufactured by Terumo Corporation. TERUMO®, MEDISAFE® and FINETOUCH® are registered trademarks of Terumo Corporation.



Terumo Corporation
+81 3 3374 8111

Terumo Europe NV
+32 16 38 12 11

EMEA Sales Offices

**Terumo Europe NV
Africa Business Division**
+32 16 38 13 08

**Terumo Europe NV
Benelux Sales Division**
Belgium:
0800 14468
The Netherlands:
0800 0220396

**Terumo Europe NV
Emerging Market Division**
+32 16 38 12 11

Terumo Deutschland GmbH
+49 6196 80 230

**Terumo Deutschland GmbH
Switzerland**
+41 56 419 10 10

Terumo Europe España SL
+34 902 10 12 98

Terumo France S.A.S.
+33 130 96 13 00

Terumo Italia S.r.l.
+39 06 94 80 28 00

Terumo Russia LLC
+7 495 988 4740

Terumo Sweden AB
+46 31 748 58 80

**Terumo Sweden AB
Denmark**
+45 7020 93 80

Terumo Middle East FZE
+971 4 292 0200

Terumo UK Ltd
+44 1276 480 440

**Terumo BCT Tıbbi Cihazlar
Dağıtım ve Hizmetleri A.Ş.**
+90 216 645 92 00



© Registered Trademark
Published by Terumo Europe NV
DI 142GB-0318FK-I(03.18)E