- DE** Mechanische Personenwaage  
Gebrauchsanleitung
- EN** Mechanical personal bathroom scale  
Instruction for use
- FR** Pèse-personne mécanique  
Mode d'emploi
- ES** Báscula mecánica  
Instrucciones para el uso
- IT** Bilancia pesapersone meccanica  
Istruzioni per l'uso
- TR** Mekanik terazî  
Kullanma Talimatı

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- Repairs may only be performed by Customer Service or by accredited retailers.
- Cleaning: You can clean the scale with a damp cloth and a little washing up liquid, if required. Never immerse the scale in water or rinse it under running water. Do not use aggressive or abrasive cleaning products.
- Storage: Do not place any objects on the scale when not in use.
- The scale is not intended to be used for commercial purposes.
- Remove transport safety catch.
- Never step onto the outer edge of the scales with one foot as this could cause the scales to tip!
- Keep packaging material away from children!
- Dispose of packaging in an environmentally friendly manner

### Warranty/ service

In case of a claim under the warranty please contact your local dealer or the local representation which is mentioned in the list "service international".  
In case of returning the unit please add a copy of your receipt and a short report of the defect.

The following warranty terms shall apply:

- The warranty period for BEURER products is either 5 years or - if longer- the country specific warranty period from date of Purchase.  
In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- Repairs (complete unit or parts of the unit) do not extend the warranty period.
- The warranty shall not be valid for damages because of
  - improper treatment, e.g. nonobservance of the user instructions.
  - repairs or tampering by the customer or unauthorised third parties.
  - transport from the manufacturer to the consumer or during transport to the service centre.
  - The warranty shall not be valid for accessories which are subject to normal wear and tear (cuff, batteries etc.).
- Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

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