


| REF        | Désignation / Omschrijving<br>Bezeichnung / Designation | Mesures / Maten /<br>Maßnahmen /<br>Measurements |
|------------|---|--|
| RFB11650   | AMBULATION BELT   | 14-22cm / 82cm-132cm                             |
| RFW16400   | AMBULATION WRIST  | 14-22cm  |
| RFA17400   | AMBULATION ANKEL  | 18-28cm  |
| RFK01111   | MAGNET KEY (M)  |  |
| RFK01222   | MAGNET BUTTON + PIN (M)                                 |  |
| RFK01201   | MAGNET PIN  |  |
| RFK01202   | MAGNET BUTTON   |  |
| RFK00100   | STEEL KEY (RL)  |  |
| RFK00433   | STEEL BUTTON + PIN (RL)                                 |  |
| RFK00300   | STEEL PIN   |  |
| RFK00400   | STEEL BUTTON  |  |
| REF + M =  | BELT + MAGNET LOCKS                                     |  |
| REF + RL = | BELT + STEEL LOCKS                                      |  |

REF + M =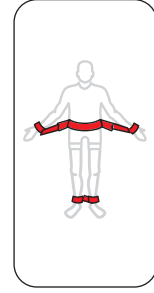

REF + RL =

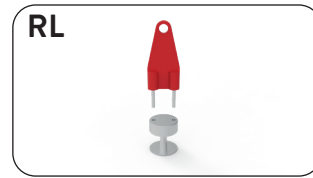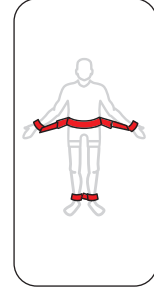Lire la notice (IFU) / Lezen IFU / Lesen IFU / Read IFU

Regarder la Video / Bekijk Video / Shau Video / Watch Video

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## Indications

1. RENOL products are used for a medical purpose to ensure the safety and security of the patient.
2. RENOL products are only used under medical prescription.
3. The products must be used by trained staff following the RENOL's instructions for safe use and according to the establishment's safety rules.

## Precautions of use

4. To avoid any accident and in order to ensure the person's comfort and safety, it is essential to read the instruction leaflets carefully and understand them before operating belts. All users follow the instructions and procedures, patients' safety depends on using belts correctly.
5. Ensure that a RENOL product is not damaged before use, all products must be clean and in good condition.
6. We recommend monitoring the immobilised patient frequently; restless and combative patients must be under constant observation.
7. Ensure that RENOL products do not come into contact with injured skin, do not obstruct breathing nor impede blood pressure.
8. Always use the correct size: too small or too large may compromise safety or comfort. Measure the patient's waist size, wrist and ankle, select the appropriate RENOL product.
9. Remove all sharp objects from the patient (jewellery, watches, glasses, etc.).
10. When supporting or immobilising the patient, don't forget the patient's ability to turn over.
11. Ensure that the RENOL key, part number: RFK00100 (RL) and RFK01111 (M) is always available. Use only RENOL locks.
12. In an emergency, if the key is not available or problem with lock system, cut the strap.
13. Before use, check operation of the magnet (M) lock system, check correct operation of the RFK01202 Button with the key RFK01111 and pin RFK01101. We recommend replacing the Button RFK01202 after 2 years. The lifetime may be shorter or longer, depending on use.
14. Before use, check operation of the mechanical lock system (RL), check correct operation of the RFK00400 Button with the key RFK00100 and long pin RFK00300. We recommend replacing the Button RFK00400 every 3 years. The lifetime may be shorter or longer, depending on use.
15. The magnetic key (M) must be placed 10 cm minimum from a pacemaker or any other medical system that could be disturbed by a magnet.

## Maintenance and materials

20. Follow the washing instructions marked on the products.
  21. Close Velcro fastenings before washing and drying to prevent them wearing.
  22. Detergents: avoid using chlorine.
  23. Temperature: maximum temperature 95° C.
  24. Drying: delicate cycle at low temperature, about 5% shrinkage possible.
  25. Remove all locking systems before washing and drying.
  26. Never immerse the magnetic (M) and mechanical closures (RL) in a liquid.
  27. Material: 100% Terylene (polyester).
  28. The raw materials are treated with fire retardant and anti bacterial.
  29. 2 years warranty, this warranty does not cover damage resulting from normal wear and tear.
  - All our products are CE-approved - Class I medical devices.
- For more information, contact your support specialist, your distributor or [www.renolcare.com](http://www.renolcare.com)