


REF	Désignation / Omschrijving Bezeichnung / Designation	Mesures / Maten / Maßnahmen / Measurements
RFB09115P	ROLLBELT with 2 side straps S/M-14cm + PELVIC	60-110cm
RFB09140P	ROLLBELT with 2 side straps L -14cm + PELVIC	85-135cm
RFB10200	ROLLBELT extender 20cm	20cm
RFB10300	ROLLBELT extender 40cm	40cm
RFW16100	WRIST HOLDERS 14-22cm	14-22cm
RFY21000	SHOULDER BELT S/M	
RFY21500	SHOULDER BELT L	
RFK01111	MAGNET KEY (M)	
RFK01222	MAGNET BUTTON + PIN (M)	
RFK01201	MAGNET PIN	
RFK01202	MAGNET BUTTON	
RFK00100	STEEL KEY (RL)	
RFK00433	STEEL BUTTON + PIN (RL)	
RFK00300	STEEL PIN	
RFK00400	STEEL BUTTON	
REF +M=	BELT + MAGNET LOCKS	
REF +RL=	BELT + STEEL LOCKS	

REF + M =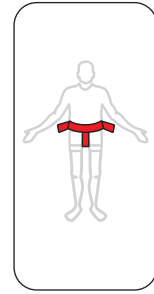

REF + RL =

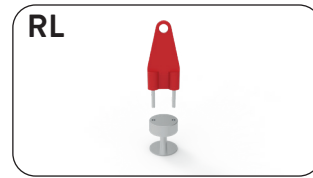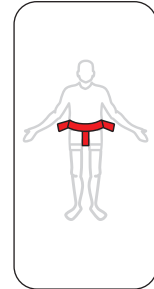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Indications

1. RENOL products are used for a medical purpose to ensure the safety and security of the patient in bed, in stretcher and in the chair.
2. RENOL products are only used under medical prescription.
3. The products must be used by trained staff following the RENOL's instructions for safe use and according to the establishment's safety rules.

Precautions of use

4. To avoid any accident and in order to ensure the person's comfort and safety, it is essential to read the instruction leaflets carefully and understand them before operating belts. All users follow the instructions and procedures, patients' safety depends on using belts correctly.
5. Ensure that a RENOL product is not damaged before use, all products must be clean and in good condition.
6. We recommend monitoring the immobilised patient frequently; restless and combative patients must be under constant observation.
7. Ensure that RENOL products do not come into contact with injured skin, do not obstruct breathing nor impede blood pressure.
8. Always use the correct size: too small or too large may compromise safety or comfort. Measure the patient's waist size, wrist and ankle, select the appropriate RENOL product.
9. Remove all sharp objects from the patient (jewellery, watches, glasses, etc.).
10. Only use products RENOL on suitable support. Never use RENOL products as a safety belt in vehicles.
11. When supporting or immobilising the patient in a bed or in a chair, don't forget the patient's ability to turn over.
12. Connect RENOL products to a solid part of the bed, chair or stretcher. The straps must be secured; they cannot slide or change position; the fastening point of the strap on the bed must not damage the product (sharp edge, etc.).
13. The straps are placed on a part of the bed that can move to match possible movements of the bed base; this also applies to stretchers, wheelchairs and chairs where certain parts can be modified or tipped. Check the straps every time the position changes. The abdominal strap attached to the bed frame should be strapped.
14. When using the abdominal belt, always use the pelvic belt RFP22000 and lateral belts RFE24000 unless otherwise specified by the prescribing doctor.
15. Ensure that the RENOL key, part number: RFK00100 (RL) and RFK01111 (M) is always available. Use only RENOL locks.
16. In an emergency, if the key is not available or problem with lock system, cut the strap close to the frame.
17. Before use, check operation of the magnet (M) lock system, check correct operation of the RFK01202 Button with the key RFK01111 and pin RFK01101. We recommend replacing the Button RFK01202 after 2 years. The lifetime may be shorter or longer, depending on use.
18. Before use, check operation of the mechanical lock system (RL), check correct operation of the RFK00400 Button with the key RFK00100 and long pin RFK00300. We recommend replacing the Button RFK00400 every 3 years. The lifetime may be shorter or longer, depending on use.
19. The magnetic key (M) must be placed 10 cm minimum from a pacemaker or any other medical system that could be disturbed by a magnet.

Maintenance and materials

20. Follow the washing instructions marked on the products.
 21. Close Velcro fastenings before washing and drying to prevent them wearing.
 22. Detergents: avoid using chlorine.
 23. Temperature: maximum temperature 95°C.
 24. Drying: delicate cycle at low temperature, about 5% shrinkage possible.
 25. Remove all locking systems before washing and drying.
 26. Never immerse the magnetic (M) and mechanical closures (RL) in a liquid.
 27. Material: 100% Terylene (polyester).
 28. The raw materials are treated with fire retardant and anti bacterial.
 29. 2 years warranty, this warranty does not cover damage resulting from normal wear and tear.
- All our products are CE-approved - Class I medical devices.
 For more information, contact your support specialist, your distributor or www.renolcare.com