|   |   |
|---|---|
| <b>DE</b> Shiatsu-Massagekissen<br>Gebrauchsanweisung .....         | <b>IT</b> Cuscino massaggiante shiatsu<br>Istruzioni per l'uso..... |
| <b>EN</b> Shiatsu massage cushion<br>Instruction for Use .....      | <b>TR</b> Shiatsu masaj yastığı<br>Kullanma Talimatı.....           |
| <b>FR</b> Coussin de massage Shiatsu<br>Mode d'emploi.....          | <b>RU</b> Массажная подушка шиацу<br>Инструкция по применению.....  |
| <b>ES</b> Cojín de masaje Shiatsu<br>Instrucciones para el uso..... | <b>PL</b> Shiatsu – poduszka do masażu<br>Instrukcja obsługi.....   |

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Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

## **WARNING**

- The device is intended only for domestic/private use, not for commercial use.
- This device can be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely and are aware of the consequent risks of use.
- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.
- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.
- The device has a hot surface. Persons insensitive to heat must be careful when using the device.
- Do not insert any objects into device openings, or in the moving parts. Ensure that the moving parts can move freely at all times.
- Under no circumstances should you open or repair the device yourself, as faultless functionality can no longer be guaranteed thereafter. Failure to comply will invalidate the warranty.

### **Dear customer,**

Thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, baby and air. Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

With kind regards,  
Your Beurer team

### **Contents**

|   |    |                                    |    |
|---|----|------------------------------------|----|
| 1. Items included in the package .....  | 11 | 8. Initial use.....                | 14 |
| 2. Explanation of symbols .....         | 11 | 9. Usage .....                     | 15 |
| 3. About Shiatsu massage .....          | 12 | 10. Cleaning and maintenance.....  | 15 |
| 4. Getting to know your appliance ..... | 12 | 11. In the event of problems.....  | 16 |
| 5. Proper use .....                     | 12 | 12. Disposal.....                  | 16 |
| 6. Warnings and safety notes .....      | 13 | 13. Technical specifications ..... | 16 |
| 7. Device description .....             | 14 | 14. Warranty/service.....          | 16 |

## 1. Items included in the package

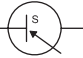
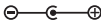
Check that the device packaging has not been tampered with and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

- 1 x Shiatsu massage cushion
- 1 x Removable cover
- 1 x Power supply unit
- 1 x These operating instructions

## 2. Explanation of symbols

The following symbols appear in the instructions for use and on the device:

|   |   |
|---|---|
|    | <b>WARNING</b><br>Warning notice indicating a risk of injury or damage to health.             |
|    | <b>IMPORTANT</b><br>Safety note indicating possible damage to the device/accessory.           |
|    | <b>Note</b><br>Note on important information.   |
|    | The unit is double-protected and thus corresponds to protection class 2.                      |
|    | Only use indoors.   |
|     | Dispose of packaging in an environmentally friendly manner                                    |
|    | Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE |
|   | Manufacturer  |
|  | This product satisfies the requirements of the applicable European and national directives.   |
|  | The cover is machine-washable. Set the washing machine to a delicate cycle at 30°C.           |
|  | Do not bleach   |
|  | Do not tumble dry   |
|  | Do not iron   |
|  | Do not dry clean  |
| <b>IP20</b>   | Protected against solid foreign objects 12.5 mm in diameter and larger                        |

|   |   |
|---|---|
|   | Short-circuit-proof safety isolating transformer  |
|  | The products demonstrably meet the requirements of the Technical Regulations of the EAEU. |
|  | Direct current  |
|  | Switching mains part  |
|  | Polarity  |
|  | Energy efficiency level 6   |

### 3. About Shiatsu massage

Shiatsu is a form of body massage which was originally developed in Japan around 100 years ago. It is based on ideas anchored in traditional Chinese medicine, in particular the meridian system, i.e. the paths in the human body along which energy is conveyed.

The aim of any Shiatsu massage is to promote the physical, emotional and mental wellbeing of the person being treated. To do this, energy blocks and bottlenecks in the meridians need to be dissolved and the self-regulating ability of the body needs to be stimulated.

Shiatsu therapists achieve this by exercising pressure in flowing movements along the meridians. Although Shiatsu can be literally translated as “finger pressure”, practitioners also use the heels of their hands, their elbows and knees as well as their fingers. In the process, the therapist will use his or her full body weight in order to generate the required pressure.

### 4. Getting to know your appliance

Your Shiatsu massage device has rotating massage heads which imitate the pressure and kneading motions of a Shiatsu massage.

With the aid of this electrical Shiatsu massage device, you can give yourself or somebody else an effective and pleasant massage. Massages can be both relaxing and stimulating, and are popular for treatment of stiff or tense muscles, pains and fatigue. The device delivers powerful and intensive massages for neck, legs, arms and back.

The device is for **private use only and is not intended** for medical or commercial purposes.

In addition, the Shiatsu massage device offers the following advantages:

- easy operation with manual control switch,
- covers can be removed for washing,
- heat and lighting function,
- 2 massage directions.

### 5. Proper use

This device is intended only for the massaging of individual parts of the human body. It cannot replace medical treatment. Do not use the massage device if any of the following warnings apply to you. Please consult your doctor if you are unsure whether or not the massage device is suitable for you.

#### WARNING

Use of the massage device is prohibited in the following cases:

- On animals.
- Not with changes caused by disease or injury in the back region (e.g. disc problems, open wounds),
- Never use the massage device to massage the heart region of persons fitted with a pacemaker. Always consult a doctor before using the massage device on other parts of the body of a person fitted with a pacemaker.
- Never use the massage device on swollen, burned, inflamed or damaged skin or body parts.

- The massage device must not be used on lacerated wounds, capillaries, varicose veins, acne, coup-erose, herpes or other skin diseases.
- The massage device must not be used during pregnancy.
- Do not use the massage device on the face (eyes), on the larynx or on other particularly sensitive parts of the body.
- Do not use the device for foot massage.
- Never use the device while you are asleep.
- Never use the device after taking medication or consuming alcohol (reduced levels of awareness).
- Not in the car.

The device has a hot surface. Persons who are sensitive to heat must be careful when using the device.

You should consult your doctor before using the massage device if:

- you suffer from a serious illness or are recovering from an operation on the upper body,
- you are fitted with a cardiac pacemaker, implant or other aid,
- Thromboses,
- Diabetes,
- in case of pains of unclarified origin.

The device is only intended for the purpose described in these operating instructions. The manufacturer cannot be held liable for damage occurring as a result of inappropriate or careless use.

## 6. Warnings and safety notes

Please read these operating instructions carefully. Failure to observe the information provided below can lead to personal injury or material damage. Keep these instructions in a safe place where they can also be read by other persons using the device. Always pass on these instructions together with the appliance if you sell it or pass it on.

### **WARNING**

Keep children away from the packaging materials. Risk of suffocation.

### **Electric shock**

### **WARNING**

Just like any other electrical appliance, this massage device must be handled with due care and attention in order to prevent the risk of electric shock.

For this reason, the device must only be operated as follows:

- only with the supplied power supply unit at the mains voltage indicated on the power supply unit,
- do not use the device if damage to the device itself or any of its accessories is evident,
- never use the appliance during a thunderstorm.

In the event of a defect or malfunction, switch the device off immediately and disconnect it from the power supply. Never pull on the power cable or on the appliance in order to disconnect the power supply from the mains outlet. Never hold or carry the device by the power cable. Keep the cables away from hot surfaces. Do not pinch, kink or twist the cables. Do not stick in any pins, needles or sharp objects.

Make sure that the massage device, the switch, the power supply unit and the cable do not come into contact with water, damp or other liquids. Therefore, use the device only in dry interior rooms (e.g. never in the bath, sauna).

Never try to retrieve the device if it falls into water. Disconnect the plug immediately from the mains outlet. Do not use the device if damage to the device itself or any of its accessories is evident. Do not subject the device to any impact and do not drop it.

### **Repairs**

### **WARNING**

- Repairs to electrical appliances must only be carried out by qualified persons. Incorrect repairs can lead to considerable danger for the user. In the event of repairs, please contact our customer service or an authorized dealer.
- The zip fastener on the Shiatsu massage device must not be opened. It is only fitted for reasons relating to the manufacturing of the device.
- If the power supply unit or cord is damaged, it must be replaced by an authorised dealer.

## Fire hazard

### WARNING

There is a risk of fire if the device is used incorrectly or if it is not used according to these operating instructions.

For this reason, the massage device must only be operated as follows:

- Never use the device under covers, such as blankets or pillows.
- Never use the device in the vicinity of petrol or other easily flammable materials.

## Handling


### CAUTION

Switch the device off and disconnect the power supply after every use and before every cleaning.

- Never insert any objects into openings on the device, and never insert anything into the rotating parts. Make sure that all moving parts can move freely at all times.
- Never sit or stand on the moving parts of the device with all your weight and do not place objects on the device.
- Keep the device away from high temperatures.


## 7. Device description

### Shiatsu massage pillow (front)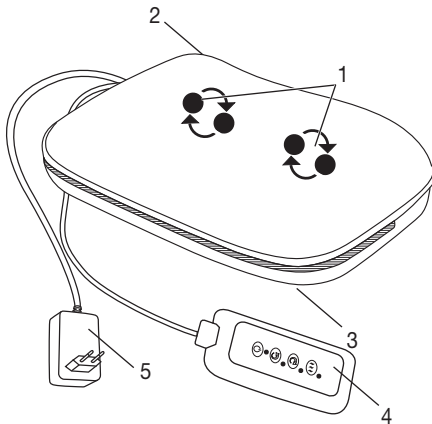

1. 4 heated and illuminated massage heads, rotating in pairs
2. Zip fastening (to attach and remove the cover)
3. Hook-and-loop fastener (to attach and remove the cover)
4. Hand switch
5. Power supply unit

### Hand switch

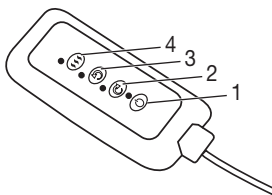1. Button
2. Button Clockwise rotation
3. Button Anti-clockwise rotation
4. Button Heating

## 8. Initial use

- Remove the packaging
- Check the device, power supply and cables for damage.
- The device can be used with or without a cover.
- Connect the power supply plug to the massage device.
- Route the cable safely so that there is no risk of anyone tripping over it.

## 9. Usage


To start with, gently press the part of your body you wish to massage against the massage device. Then make sure that you are in a comfortable position before gradually shifting your weight onto the massage device. The massage must be pleasant and relaxing at all times. Stop the massage or change your position or the contact pressure if you find it is painful or unpleasant.

- Do not trap or squeeze any part of your body, especially feet, hands and fingers, between the rotating massage heads and their mountings in the device.
- Switch on the massager with the  button. Next press the buttons  or  to activate the desired massage direction. You can switch on the heat function separately by pressing the  button. This can also be activated without the massage function. Press the  button again to switch the device off again.
- The device features an automatic switch-off function, which is set to the maximum operating duration of 15 minutes.

### Note


- The device is more comfortable if used with the included cover. It is also possible to use the massager without the cover. This is generally considered to be a “harder” massage.
- Do not use the massage device for longer than 15 minutes. Otherwise, using the device for longer can over-stimulate the muscles and cause tension instead of relaxation.
- The device has an automatic shut-off mechanism set to a maximum operating period of 15 minutes.

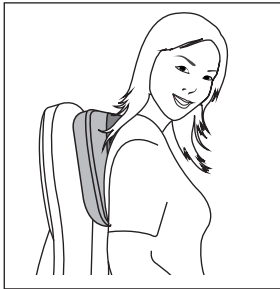Back massage: upper back area

Back massage: lower back area (lumbar vertebrae)

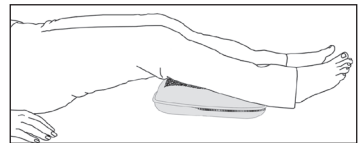Leg massage

## 10. Cleaning and maintenance

### Cleaning

### WARNING

- Before cleaning disconnect the power supply unit from the mains socket and disconnect it from the device.
- Only clean the appliance in the manner specified. Fluid must never be allowed to enter into the device or accessories.
- Smaller stains can be removed using a cloth or moist sponge together with a little liquid detergent for delicate fabrics if necessary. Do not use any cleaning agents containing solvents.
- The removable cover is machine-washable at 30°C. Please follow the cleaning and care symbols on the label which is sewn into the cover.
- Do not use the device again until it is completely dry.

### Storage

If you do not use the device for longer periods, we recommend storing it in the original packaging in a dry place without weighting it down.

## 11. In the event of problems

| Problem                                  | Cause  | Remedy  |
|--|--|---|
| Massage heads rotating at reduced speed. | The load on the massage heads is too great.  | Release the massage heads.  |
| Massage heads not moving.                | Appliance not connected to the power supply. | Connect the plug to the mains supply and switch on the appliance. |

## 12. Disposal

### IMPORTANT

For environmental reasons, do not dispose of the device in the household waste at the end of its service life. Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

## 13. Technical specifications

|                               |                                  |
|-------------------------------|----------------------------------|
| Mains operation:              |                                  |
| Input                         | 100-240 V~, 50/60 Hz, 0.5 A      |
| Output                        | 12.0 V DC $\equiv$ 1.0 A, 12.0 W |
| Average active efficiency     | $\geq$ 84.44 %                   |
| Efficiency at low load (10 %) | $\geq$ 76.40 %                   |
| No-load power consumption     | $\leq$ 0.093 W                   |
| Dimensions                    | approx. 40.0 x 27.0 x 12.0 cm    |
| Weight                        | approx. 1.47 kg                  |

## 14. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

**The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.**

**The warranty shall apply without prejudice to any mandatory statutory provisions on liability.**

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

**If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.**

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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