

## Are you using the right mushroom valve and washer for your Propulse Ear Irrigator?

### Mushroom Valves



The mushroom valve on the left is for the Propulse G5 & PP17, and the one on the right is for the older model Propulse units (II, III & NG). As you can see, they differ in a couple of key areas:

- The G5 / PP17 mushroom valve has a longer central spoke
- The G5 / PP17 mushroom valve has wider outside spokes
- The G5 / PP17 mushroom valve is a brighter white in colour

### Washers / 'O-rings'



The washer on the left is for the Propulse G5 & PP17. The most obvious difference is that it is made of red rubber instead of black. It is also slightly thicker, and smaller in diameter.

### Key Points:

- When the mushroom valve is fitted correctly, it should lift slightly with the tension of your finger, when pressed.
- Be careful when changing the valves and washers, as any damaged spokes will cause the tank to leak.
- Be sure to check the valves and washers prior to each use, as they do wear through normal use and may need changing.