

**MOTOmed®**  
next generation

**Product overview**


No. 38 - valid from 04/2020

International

# MOTOMed MOTOMed MOTOMed



NEW

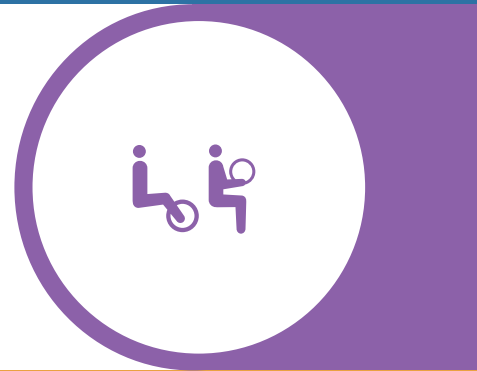

NEW



NEW



MOTOmed

# MUVI

Leg and/or arm/upper body trainer with simultaneous movement

04

MUVI




MOTOmed

# LOOP

Leg or arm/upper body trainer

06

LOOP

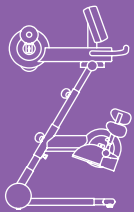


MOTOmed

# GRACILE12

Leg or arm/upper body trainer for children

22

GRACILE12

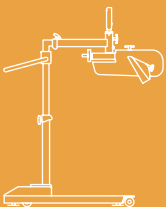


MOTOmed

# LAYSON

Leg or arm/upper body trainer

26

LAYSON

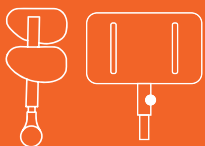


MOTOmed

# ACCESSORIES

Accessories and unique product configurations

40

ACCESSORIES

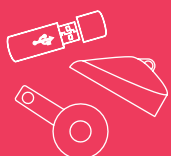

MOTOmed

# UPGRADES

58

UPGRADES



MOTomed®  
MUVI



## Uniquely simultaneous

MOTomed muvi users can train either **separately or simultaneously with their legs, arms and upper body**. The speed and resistance of the leg and arm/upper body trainer can be separately adjusted. The holistic MOTomed movement can effectively improve muscle strength and mobility, as well as ideally strengthen the cardiovascular system.

Non-binding fig. of MOTomed muvi




# MOTomed® muvi

## Leg and/or arm/upper body trainer with simultaneous movement



Order number: 300.000

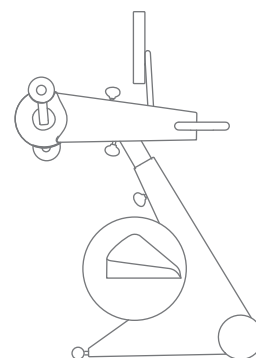

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 95–108 × 60 × 124–135 cm                           |
| Weight:                                                                                          | 51 kg                                              |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 12.1" / 30.7 cm                                    |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

### Basic equipment

- Simultaneous training, separately adjustable training modes for leg and arm/upper body trainer
- Colour touch display (12.1"), rotatable, inclinable
- Plastic-coated safety foot shells with fastening straps
- Plastic-coated handle
- Rotatable arm/upper body trainer, adjustable in height and distance
- Plastic-coated handles with quick release system
- 2-stage pedal radius adjustment (7 cm or 12.5 cm)
- Sturdy all-metal construction, high quality and stable, with telescopic front stand
- Easily movable with large castors (Ø 15 cm)
- Integrated operating film
- Serial interface, USB interface
- Colour: white/chrome/black

### Functions

- Passive, Assistive and Active Training
- Simultaneous training of leg and arm/upper body trainer, separately adjustable (concurrent, simultaneous training)
- Individual arrangement of operating keys possible
- Electronic foot insertion aid
- Speed from 1 to 60 rpm (passive)
- Resistance from 0 to 20 (active)
- Motor power from 1 to 10 (passive)
- Presetting of therapy time from 0 to 120 minutes
- MovementProtection & SpasmControl
- Safety stop
- SymmetryTraining of leg and arm/upper body trainer
- Quiet, gentle and harmonious cycle movement (SmoothDriveSystem)
- Gentle starting and stopping of the movement
- Therapy and motivation programs
- Detailed biofeedback during and after training
- Export of training data and import of own programs
- Games & videos
- Slideshow via USB (delivered without USB stick)
- Language selection

### Frequently used accessories (More accessories from [page 40](#) onwards)

| Designation                                       | Order No. | Page                    |
|---------------------------------------------------|-----------|-------------------------|
| Leg guides with plastic-coated calf shells (pair) | 303.000   | <a href="#">p. 45</a> ⓘ |
| »QuickFix« foot fastening system (pair)           | 265.004   | <a href="#">p. 46</a> ⓘ |
| Hand fixation by wrist cuff (piece)               | 562.000   | <a href="#">p. 48</a> ⓘ |



# MOTOmed® LOOP EDITION



## MOTOmed loop on top

With the new and innovative height adjustment, users can adjust the height of the MOTOMed loop without using tools. This feature allows the leg and arm trainer to be optimally adapted to the user, enabling patients to use the MOTOMed loop from an active wheelchair, electric wheelchair, care wheelchair, a seat shell or even a chair.



Non-binding fig. of MOTOMed loop.la

# MOTOMed® loop edition

The MOTOMed loop models stand for a future-oriented movement therapy in a sitting position. The innovative height adjustment, modern hygiene system for professional use, a minimalistic design with a new feel, optimized dimensions and proportions, and intuitive operation have all contributed to a revamp of the MOTOMed devices to provide users with a progressive training experience.

## Basic equipment

- Height adjustment of leg or arm/upper body trainer adjustable by 15 cm without tools. The pedal axle height of the leg trainer is height adjustable in the range 30 to 45 cm (measured from floor).
- Colour touch display (7"), inclinable, retractable
- Sturdy metal construction, high quality and stable
- 2-stage pedal radius adjustment (7 cm or 12.5 cm)\*<sup>1</sup>
- Closed plastic body (PC/ABS), thus easy to clean and disinfect
- Easily movable with large castors (Ø 13 cm)
- Soft Grip transport handle
- USB interface
- Device stand width 38.5 cm
- Colour: white/chrome/black

\*<sup>1</sup> Not valid for MOTOMed loop models (.a)

\*<sup>2</sup> Not valid for MOTOMed loop light models

\*<sup>3</sup> Only valid for MOTOMed loop models (.la)

\*<sup>4</sup> Only valid for MOTOMed loop kidz models

## Functions


- Passive, Assistive and Active Training
- Change from leg to arm training via Touch\*<sup>3</sup>
- Electronic foot insertion aid
- Passive speed from 1 to 60 rpm (90 rpm for MOTOMed loop parkinson)
- Resistance from 0 to 20 (active)
- Motor power from 1 to 10 (passive)
- Adjustable flywheel mass\*<sup>2</sup>
- Presetting of therapy time from 0 to 120 minutes
- MovementProtection & SpasmControl
- Safety Stop
- SymmetryTraining
- Quiet, gentle and harmonious cycle movement (SmoothDriveSystem)
- Gentle starting and stopping of the movement
- Therapy and motivation programs\*<sup>2</sup>
- Detailed biofeedback during and after training
- Presetable training parameters
- Export of training data and import of own training programs\*<sup>2</sup>
- Games\*<sup>2</sup>
- Slideshow via USB (delivered without USB stick)\*<sup>2</sup>
- Child-friendly software\*<sup>4</sup>
- Language selection

## Uniquely upgradeable

The individual MOTOMed loop devices can be adapted to different requirements by means of an upgrade bundle. Detailed information on the available MOTOMed loop extensions can be found in the [upgrade table on page 60/61](#).



MOTOMed loop.l

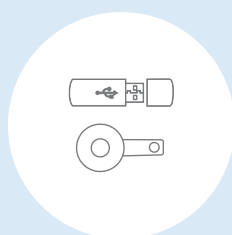

Upgrade Bundle



MOTOMed loop.la\*

\*Example of upgrade

# MOTomed® loop model overview


|  | Order No. | Designation/Description                                                                                    | Upgradeable | Page |
|--|-----------|------------------------------------------------------------------------------------------------------------|-------------|------|
|  | 260.030   | <b>MOTomed loop.la</b><br>Leg or arm/upper body trainer                                                    | ✓           | 9    |
|  | 260.010   | <b>MOTomed loop.l</b><br>Leg trainer                                                                       | ✓           | 10   |
|  | 260.020   | <b>MOTomed loop.a</b><br>Arm/upper body trainer                                                            | ✓           | 11   |
|  | 260.060   | <b>MOTomed loop p.la (Parkinson)</b><br>Leg or arm/upper body trainer for Parkinson's patients with 90 rpm | ✓           | 12   |
|  | 260.040   | <b>MOTomed loop p.l (Parkinson)</b><br>Leg trainer for Parkinson's patients with 90 rpm                    | ✓           | 13   |
|  | 260.120   | <b>MOTomed loop light.la</b><br>Leg and/or arm/upper body trainer, only with basic functions               | ✓           | 14   |
|  | 260.100   | <b>MOTomed loop light.l</b><br>Leg trainer only with basic functions                                       |             | 15   |
|  | 260.110   | <b>MOTomed loop light.a</b><br>Arm/upper body trainer only with basic functions                            |             | 16   |
|  | 260.090   | <b>MOTomed loop kidz.la</b><br>Leg or arm/upper body trainer for children and adolescents                  | ✓           | 17   |
|  | 260.070   | <b>MOTomed loop kidz.l</b><br>Leg trainer for children and adolescents                                     | ✓           | 18   |
|  | 260.080   | <b>MOTomed loop kidz.a</b><br>Arm/upper body trainer for children and adolescents                          | ✓           | 19   |
|  | 260.039   | <b>MOTomed loop.la prof</b><br>Leg or arm/upper body trainer for professional use in hospitals             | ✓           | 21   |

# MOTomed® loop.la

## Leg or arm/upper body trainer



Order number: 260.030

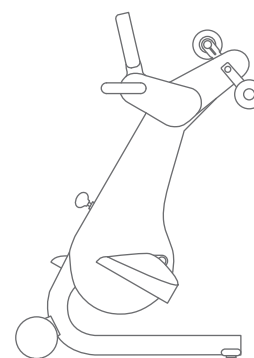

### Basic equipment (For all details, see equipment and functions [page 7](#))

- Plastic-coated safety foot shells with fastening straps (pair, incl. pedal cranks)
- Soft Grip handles (pair, incl. pedal cranks)

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 70 × 60 × 107–122 cm                               |
| Weight:                                                                                          | 36 kg                                              |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

### Optional product configuration (cannot be changed later)

| Designation                              | Order No. | Page                    |
|------------------------------------------|-----------|-------------------------|
| Gas spring support for height adjustment | 265.002   | <a href="#">p. 57</a> ⓘ |
| Special stand with reduced stand height  | 265.001   | <a href="#">p. 57</a> ⓘ |
| Activation of serial interface           | 265.500   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#))

|                                                   |         |                         |
|---------------------------------------------------|---------|-------------------------|
| Leg guides with plastic-coated calf shells (pair) | 303.000 | <a href="#">p. 45</a> ⓘ |
| »QuickFix« foot fastening system (pair)           | 265.004 | <a href="#">p. 46</a> ⓘ |
| Pedal radius quick adjustment (leg trainer, pair) | 507.020 | <a href="#">p. 46</a> ⓘ |
| Hand fixation by wrist cuff (piece)               | 562.000 | <a href="#">p. 48</a> ⓘ |
| Wheelchair stabilizer                             | 100.000 | <a href="#">p. 54</a> ⓘ |

### These upgrades are possible with MOTomed loop.la (More information from [page 58](#) onwards)


|         | loop.l | loop.a | loop light.la | loop light.l | loop light.a | loop p.la | loop p.l | loop kidz.la | loop kidz.l | loop kidz.a | loop.la prof |
|---------|--------|--------|---------------|--------------|--------------|-----------|----------|--------------|-------------|-------------|--------------|
| loop.la | ✓      | ✓      | ✓             |              |              | ✓         | ✓        | ✓            | ✓           | ✓           |              |

# MOTomed® loop.l

## Leg trainer



Order number: 260.010

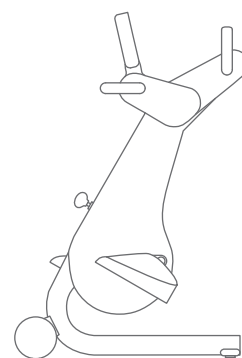

### Basic equipment (For all details, see equipment and functions [page 7](#))

Plastic-coated safety foot shells with fastening straps (pair, incl. pedal cranks)

Ergonomic Soft Grip handles (pair) »cornets«

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 70 × 60 × 107–122 cm                               |
| Weight:                                                                                          | 35 kg                                              |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

### Optional product configuration (cannot be changed later)

| Designation                              | Order No. | Page                    |
|------------------------------------------|-----------|-------------------------|
| Gas spring support for height adjustment | 265.002   | <a href="#">p. 57</a> ⓘ |
| Special stand with reduced stand height  | 265.001   | <a href="#">p. 57</a> ⓘ |
| Activation of serial interface           | 265.500   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#))


|                                                   |         |                         |
|---------------------------------------------------|---------|-------------------------|
| Leg guides with plastic-coated calf shells (pair) | 303.000 | <a href="#">p. 45</a> ⓘ |
| »QuickFix« foot fastening system (pair)           | 265.004 | <a href="#">p. 46</a> ⓘ |
| Pedal radius quick adjustment (leg trainer, pair) | 507.020 | <a href="#">p. 46</a> ⓘ |

### These upgrades are possible with MOTomed loop.l (More information from [page 58](#) onwards)

|        | loop.la | loop.a | loop light.la | loop light.l | loop light.a | loop p.la | loop p.l | loop kidz.la | loop kidz.l | loop kidz.a | loop.la prof |
|--------|---------|--------|---------------|--------------|--------------|-----------|----------|--------------|-------------|-------------|--------------|
| loop.l | ✓       |        | ✓             |              |              | ✓         | ✓        | ✓            | ✓           |             |              |

# MOTOMed® loop.a

## Arm/upper body trainer

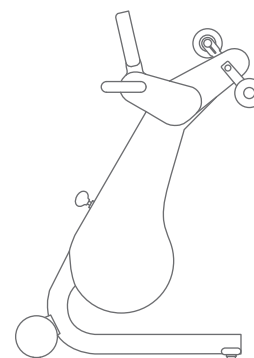

Order number: 260.020

### Basic equipment (For all details, see equipment and functions [page 7](#))

Soft Grip handles (pair, incl. pedal cranks)

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 70 × 60 × 107-122 cm                               |
| Weight:                                                                                          | 31 kg                                              |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100-240 V~ / max. 120 VA, mains frequency: 47-63 Hz, class II equipment / Type BF |                                                    |

### Optional product configuration (cannot be changed later)

| Designation                              | Order No. | Page                    |
|------------------------------------------|-----------|-------------------------|
| Gas spring support for height adjustment | 265.002   | <a href="#">p. 57</a> ⓘ |
| Special stand with reduced stand height  | 265.001   | <a href="#">p. 57</a> ⓘ |
| Activation of serial interface           | 265.500   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#))

|                                         |         |                         |
|-----------------------------------------|---------|-------------------------|
| Hand fixation by wrist cuff (piece)     | 562.000 | <a href="#">p. 48</a> ⓘ |
| Tetra handles (pair)                    | 555.000 | <a href="#">p. 50</a> ⓘ |
| Vertical handles with hand rests (pair) | 557.000 | <a href="#">p. 51</a> ⓘ |

### These upgrades are possible with MOTOMed loop.a (More information from [page 58](#) onwards)


|        | loop.la | loop.l | loop light.la | loop light.l | loop light.a | loop p.la | loop p.l | loop kidz.la | loop kidz.l | loop kidz.a | loop.la prof |
|--------|---------|--------|---------------|--------------|--------------|-----------|----------|--------------|-------------|-------------|--------------|
| loop.a | ✓       |        | ✓             |              |              | ✓         |          | ✓            |             | ✓           |              |

# MOTomed® loop p.la

## Leg or arm/upper body trainer (90 rpm)



Order number: 260.060

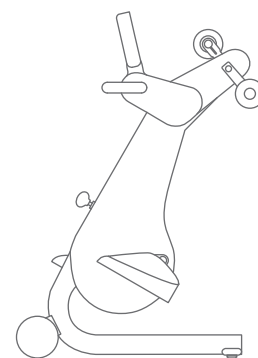

### Basic equipment (For all details, see equipment and functions [page 7](#))

Plastic-coated safety foot shells with fastening straps (pair, incl. pedal cranks)  
Soft Grip handles (pair, incl. pedal cranks)

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 70 × 60 × 107–122 cm                               |
| Weight:                                                                                          | 36 kg                                              |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

### Optional product configuration (cannot be changed later)

| Designation                              | Order No. | Page                    |
|------------------------------------------|-----------|-------------------------|
| Gas spring support for height adjustment | 265.002   | <a href="#">p. 57</a> ⓘ |
| Special stand with reduced stand height  | 265.001   | <a href="#">p. 57</a> ⓘ |
| Activation of serial interface           | 265.500   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#) onwards)

|                                                   |         |                         |
|---------------------------------------------------|---------|-------------------------|
| Leg guides with plastic-coated calf shells (pair) | 303.000 | <a href="#">p. 45</a> ⓘ |
| »QuickFix« foot fastening system (pair)           | 265.004 | <a href="#">p. 46</a> ⓘ |
| Hand fixation by wrist cuff (piece)               | 562.000 | <a href="#">p. 48</a> ⓘ |
| Vertical handles with hand rests (pair)           | 557.000 | <a href="#">p. 51</a> ⓘ |

### These upgrades are possible with MOTomed loop p.la (More information from [page 58](#) onwards)

|           | loop.la | loop.l | loop.a | loop light.la | loop light.l | loop light.a | loop p.l | loop kidz.la | loop kidz.l | loop kidz.a | loop.la prof |
|-----------|---------|--------|--------|---------------|--------------|--------------|----------|--------------|-------------|-------------|--------------|
| loop p.la | ✓       | ✓      | ✓      | ✓             |              |              | ✓        | ✓            | ✓           | ✓           |              |




# MOTomed® loop p.l

## Leg trainer (90 rpm)



Order number: 260.040

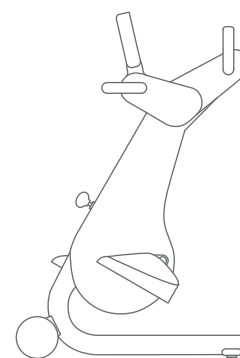

### Basic equipment (For all details, see equipment and functions [page 7](#))

Plastic-coated safety foot shells with fastening straps (pair, incl. pedal cranks)  
Ergonomic Soft Grip handles (pair) »cornets«

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 70 × 60 × 107–122 cm                               |
| Weight:                                                                                          | 35 kg                                              |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | IIa according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

### Optional product configuration (cannot be changed later)

| Designation                              | Order No. | Page                    |
|------------------------------------------|-----------|-------------------------|
| Gas spring support for height adjustment | 265.002   | <a href="#">p. 57</a> ⓘ |
| Special stand with reduced stand height  | 265.001   | <a href="#">p. 57</a> ⓘ |
| Activation of serial interface           | 265.500   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#) onwards)

|                                                   |         |                         |
|---------------------------------------------------|---------|-------------------------|
| Leg guides with plastic-coated calf shells (pair) | 303.000 | <a href="#">p. 45</a> ⓘ |
| »QuickFix« foot fastening system (pair)           | 265.004 | <a href="#">p. 46</a> ⓘ |
| Pedal radius quick adjustment (leg trainer, pair) | 507.020 | <a href="#">p. 46</a> ⓘ |
| Wheelchair stabilizer                             | 100.000 | <a href="#">p. 54</a> ⓘ |

### These upgrades are possible with MOTomed loop p.l (More information from [page 58](#) onwards)


|          | loop.la | loop.l | loop.a | loop light.la | loop light.l | loop light.a | loop p.la | loop kidz.la | loop kidz.l | loop kidz.a | loop.la prof |
|----------|---------|--------|--------|---------------|--------------|--------------|-----------|--------------|-------------|-------------|--------------|
| loop p.l | ✓       | ✓      |        | ✓             |              |              | ✓         | ✓            | ✓           |             |              |

# MOTomed® loop light.la

Leg or arm/upper body trainer



Order number: 260.120

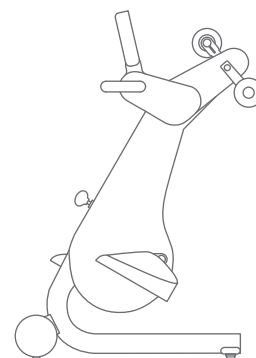

## Basic equipment (For all details, see equipment and functions [page 7](#))

Soft Grip safety foot shells with fastening straps (pair, incl. pedal cranks)

Soft Grip handles (pair, incl. pedal cranks)

## Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 70 × 60 × 107–122 cm                               |
| Weight:                                                                                          | 36 kg                                              |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | IIa according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

## Optional product configuration (cannot be changed later)

| Designation                              | Order No. | Page                    |
|------------------------------------------|-----------|-------------------------|
| Gas spring support for height adjustment | 265.002   | <a href="#">p. 57</a> ⓘ |
| Special stand with reduced stand height  | 265.001   | <a href="#">p. 57</a> ⓘ |

## Frequently used accessories (More accessories from [page 40](#) onwards)


|                                                   |         |                         |
|---------------------------------------------------|---------|-------------------------|
| Leg guides with calf shells (pair)                | 302.000 | <a href="#">p. 44</a> ⓘ |
| »QuickFix« foot fastening system (pair)           | 265.004 | <a href="#">p. 46</a> ⓘ |
| Pedal radius quick adjustment (leg trainer, pair) | 507.020 | <a href="#">p. 46</a> ⓘ |
| Hand fixation by wrist cuff (piece)               | 562.000 | <a href="#">p. 48</a> ⓘ |

## These upgrades are possible with MOTomed loop light.la (More information from [page 58](#) onwards)

|               | loop.la | loop.l | loop.a | loop light.l | loop light.a | loop p.la | loop p.l | loop kidz.la | loop kidz.l | loop kidz.a | loop.la prof |
|---------------|---------|--------|--------|--------------|--------------|-----------|----------|--------------|-------------|-------------|--------------|
| loop light.la | ✓       | ✓      | ✓      |              |              | ✓         | ✓        | ✓            | ✓           | ✓           |              |

# MOTOMed® loop light.I

## Leg trainer

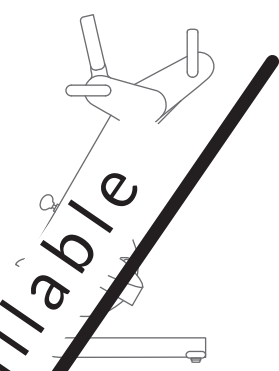

no longer available

### Basic equipment (For all details, see equipment and functions [page 7](#))

Soft Grip safety foot shells with fastening straps (pair, incl. pedal cranks)  
Ergonomic Soft Grip handles (pair) »cornets«

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 33 × 107–122 cm                                    |
| Weight:                                                                                          | 33 kg                                              |
| Maximum patient weight:                                                                          | 85 kg                                              |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, Class II equipment / Type BF |                                                    |

### Optional product configuration (cannot be changed later)

| Designation                              | Order No. | Page                    |
|------------------------------------------|-----------|-------------------------|
| Gas spring support for height adjustment | 265.002   | <a href="#">p. 57</a> ⓘ |
| Special stand with reduced stand height  | 265.001   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#) onwards)


|                                                   |         |                         |
|---------------------------------------------------|---------|-------------------------|
| Leg guides with plastic-coated half shells (pair) | 302.000 | <a href="#">p. 44</a> ⓘ |
| »QuickFix« foot fastening system (pair)           | 265.004 | <a href="#">p. 46</a> ⓘ |
| Pedal radius quick adjustment (leg trainer, pair) | 507.020 | <a href="#">p. 46</a> ⓘ |
| Wheelchair stabilizer                             | 100.000 | <a href="#">p. 54</a> ⓘ |

### These upgrades are possible with MOTOMed loop light.I (More information from [page 58](#) onwards)

|              | loop.la                                       | loop.l | loop.a | loop light.la | loop light.a | loop p.la | loop p.l | loop kidz.la | loop kidz.l | loop kidz.a | loop prod |
|--------------|-----------------------------------------------|--------|--------|---------------|--------------|-----------|----------|--------------|-------------|-------------|-----------|
| loop light.I | No upgrades possible for MOTOMed loop light.I |        |        |               |              |           |          |              |             |             |           |

# MOTomed® loop light.a

Arm/upper body trainer

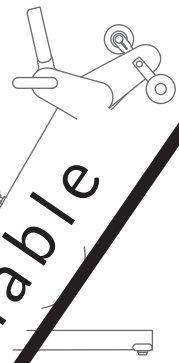

no longer available

## Basic equipment (For all details, see equipment and functions [page 7](#))

Soft Grip handles (pair, incl. pedal cranks)

## Technical data

|                                                                    |                                                          |
|--------------------------------------------------------------------|----------------------------------------------------------|
| Dimensions (L×W×H):                                                | 70 × 50 × 122 cm                                         |
| Weight:                                                            | 35 kg                                                    |
| Maximum patient weight:                                            | 135 kg                                                   |
| Display size:                                                      | 7" / 18 cm                                               |
| Classification:                                                    | Class IIa according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100-240 V~ / max. 120 VA, mains frequency: 47-63 Hz | Class II equipment / Type BF                             |

## Optional product configuration (cannot be changed later)

| Designation                              | Order No. | Page                    |
|------------------------------------------|-----------|-------------------------|
| Gas spring support for height adjustment | 265.002   | <a href="#">p. 57</a> ⓘ |
| Special stand with reduced stand height  | 265.001   | <a href="#">p. 57</a> ⓘ |

## Frequently used accessories (More accessories from [page 40](#) onwards)


|                                         |         |                         |
|-----------------------------------------|---------|-------------------------|
| Hand fixation by wrist cuff (piece)     | 562.000 | <a href="#">p. 48</a> ⓘ |
| Tetra handles (pair)                    | 555.000 | <a href="#">p. 50</a> ⓘ |
| Vertical handles with hand rests (pair) | 557.000 | <a href="#">p. 51</a> ⓘ |
| Wheelchair stabilizer                   | 100.000 | <a href="#">p. 54</a> ⓘ |

## These upgrades are possible with MOTomed loop light.a (More information from [page 58](#) onwards)

|                                               | loop.la | loop.l | loop.a | loop light.la | loop light.l | loop p.la | loop p.l | loop kidz.la | loop kidz.l | loop kidz.a | loop la pro |
|-----------------------------------------------|---------|--------|--------|---------------|--------------|-----------|----------|--------------|-------------|-------------|-------------|
| loop light.a                                  |         |        |        |               |              |           |          |              |             |             |             |
| No upgrades possible for MOTomed loop light.a |         |        |        |               |              |           |          |              |             |             |             |

# MOTomed® loop kidz.la

## Leg or arm/upper body trainer

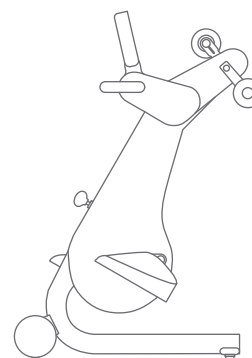

Order number: 260.090

### Basic equipment (For all details, see equipment and functions [page 7](#))

Plastic-coated safety foot shells with fastening straps (pair, incl. pedal cranks)

Soft Grip handles (pair, incl. pedal cranks)

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 70 × 60 × 107–122 cm                               |
| Weight:                                                                                          | 36 kg                                              |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

### Optional product configuration (cannot be changed later)

| Designation                              | Order No. | Page                    |
|------------------------------------------|-----------|-------------------------|
| Gas spring support for height adjustment | 265.002   | <a href="#">p. 57</a> ⓘ |
| Special stand with reduced stand height  | 265.001   | <a href="#">p. 57</a> ⓘ |
| Activation of serial interface           | 265.500   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#) onwards)

|                                                   |         |                         |
|---------------------------------------------------|---------|-------------------------|
| Leg guides with plastic-coated calf shells (pair) | 303.000 | <a href="#">p. 45</a> ⓘ |
| »QuickFix« foot fastening system (pair)           | 265.004 | <a href="#">p. 46</a> ⓘ |
| Hand fixation by wrist cuff, size S (piece)       | 562.020 | <a href="#">p. 48</a> ⓘ |

### These upgrades are possible with MOTomed kidz.la (More information from [page 58](#) onwards)


|              | loop.la | loop.l | loop.a | loop light.la | loop light.l | loop light.a | loop p.la | loop p.l | loop kidz.l | loop kidz.a | loop.la prof |
|--------------|---------|--------|--------|---------------|--------------|--------------|-----------|----------|-------------|-------------|--------------|
| loop kidz.la | ✓       | ✓      | ✓      | ✓             |              |              | ✓         | ✓        | ✓           | ✓           |              |

# MOTomed® loop kidz.I

## Leg trainer



Order number: 260.070

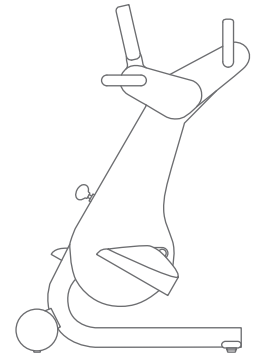

### Basic equipment (For all details, see equipment and functions [page 7](#))

Plastic-coated safety foot shells with fastening straps (pair, incl. pedal cranks)  
Ergonomic Soft Grip handles (pair) »cornets«

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 70 × 60 × 107–122 cm                               |
| Weight:                                                                                          | 35 kg                                              |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

### Optional product configuration (cannot be changed later)

| Designation                              | Order No. | Page                    |
|------------------------------------------|-----------|-------------------------|
| Gas spring support for height adjustment | 265.002   | <a href="#">p. 57</a> ⓘ |
| Special stand with reduced stand height  | 265.001   | <a href="#">p. 57</a> ⓘ |
| Activation of serial interface           | 265.500   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#) onwards)


|                                                   |         |                         |
|---------------------------------------------------|---------|-------------------------|
| Leg guides with plastic-coated calf shells (pair) | 303.000 | <a href="#">p. 45</a> ⓘ |
| »QuickFix« foot fastening system (pair)           | 265.004 | <a href="#">p. 46</a> ⓘ |
| Pedal radius quick adjustment (leg trainer, pair) | 507.020 | <a href="#">p. 46</a> ⓘ |

### These upgrades are possible with MOTomed kidz.I (More information from [page 58](#) onwards)

|             | loop.la | loop.l | loop.a | loop light.la | loop light.l | loop light.a | loop p.la | loop p.l | loop kidz.la | loop kidz.a | loop.la prof |
|-------------|---------|--------|--------|---------------|--------------|--------------|-----------|----------|--------------|-------------|--------------|
| loop kidz.I | ✓       | ✓      |        | ✓             |              |              | ✓         | ✓        | ✓            |             |              |

# MOTomed® loop kidz.a

## Arm/upper body trainer

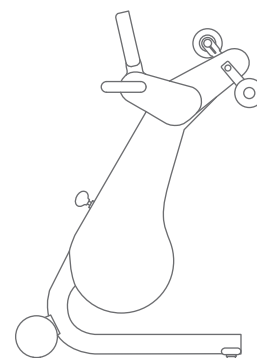

Order number: 260.080

### Basic equipment (For all details, see equipment and functions [page 7](#))

Soft Grip handles (pair, incl. pedal cranks)

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 70 × 60 × 107–122 cm                               |
| Weight:                                                                                          | 31 kg                                              |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | IIa according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

### Optional product configuration (cannot be changed later)

| Designation                              | Order No. | Page                    |
|------------------------------------------|-----------|-------------------------|
| Gas spring support for height adjustment | 265.002   | <a href="#">p. 57</a> ⓘ |
| Special stand with reduced stand height  | 265.001   | <a href="#">p. 57</a> ⓘ |
| Activation of serial interface           | 265.500   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#) onwards)

|                                              |         |                         |
|----------------------------------------------|---------|-------------------------|
| Hand fixation by wrist cuff, size S (piece)  | 562.020 | <a href="#">p. 48</a> ⓘ |
| Tetra handles (pair)                         | 555.000 | <a href="#">p. 50</a> ⓘ |
| Forearm shells with arm cuffs, size S (pair) | 556.004 | <a href="#">p. 53</a> ⓘ |
| Wheelchair stabilizer for children           | 100.020 | <a href="#">p. 54</a> ⓘ |

### These upgrades are possible with MOTomed kidz.a (More information from [page 58](#) onwards)

|             | loop.la | loop.l | loop.a | loop light.la | loop light.l | loop light.a | loop p.la | loop p.l | loop kidz.la | loop kidz.l | loop.la prof |
|-------------|---------|--------|--------|---------------|--------------|--------------|-----------|----------|--------------|-------------|--------------|
| loop kidz.a | ✓       |        | ✓      | ✓             |              |              | ✓         |          | ✓            |             |              |

# MOTomed® PROFESSIONAL SERIES



## For clinics and rehabilitation facilities

The MOTomed Professional Series are tailor-made devices for versatile use in clinics and rehabilitation facilities.

**MOTomed loop.la prof includes the following special features:**

- 1 Plastic-coated safety foot shells
- 2 Leg guides with plastic-coated calf shells and disinfectable fastening straps
- 3 »QuickFix« foot fastening system
- 4 Plastic-coated handles with quick release system
- 5 Gas spring support for height adjustment
- 6 Pedal radius quick adjustment (leg trainer, pair)



Non-binding fig. of MOTomed loop.la prof




# MOTomed® loop.la prof

## Leg or arm/upper body trainer



Order number: 260.039

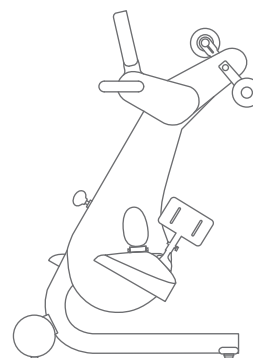

### Basic equipment (For all details, see equipment and functions [page 7](#))

- Plastic-coated safety foot shells (pair, incl. pedal cranks)
- Leg guides with plastic-coated calf shells (pair) and disinfectable fastening straps
- »QuickFix« foot fastening system (pair)
- Plastic-coated handles with quick release system (pair, incl. pedal cranks)
- Gas spring support for height adjustment
- Pedal radius quick adjustment (leg trainer, pair)

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 70 × 60 × 107–122 cm                               |
| Weight:                                                                                          | 39 kg                                              |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | IIa according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

### Optional product configuration (cannot be changed later)

| Designation                             | Order No. | Page                    |
|-----------------------------------------|-----------|-------------------------|
| Special stand with reduced stand height | 265.001   | <a href="#">p. 57</a> ⓘ |
| Activation of serial interface          | 265.500   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#) onwards)

|                                     |         |                         |
|-------------------------------------|---------|-------------------------|
| Hand fixation by wrist cuff (piece) | 562.000 | <a href="#">p. 48</a> ⓘ |
| Wheelchair stabilizer               | 100.000 | <a href="#">p. 54</a> ⓘ |

### These upgrades are possible with MOTomed loop.la (More information from [page 58](#) onwards)

|              | loop.la | loop.l | loop.a | loop light.la | loop light.l | loop light.a | loop p.la | loop p.l | loop kidz.la | loop kidz.l | loop kidz.a |
|--------------|---------|--------|--------|---------------|--------------|--------------|-----------|----------|--------------|-------------|-------------|
| loop.la prof |         |        |        |               |              |              | ✓         |          |              |             |             |



# MOTOmed® GRACILE12



## MOTOmed gracile12 grows with you

The MOTOMed gracile12 grows with the child thanks to a wide range of adjustment options. For example, the **pedal axle with foot shells is infinitely height adjustable**. The MOTOMed gracile12 is available as a leg trainer but also as a leg or arm/upper body trainer to meet the different patient requirements.

Non-binding fig. of MOTOMed gracile12 leg trainer


# MOTomed® gracile12

## Uniquely child-friendly

The MOTomed gracile12 was specially developed for children and small people with limited mobility. From a height of about 90 cm, children and adults can enjoy the efficient MOTomed movement therapy with safety functions, motivational games and therapy programs. If required, the leg trainer can be extended with the arm/upper body trainer and other accessories. The quiet motor is conducive for both concentrated therapy sessions as well as for exercising at home.

### Basic equipment


- Pediatric safety foot shells and pediatric leg guides with calf shells
- Smaller inner diameter of 12 cm
- Sturdy full metal construction, high quality and stable
- Large, user-friendly display (5.7")
- Height adjustment of the pedal axle from 26 to 46 cm
- 2-stage pedal radius adjustment (3.5 cm or 7 cm)
- Easily movable with large castors (Ø 13 cm)
- Telescopic front stand
- Colour: blue/white/chrome

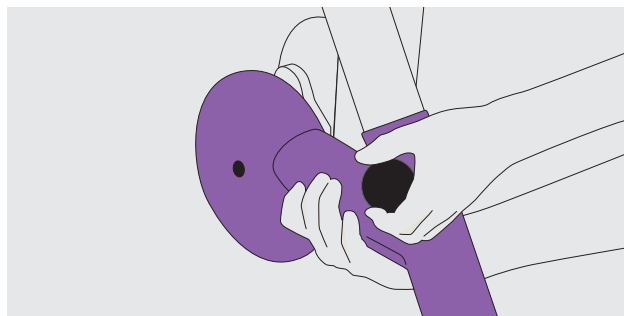

The pedal axle of the leg trainer is steplessly height adjustable from 26 to 46 cm (measured from floor).

### Functions

- Passive, Assistive and Active Training
- Electronic foot insertion aid
- Speed from 1 to 60 rpm (passive)
- Resistance from 0 to 20 (active)
- Motor power from 1 to 10 (passive)
- Presetting of therapy time from 0 to 120 minutes
- MovementProtection & SpasmControl
- Quiet, gentle and harmonious cycle movement (SmoothDriveSystem)
- Gentle starting and stopping of the movement
- Safety stop and SymmetryTraining
- Therapy and motivation programs
- Detailed biofeedback during and after training
- Games
- Language selection

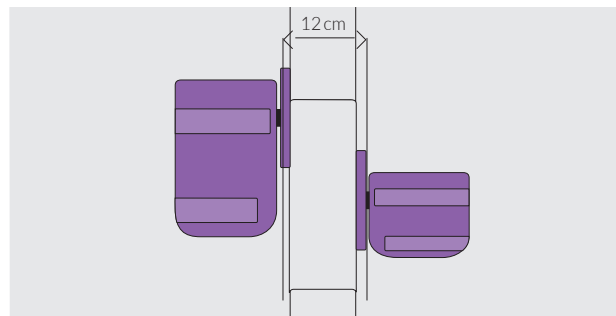


The child-friendly pedal distance of 12 cm avoids misalignment of the hip joint.

# MOTomed® gracile12

## Leg or arm/upper body trainer



Order number: 594.003 + 599.000

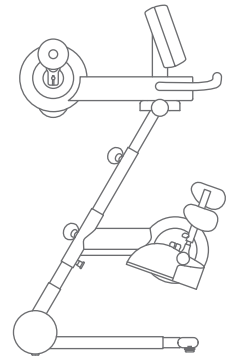

### Basic equipment (For all details, see equipment and functions [page 23](#))

Pediatric safety foot shells (pair, incl. pedal cranks)

Pediatric leg guides with calf shells (pair)

Soft Grip handles (pair, incl. pedal cranks)

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 60–88 × 55 × 99–114 cm                             |
| Weight:                                                                                          | 33 kg                                              |
| Maximum patient weight:                                                                          | 90 kg                                              |
| Display size:                                                                                    | 5.7" / 14.5 cm                                     |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

### Frequently used accessories (More accessories from [page 40](#) onwards)


| Designation                                                                        | Order No. | Page                    |
|------------------------------------------------------------------------------------|-----------|-------------------------|
| Plastic-coated pediatric safety foot shells (pair, incl. pedal cranks)             | 152.120   | <a href="#">p. 44</a> ⓘ |
| Pediatric leg guides with plastic-coated calf shells (pair)                        | 170.100   | <a href="#">p. 45</a> ⓘ |
| Pedal radius quick adjustment for pediatric safety foot shells (leg trainer, pair) | 588.020   | <a href="#">p. 46</a> ⓘ |
| Self-operated foot holders (pair)                                                  | 598.000   | <a href="#">p. 45</a> ⓘ |
| Pediatric forearm shells with arm cuffs, size S (pair)                             | 556.024   | <a href="#">p. 52</a> ⓘ |
| Hand fixation by wrist cuff, size S (piece)                                        | 562.020   | <a href="#">p. 48</a> ⓘ |
| Pediatric tetra handles (pair)                                                     | 555.020   | <a href="#">p. 50</a> ⓘ |
| Vertical handles with hand rests for children (pair)                               | 557.020   | <a href="#">p. 51</a> ⓘ |
| Wheelchair stabilizer for children                                                 | 100.020   | <a href="#">p. 54</a> ⓘ |

# MOTOmed® gracile12

## Leg trainer



Order number: 594.003

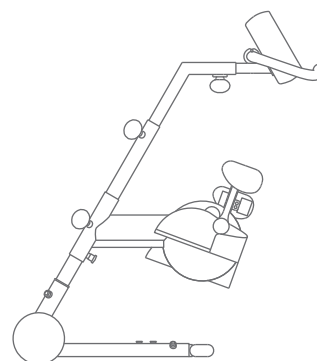

### Basic equipment (For all details, see equipment and functions [page 23](#))

Pediatric safety foot shells (pair, incl. pedal cranks)

Pediatric leg guides with calf shells (pair)

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 68–85 × 55 × 76–106 cm                             |
| Weight:                                                                                          | 25 kg                                              |
| Maximum patient weight:                                                                          | 90 kg                                              |
| Display size:                                                                                    | 5,7" / 14,5 cm                                     |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

### Frequently used accessories (More accessories from [page 40](#) onwards)

| Designation                                                                        | Order No. | Page                    |
|------------------------------------------------------------------------------------|-----------|-------------------------|
| Plastic-coated pediatric safety foot shells (pair, incl. pedal cranks)             | 152.120   | <a href="#">p. 44</a> ⓘ |
| Pediatric leg guides with plastic-coated calf shells (pair)                        | 170.100   | <a href="#">p. 45</a> ⓘ |
| Pedal radius quick adjustment for pediatric safety foot shells (leg trainer, pair) | 588.020   | <a href="#">p. 46</a> ⓘ |
| Self-operated foot holders (pair)                                                  | 598.000   | <a href="#">p. 45</a> ⓘ |
| Safety foot shells for adults, gracile12 (pair, incl. pedal cranks)*               | 586.100   | <a href="#">p. 44</a> ⓘ |
| Flat stand (flat steel)                                                            | 587.100   | <a href="#">p. 55</a> ⓘ |

\*Only available in combination with flat stand (order no. 587.100)





# MOTomed® LAYSON



## Simple operation and modern hygiene management

MOTomed layson is flexible and easy to move thanks to free rolling casters. The high-quality material of the softly rounded surfaces also meets the high requirements of hygiene management in rehabilitation facilities and clinics.




Non-binding illustration of MOTomed layson.la (expandable chassis)

# MOTomed® layson edition


The new generation of movement therapy in supine position: MOTomed layson offers motor-assisted and active training of legs or arms with impressively intuitive and easy operation. The height of the MOTomed layson can be adjusted steplessly without effort by means of a gas spring. It can be easily and comfortably driven and can be individually adjusted in just a few simple steps. The orange control elements have an intuitive calling character. The reduction to just a few operating elements enables fast and effective handling in everyday clinical routine.

## Two chassis types available

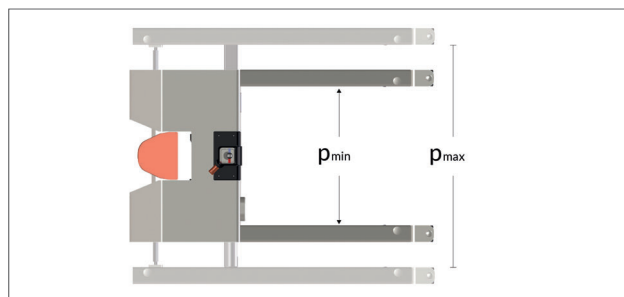

### Parallel chassis

By moving the rails parallel (manually with Allen key), the width of the chassis can be adjusted to different requirements. Suitable for one single adjustment.

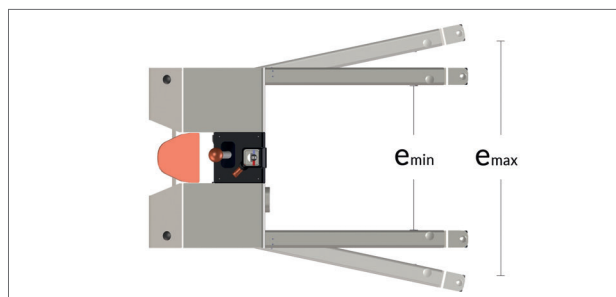

### Expandable chassis

The two chassis rails can be spread (by using a lever) to enable quick and tool-free adaptation to different applications. Suitable for multiple daily adjustments.

## Basic equipment

- Gas spring support for height adjustment
- Selectable parallel chassis or expandable chassis with ground fixation
- Knee bending adjustment incl. adjustment wheel<sup>\*1</sup>
- Manual horizontal adjustment
- Colour touch display (7"), rotatable, inclinable<sup>\*2</sup>
- Quick release system for foot shells and handles<sup>\*1</sup>
- Pedal radius: 7 cm
- Sturdy metal construction, high quality and stable
- Closed plastic body (PC/ABS), easy to clean and disinfect
- Easily movable with large castors (Ø 7.5 cm)
- USB interface
- Plastic-coated transport handle
- Patient remote control
- Colour: white/chrome/black/orange


















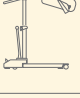

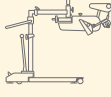
<sup>\*1</sup> Basic equipment of the MOTomed layson leg or arm/upper body trainer

<sup>\*2</sup> Basic equipment with fixed operating panel

## Functions

- Passive, Assistive and Active Training
- Electronic foot insertion aid
- Speed from 1 to 60 rpm (passive)
- Resistance from 0 to 20 (active)
- Motor power from 1 to 10 (passive)
- Presetting of therapy time from 0 to 120 minutes
- MovementProtection & SpasmControl
- Safety stop
- SymmetryTraining
- Quiet, gentle and harmonious cycle movement (SmoothDriveSystem)
- Gentle starting and stopping of the movement
- Therapy and motivation programs
- Detailed biofeedback during and after training
- Export of training data, import of your own training programs
- Games
- Slideshow via USB (delivered without USB stick)
- Language selection

# MOTomed® layson model overview

|                                                                                     | Order No. | Description/Designation                                                                                                                                                 | Page |
|-------------------------------------------------------------------------------------|-----------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------|
|    | 261.130   |  <b>MOTomed layson.la</b><br>Leg or arm/upper body trainer (expandable chassis)        | 29   |
|    | 261.030   |  <b>MOTomed layson.la</b><br>Leg or arm/upper body trainer (parallel chassis)          | 30   |
|    | 261.110   |  <b>MOTomed layson.l</b><br>Leg trainer (expandable chassis)                           | 31   |
|    | 261.010   |  <b>MOTomed layson.l</b><br>Leg trainer (parallel chassis)                             | 32   |
|  | 261.330   |  <b>MOTomed layson kidz.la</b><br>Leg or arm/upper body trainer (expandable chassis) | 33   |
|  | 261.230   |  <b>MOTomed layson kidz.la</b><br>Leg or arm/upper body trainer (parallel chassis)   | 34   |
|  | 261.310   |  <b>MOTomed layson kidz.l</b><br>Leg trainer (expandable chassis)                    | 35   |
|  | 261.210   |  <b>MOTomed layson kidz.l</b><br>Leg trainer (parallel chassis)                      | 36   |
|  | 261.119   |  <b>MOTomed layson.l dia</b><br>Leg trainer (expandable chassis)                     | 37   |
|  | 261.139   |  <b>MOTomed layson.la prof</b><br>Leg or arm/upper body trainer (expandable chassis) | 39   |




# MOTomed® layson.la (expandable chassis)

## Leg or arm/upper body trainer



Order number: 261.130  
on request

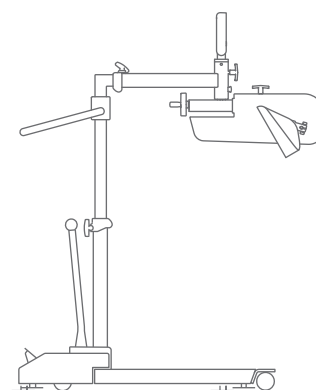

### Basic equipment (For all details, see equipment and functions [page 27](#))

Plastic-coated safety foot shells with disinfectable fastening straps and quick release system (pair, incl. pedal cranks)  
Plastic-coated handles with quick release system (pair)  
Knee bending adjustment incl. adjustment wheel

### Technical data

|                                                                            |                                                    |
|----------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                        | 134–160 × 67–105 × 118–166 cm                      |
| Weight:                                                                    | 86 kg                                              |
| Height adjustment (max. bed height):                                       | 108 cm                                             |
| Chassis height (ground clearance height):                                  | 10 cm                                              |
| Inner dimensions expandable chassis (e <sub>min</sub> –e <sub>max</sub> ): | 55–93 cm                                           |
| Maximum patient weight:                                                    | 135 kg                                             |
| Display size:                                                              | 7" / 18 cm                                         |
| Classification:                                                            | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |

Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF

### Optional product configuration (cannot be changed later)

| Designation                    | Order No. | Page                    |
|--------------------------------|-----------|-------------------------|
| Removable operating panel      | 261.520   | <a href="#">p. 57</a> ⓘ |
| Activation of serial interface | 265.500   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#) onwards)


|                                                                                                   |         |                         |
|---------------------------------------------------------------------------------------------------|---------|-------------------------|
| »QuickFix« foot fastening system (pair)                                                           | 265.004 | <a href="#">p. 46</a> ⓘ |
| Hand fixation by wrist cuff (piece)                                                               | 562.000 | <a href="#">p. 48</a> ⓘ |
| Plastic-coated forearm shells with disinfectable fastening straps and quick release system (pair) | 356.200 | <a href="#">p. 52</a> ⓘ |
| »TrainCare« with plastic-coated leg guides                                                        | 261.500 | <a href="#">p. 47</a> ⓘ |

# MOTomed® layson.la (parallel chassis)

## Leg or arm/upper body trainer



Order number: 261.030  
on request

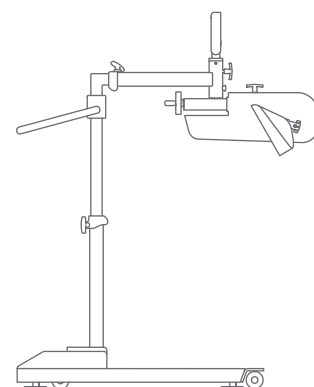

### Basic equipment (For all details, see equipment and functions [page 27](#))

Plastic-coated safety foot shells with disinfectable fastening straps and quick release system (pair, incl. pedal cranks)  
Plastic-coated handles with quick release system (pair)  
Knee bending adjustment incl. adjustment wheel

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 134–160 × 67–97 × 118–166 cm                       |
| Weight:                                                                                          | 86 kg                                              |
| Height adjustment (max. bed height):                                                             | 108 cm                                             |
| Chassis height (ground clearance height):                                                        | 10 cm                                              |
| Inner dimensions parallel chassis (p <sub>min</sub> –p <sub>max</sub> ):                         | 55–85 cm                                           |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

### Optional product configuration (cannot be changed later)

| Designation                    | Order No. | Page                    |
|--------------------------------|-----------|-------------------------|
| Removable operating panel      | 261.520   | <a href="#">p. 57</a> ⓘ |
| Activation of serial interface | 265.500   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#) onwards)


|                                                                                                   |         |                         |
|---------------------------------------------------------------------------------------------------|---------|-------------------------|
| »QuickFix« foot fastening system (pair)                                                           | 265.004 | <a href="#">p. 46</a> ⓘ |
| Hand fixation by wrist cuff (piece)                                                               | 562.000 | <a href="#">p. 48</a> ⓘ |
| Plastic-coated forearm shells with disinfectable fastening straps and quick release system (pair) | 356.200 | <a href="#">p. 52</a> ⓘ |
| »TrainCare« with plastic-coated leg guides                                                        | 261.500 | <a href="#">p. 47</a> ⓘ |

# MOTomed® layson.I (expandable chassis)

## Leg trainer



Order number: 261.110

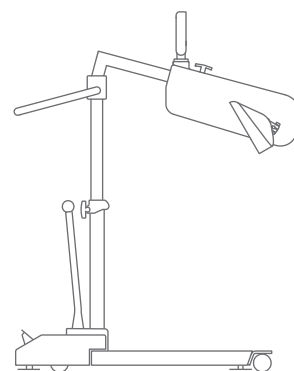

### Basic equipment (For all details, see equipment and functions [page 27](#))

Plastic-coated safety foot shells with disinfected fastening straps (pair, incl. pedal cranks)

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 110–130 × 62–100 × 109–167 cm                      |
| Weight:                                                                                          | 79 kg                                              |
| Height adjustment (max. bed height):                                                             | 103 cm                                             |
| Chassis height (ground clearance height):                                                        | 10 cm                                              |
| Inner dimensions expandable chassis (e <sub>min</sub> –e <sub>max</sub> ):                       | 34–88 cm                                           |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

### Optional product configuration (cannot be changed later)

| Designation                                    | Order No. | Page                    |
|------------------------------------------------|-----------|-------------------------|
| Knee bending adjustment incl. adjustment wheel | 261.501   | <a href="#">p. 57</a> ⓘ |
| Removable operating panel                      | 261.520   | <a href="#">p. 57</a> ⓘ |
| Activation of serial interface                 | 265.500   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#) onwards)


|                                            |         |                         |
|--------------------------------------------|---------|-------------------------|
| »QuickFix« foot fastening system (pair)    | 265.004 | <a href="#">p. 46</a> ⓘ |
| »TrainCare« with plastic-coated leg guides | 261.500 | <a href="#">p. 47</a> ⓘ |

# MOTomed® layson.I (parallel chassis)

## Leg trainer



Order number: 261.010

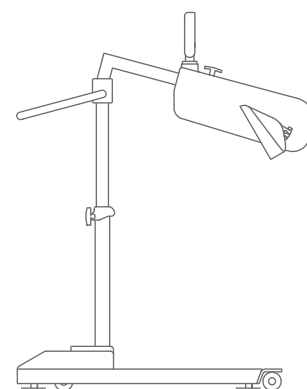

### Basic equipment (For all details, see equipment and functions [page 27](#))

Plastic-coated safety foot shells with disinfectable fastening straps (pair, incl. pedal cranks)

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 110–130 × 62–92 × 123–167 cm                       |
| Weight:                                                                                          | 79 kg                                              |
| Height adjustment (max. bed height):                                                             | 103 cm                                             |
| Chassis height (ground clearance height):                                                        | 10 cm                                              |
| Inner dimensions parallel chassis (p <sub>min</sub> –p <sub>max</sub> ):                         | 50–80 cm                                           |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

### Optional product configuration (cannot be changed later)

| Designation                                    | Order No. | Page                    |
|------------------------------------------------|-----------|-------------------------|
| Knee bending adjustment incl. adjustment wheel | 261.501   | <a href="#">p. 57</a> ⓘ |
| Removable operating panel                      | 261.520   | <a href="#">p. 57</a> ⓘ |
| Activation of serial interface                 | 265.500   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#) onwards)


|                                            |         |                         |
|--------------------------------------------|---------|-------------------------|
| »QuickFix« foot fastening system (pair)    | 265.004 | <a href="#">p. 46</a> ⓘ |
| »TrainCare« with plastic-coated leg guides | 261.500 | <a href="#">p. 47</a> ⓘ |

# MOTomed® layson kidz.la (expandable chassis)

## Leg or arm/upper body trainer



Order number: 261.330  
on request

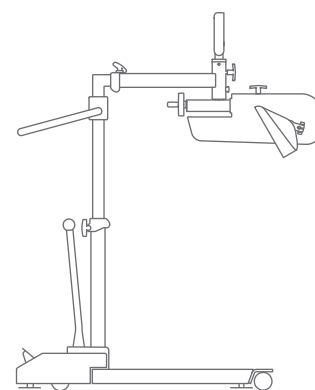

### Basic equipment (For all details, see equipment and functions [page 27](#))

Plastic-coated pediatric safety foot shells with disinfected fastening straps and quick release system (pair, incl. pedal cranks)

Plastic-coated pediatric handles with quick release system (pair)

Knee bending adjustment incl. adjustment wheel

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 134–160 × 67–105 × 118–166 cm                      |
| Weight:                                                                                          | 86 kg                                              |
| Height adjustment (max. bed height):                                                             | 108 cm                                             |
| Chassis height (ground clearance height):                                                        | 10 cm                                              |
| Inner dimensions expandable chassis (e <sub>min</sub> –e <sub>max</sub> ):                       | 55–93 cm                                           |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

### Optional product configuration (cannot be changed later)

| Designation                    | Order No. | Page                    |
|--------------------------------|-----------|-------------------------|
| Removable operating panel      | 261.520   | <a href="#">p. 57</a> ⓘ |
| Activation of serial interface | 265.500   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#) onwards)


|                                                           |         |                         |
|-----------------------------------------------------------|---------|-------------------------|
| Hand fixation by wrist cuff, size S (piece)               | 562.020 | <a href="#">p. 48</a> ⓘ |
| »TrainCare kidz« with plastic-coated pediatric leg guides | 261.502 | <a href="#">p. 47</a> ⓘ |

# MOTomed® layson kidz.la (parallel chassis)

## Leg or arm/upper body trainer



Order number: 261.230  
on request

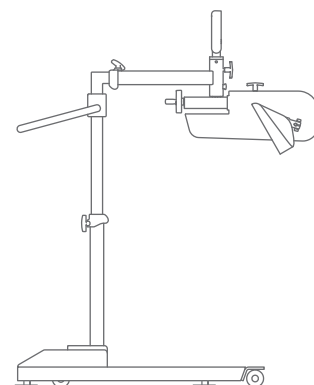

### Basic equipment (For all details, see equipment and functions [page 27](#))

Plastic-coated pediatric safety foot shells with disinfectable fastening straps and quick release system (pair, incl. pedal cranks)  
Plastic-coated pediatric handles with quick release system (pair)  
Knee bending adjustment incl. adjustment wheel

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 134–160 × 67–97 × 131–166 cm                       |
| Weight:                                                                                          | 86 kg                                              |
| Height adjustment (max. bed height):                                                             | 108 cm                                             |
| Chassis height (ground clearance height):                                                        | 10 cm                                              |
| Inner dimensions parallel chassis (p <sub>min</sub> –p <sub>max</sub> ):                         | 55–85 cm                                           |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

### Optional product configuration (cannot be changed later)

| Designation                    | Order No. | Page                    |
|--------------------------------|-----------|-------------------------|
| Removable operating panel      | 261.520   | <a href="#">p. 57</a> ⓘ |
| Activation of serial interface | 265.500   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#) onwards)


|                                                           |         |                         |
|-----------------------------------------------------------|---------|-------------------------|
| Hand fixation by wrist cuff, size S (piece)               | 562.020 | <a href="#">p. 48</a> ⓘ |
| »TrainCare kidz« with plastic-coated pediatric leg guides | 261.502 | <a href="#">p. 47</a> ⓘ |

# MOTomed® layson kidz.I (expandable chassis)

## Leg trainer



Order number: 261.310  
on request

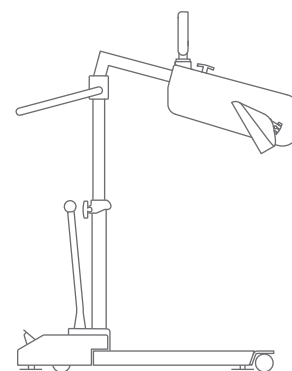

### Basic equipment (For all details, see equipment and functions [page 27](#))

Plastic-coated pediatric safety foot shells with disinfectable fastening straps (pair, incl. pedal cranks)

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 110-130 × 62-100 × 109-167 cm                      |
| Weight:                                                                                          | 79 kg                                              |
| Height adjustment (max. bed height):                                                             | 103 cm                                             |
| Chassis height (ground clearance height):                                                        | 10 cm                                              |
| Inner dimensions expandable chassis (e <sub>min</sub> -e <sub>max</sub> ):                       | 34-88 cm                                           |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100-240 V~ / max. 120 VA, mains frequency: 47-63 Hz, class II equipment / Type BF |                                                    |

### Optional product configuration (cannot be changed later)

| Designation                                    | Order No. | Page                    |
|------------------------------------------------|-----------|-------------------------|
| Knee bending adjustment incl. adjustment wheel | 261.501   | <a href="#">p. 57</a> ⓘ |
| Removable operating panel                      | 261.520   | <a href="#">p. 57</a> ⓘ |
| Activation of serial interface                 | 265.500   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#) onwards)


|                                                           |         |                         |
|-----------------------------------------------------------|---------|-------------------------|
| »TrainCare kidz« with plastic-coated pediatric leg guides | 261.502 | <a href="#">p. 47</a> ⓘ |
|-----------------------------------------------------------|---------|-------------------------|

# MOTomed® layson kidz.I (parallel chassis)

## Leg trainer



Order number: 261.210  
on request

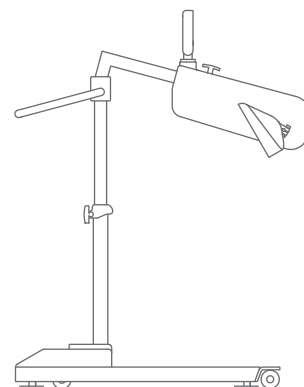

### Basic equipment (For all details, see equipment and functions [page 27](#))

Plastic-coated pediatric safety foot shells with disinfectable fastening straps (pair, incl. pedal cranks)

### Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 110–130 × 62–92 × 123–167 cm                       |
| Weight:                                                                                          | 79 kg                                              |
| Height adjustment (max. bed height):                                                             | 103 cm                                             |
| Chassis height (ground clearance height):                                                        | 10 cm                                              |
| Inner dimensions parallel chassis (p <sub>min</sub> –p <sub>max</sub> ):                         | 50–80 cm                                           |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

### Optional product configuration (cannot be changed later)

| Designation                                    | Order No. | Page                    |
|------------------------------------------------|-----------|-------------------------|
| Knee bending adjustment incl. adjustment wheel | 261.501   | <a href="#">p. 57</a> ⓘ |
| Removable operating panel                      | 261.520   | <a href="#">p. 57</a> ⓘ |
| Activation of serial interface                 | 265.500   | <a href="#">p. 57</a> ⓘ |


### Frequently used accessories (More accessories from [page 40](#) onwards)

|                                                           |         |                         |
|-----------------------------------------------------------|---------|-------------------------|
| »TrainCare kidz« with plastic-coated pediatric leg guides | 261.502 | <a href="#">p. 47</a> ⓘ |
|-----------------------------------------------------------|---------|-------------------------|



# MOTomed® layson.l dia (expandable chassis)

Leg trainer specially designed for dialysis treatment

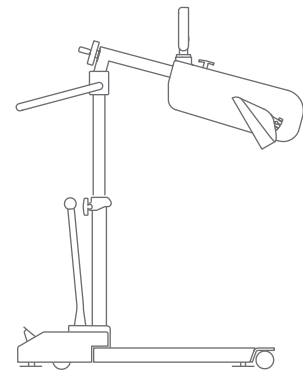

Order number: 261.119

## Basic equipment (For all details, see equipment and functions [page 27](#))

Plastic-coated safety foot shells with disinfectable fastening straps (pair, incl. pedal cranks)  
Knee bending adjustment incl. adjustment wheel  
Removable operating panel

## Technical data

|                                                                                                  |                                                    |
|--------------------------------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                                              | 110–130 × 62–100 × 123–167 cm                      |
| Weight:                                                                                          | 80 kg                                              |
| Height adjustment (max. bed height):                                                             | 103 cm                                             |
| Chassis height (ground clearance height):                                                        | 10 cm                                              |
| Inner dimensions expandable chassis (e <sub>min</sub> –e <sub>max</sub> ):                       | 34–88 cm                                           |
| Maximum patient weight:                                                                          | 135 kg                                             |
| Display size:                                                                                    | 7" / 18 cm                                         |
| Classification:                                                                                  | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |
| Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF |                                                    |

## Optional product configuration (cannot be changed later)

| Designation                    | Order No. | Page                    |
|--------------------------------|-----------|-------------------------|
| Activation of serial interface | 265.500   | <a href="#">p. 57</a> ⓘ |

## Frequently used accessories (More accessories from [page 40](#) onwards)

| Designation                                | Order No. | Page                    |
|--------------------------------------------|-----------|-------------------------|
| »QuickFix« foot fastening system (pair)    | 265.004   | <a href="#">p. 46</a> ⓘ |
| »TrainCare« with plastic-coated leg guides | 261.500   | <a href="#">p. 47</a> ⓘ |

# MOTomed® PROFESSIONAL SERIES



## For clinics and rehabilitation facilities

The MOTomed Professional Series are tailor-made devices for versatile use in clinics and rehabilitation facilities.

MOTomed layson.la prof includes the following special features:

- ❶ Plastic-coated safety foot shells with quick release system
- ❷ »TrainCare« with plastic-coated leg guides and disinfectable fastening straps
- ❸ »QuickFix« foot fastening system
- ❹ Knee bending adjustment incl. adjustment wheel
- ❺ Plastic-coated handles with quick release system
- ❻ Removable operating panel




Non-binding illustration of MOTomed layson.la prof

# MOTomed® layson.la prof (expandable chassis)

## Leg or arm/upper body trainer



Order number: 261.139  
on request

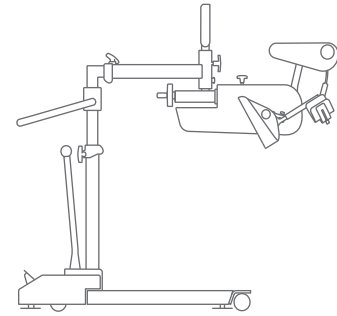

### Basic equipment (For all details, see equipment and functions [page 27](#))

Plastic-coated safety foot shells with quick release system (pair, incl. pedal cranks)  
»TrainCare« with plastic-coated leg guides and disinfectable fastening straps  
»QuickFix« foot fastening system (pair)  
Knee bending adjustment incl. adjustment wheel  
Plastic-coated handles with quick release system (pair)  
Removable operating panel

### Technical data

|                                                                            |                                                    |
|----------------------------------------------------------------------------|----------------------------------------------------|
| Dimensions (L×W×H):                                                        | 159–186 × 67–105 × 131–166 cm                      |
| Weight:                                                                    | 92 kg                                              |
| Height adjustment (max. bed height):                                       | 108 cm                                             |
| Chassis height (ground clearance height):                                  | 10 cm                                              |
| Inner dimensions expandable chassis (e <sub>min</sub> –e <sub>max</sub> ): | 55–93 cm                                           |
| Maximum patient weight:                                                    | 135 kg                                             |
| Display size:                                                              | 7" / 18 cm                                         |
| Classification:                                                            | Ila according to MDD 93/42/EEC / MDR (EU) 2017/745 |

Mains voltage: 100–240 V~ / max. 120 VA, mains frequency: 47–63 Hz, class II equipment / Type BF

### Optional product configuration (cannot be changed later)

| Designation                    | Order No. | Page                    |
|--------------------------------|-----------|-------------------------|
| Activation of serial interface | 265.500   | <a href="#">p. 57</a> ⓘ |

### Frequently used accessories (More accessories from [page 40](#) onwards)

| Designation                                                                                       | Order No. | Page                    |
|---------------------------------------------------------------------------------------------------|-----------|-------------------------|
| Hand fixation by wrist cuff (piece)                                                               | 562.000   | <a href="#">p. 48</a> ⓘ |
| Plastic-coated forearm shells with disinfectable fastening straps and quick release system (pair) | 356.200   | <a href="#">p. 52</a> ⓘ |
| Docking station for removable operating panel                                                     | 261.523   | <a href="#">p. 54</a> ⓘ |




MOTomed®  
ACCESSORIES



# MOTomed® accessories

## Leg trainer accessories

○ Basic equipment    ● Accessories    ◐ Model-dependent equipment

|                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                               |  | muvi | loop | gracile12 | layson |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|------|------|-----------|--------|
|  <p>265.400<br/><b>Soft Grip safety foot shells with fastening straps (pair, incl. pedal cranks)</b></p> <ul style="list-style-type: none"> <li>• Sturdy metal foot shells, softly padded and non-slip</li> <li>• High safety edge to the pedal bar</li> <li>• Fixation of the feet by means of fastening straps</li> <li>• Two-stage pedal radius 7.0 cm and 12.5 cm adjustable</li> </ul> <p>● loop   ○ loop light   ● loop parkinson   ● loop kidz   ● loop prof</p>                                                                                      |  |      | ◐    |           |        |
|  <p>265.421<br/><b>Plastic-coated safety foot shells with fixation by fastening straps (pair, incl. pedal cranks)</b></p> <ul style="list-style-type: none"> <li>• Stable metal foot shells coated with disinfectable soft PVC</li> <li>• Recommended for use in rehabilitation facilities</li> <li>• High safety edge to the pedal bar</li> <li>• Fixation of the feet by means of fastening straps</li> <li>• Two-stage pedal radius 7.0 cm and 12.5 cm adjustable</li> </ul> <p>○ loop   ● loop light   ○ loop parkinson   ○ loop kidz   ○ loop prof</p> |  |      | ◐    |           |        |
|  <p>265.440<br/><b>Plastic-coated safety foot shells with disinfectable fastening straps (pair)</b></p> <ul style="list-style-type: none"> <li>• Stable metal foot shells coated with disinfectable soft PVC</li> <li>• Recommended for use in rehabilitation facilities</li> <li>• High safety edge to the pedal bar</li> <li>• Fixation of the feet by means of disinfectable fastening straps</li> <li>• Two-stage pedal radius 7.0 cm and 12.5 cm adjustable</li> </ul>                                                                                |  | ●    |      |           |        |



Basic equipment   
  Accessories   
  Model-dependent equipment

|      |      |           |        |
|------|------|-----------|--------|
| muvi | loop | gracile12 | layson |
|------|------|-----------|--------|



265.441

**Plastic-coated safety foot shells with disinfected fastening straps (pair, incl. pedal cranks)**

- Stable metal foot shells coated with disinfected soft PVC
- Recommended for use in rehabilitation facilities
- High safety edge to the pedal bar
- Fixation of the feet by means of disinfected fastening straps
- Two-stage pedal radius 7.0 cm and 12.5 cm adjustable



- loop   
  loop light   
  loop parkinson   
  loop kidz   
  loop prof



261.440

**Plastic-coated safety foot shells with disinfected fastening straps (pair, incl. pedal cranks, with external rotation)**

- Stable metal foot shells coated with disinfected soft PVC
- Recommended for use in rehabilitation facilities
- High safety edge to the pedal bar
- Fixation of the feet by means of disinfected fastening straps



- layson (.l)   
  layson kidz (.l)   
 layson dia

261.460







**Plastic-coated safety foot shells with disinfected fastening straps and quick release system (pair, incl. pedal cranks, with external rotation)**

- layson (.l)   
 layson (.la)   
 layson kidz   
 layson dia   
 layson prof

on request



○ Basic equipment   ● Accessories   ◐ Model-dependent equipment

|                                                                                                                                                                                                                                                                                                                                                   |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                       | muvi | loop | gracile12 | layson |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------|------|-----------|--------|
| <br>                                                                                                                                                                            | <p><b>265.460</b><br/> <b>Plastic-coated pediatric safety foot shells with fixation by fastening straps (pair, incl. pedal cranks)</b></p> <ul style="list-style-type: none"> <li>• Stable metal foot shells coated with disinfectable soft PVC</li> <li>• Recommended for use in rehabilitation facilities</li> <li>• High safety edge to the pedal bar</li> <li>• Fixation of the feet by means of fastening straps</li> <li>• Foot shell inside 8.8 cm wide and 20.6 cm long</li> <li>• Two-stage pedal radius 7.0 cm and 12.5 cm adjustable</li> <li>• Specially designed for children</li> </ul>                 |      | ●    |           |        |
| <br>   | <p><b>265.480</b><br/> <b>Plastic-coated pediatric safety foot shells with disinfectable fastening straps (pair, incl. pedal cranks)</b></p> <ul style="list-style-type: none"> <li>• Stable metal foot shells coated with disinfectable soft PVC</li> <li>• Recommended for use in rehabilitation facilities</li> <li>• High safety edge to the pedal bar</li> <li>• Fixation of the feet by means of disinfectable fastening straps</li> <li>• Foot shell inside 8.8 cm wide and 20.6 cm long</li> <li>• Two-stage pedal radius 7.0 cm and 12.5 cm adjustable</li> <li>• Specially designed for children</li> </ul> |      | ●    |           |        |
|                                                                                                                                                                                                                                                                                                                                                   | <p><b>261.480</b><br/> <b>Plastic-coated pediatric safety foot shells with disinfectable fastening straps (pair, incl. pedal cranks, with external rotation)</b></p> <p>● layson (.l)   ○ layson kidz (.l)   ● layson dia</p>                                                                                                                                                                                                                                                                                                                                                                                         |      |      |           | ◐      |
|                                                                                                                                                                                                                                                                                                                                                   | <p><b>261.490</b><br/> <b>Plastic-coated pediatric safety foot shells with disinfectable fastening straps and quick release system (pair, incl. pedal cranks, with external rotation)</b></p> <p>● layson   ● layson kidz (.l)   ○ layson kidz (.la)   ● layson dia   ● layson prof</p> <p><b>on request</b></p>                                                                                                                                                                                                                                                                                                      |      |      |           | ◐      |

○ Basic equipment   ● Accessories   ○ Model-dependent equipment

muvi  
loop  
gracile12  
layson



152.120

**Plastic-coated pediatric safety foot shells (pair, incl. pedal cranks)**

- Sturdy metal foot shells, padded and slip-proof
- comfortable fixation of the feet with fastening straps
- Foot shell 10.1 cm wide and 21.7 cm long with open side rim
- Pedal radius adjustable 2-fold (3.5 cm and 7.0 cm)
- Recommended for use in rehabilitation facilities
- Easy to disinfect



●



586.100

**Safety foot shells for adults, gracile12 (pair, incl. pedal cranks)**

- Sturdy metal foot shells, padded and slip-proof
- Comfortable fixation of the feet with fastening straps
- Only possible in combination with flat stand (order no. 587.100)



●



302.000

**Leg guides with calf shells (pair)**

- For secure support and guidance of the legs
- Spring-loaded calf shells nestle elastically to the body
- Comfortable fixation by means of fastening straps
- Articulated and height-adjustable (9.0 cm to 16.0 cm)
- Removable calf shells, fold-down guide rail (lower part)
- Only available in combination with Soft Grip safety foot shells (order no. 265.400)



●

● loop   ● loop light   ● loop parkinson   ● loop kidz   ● loop prof



○ Basic equipment   ● Accessories   ◐ Model-dependent equipment

|  |                                                                                                                                                                                                                                                                                                                                | muvi | loop | gracile12 | layson |
|--|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------|------|-----------|--------|
|  | <p>303.000</p> <p><b>Leg guides with plastic-coated calf shells and fastening straps (pair)</b></p> <ul style="list-style-type: none"> <li>• Articulated and height-adjustable</li> <li>• Recommended for use in rehabilitation facilities</li> </ul>                                                                          | ●    | ●    |           |        |
|  | <p>186.000</p> <p><b>Leg guides with plastic-coated calf shells (pair) and disinfectable fastening straps</b></p> <ul style="list-style-type: none"> <li>• Recommended for use in rehabilitation facilities</li> <li>• Easy to disinfect</li> </ul> <p>● muvi ● loop ● loop light ● loop parkinson ● loop kidz ○ loop prof</p> | ●    | ◐    |           |        |
|  | <p>170.100</p> <p><b>Leg guides with plastic-coated calf shells and fastening straps (pair)</b></p> <ul style="list-style-type: none"> <li>• Recommended for use in rehabilitation facilities</li> <li>• Easy to disinfect</li> </ul> <p>● loop kidz* ● gracile12</p>                                                          |      | ●    | ●         |        |
|  |                                                                                                                                                                                                                                                                                                                                |      |      |           |        |
|  | <p>598.000</p> <p><b>Self-operated foot holders (pair)</b></p> <ul style="list-style-type: none"> <li>• Soft foam rollers fix the feet gently and safely</li> <li>• Firm hold even with strong spasticity due to the safety locking device</li> <li>• For quick mounting and dismounting</li> </ul>                            |      |      | ●         |        |
|  |                                                                                                                                                                                                                                                                                                                                |      |      |           |        |

\* Only available in combination with pediatric safety foot shells (order no. 265.460 and 265.480)

○ Basic equipment   ● Accessories   ◐ Model-dependent equipment

| muvi | loop | gracile12 | layson |
|------|------|-----------|--------|
|------|------|-----------|--------|



265.004

»QuickFix« foot fastening system (pair)

- For quick fixation and release of the feet
- Paddings are easy to clean and disinfect
- Not in combination with pediatric safety foot shells

- muvi   ● loop   ● loop light   ● loop parkinson   ● loop kidz   ○ loop prof  
 ● layson   ● layson dia   ○ layson prof



|   |   |  |   |
|---|---|--|---|
| ● | ◐ |  | ◐ |
|---|---|--|---|



507.020

Pedal radius quick adjustment (leg trainer, pair)

- Incl. plastic-coated safety foot shells
- Adjustable in 4 steps (5.0 / 7.5 / 10.0 / 12.5 cm)
- Recommended for joint stiffness (contractures) and severe spasticity



|  |   |   |  |
|--|---|---|--|
|  | ◐ |   |  |
|  |   | ◐ |  |

507.021

Pedal radius quick adjustment (leg trainer, pair)

- Incl. plastic-coated safety foot shells with disinfectable fastening straps (pair)

- loop   ● loop light   ● loop parkinson   ● loop kidz   ○ loop prof



588.020





Pedal radius quick adjustment for pediatric safety foot shells (leg trainer, pair)

- Incl. Pediatric safety foot shells
- For precise adjustment of the pedal radius
- Adjustable in 3 steps (5.5 / 8.0 / 10.5 cm)
- Recommended for joint stiffness (contractures) and severe spasticity
- The attachment widens the inner distance of the foot shells by 3.5 cm



|  |  |   |  |
|--|--|---|--|
|  |  | ● |  |
|--|--|---|--|

○ Basic equipment   ● Accessories   ◐ Model-dependent equipment




|                                                                                                                                                                           |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                  | muvi | loop | gracile12 | layson |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------|------|-----------|--------|
| <br>    | <p><b>261.500</b><br/> <b>»TrainCare« with plastic-coated leg guides and disinfectable fastening straps</b></p> <ul style="list-style-type: none"> <li>• For a secure hold and guidance of the legs, prevents unintentional lateral movement</li> <li>• Prevents overstretching or blocking of the knee joints</li> <li>• Knee relief individually adjustable on both sides</li> <li>• With plastic-coated calf shells (can be disinfected) and disinfectable expanders</li> <li>• Fixation by means of disinfectable fastening straps</li> </ul> <p>● layson   ● layson dia   ○ layson prof</p> |      |      |           | ◐      |
| <br> | <p><b>261.502</b><br/> <b>»TrainCare kidz« with plastic-coated pediatric leg guides and disinfectable fastening straps</b></p> <ul style="list-style-type: none"> <li>• Only available in combination with Pediatric safety foot shells (order no. 261.480 and 261.490)</li> </ul> <p>● layson kidz</p>                                                                                                                                                                                                                                                                                          |      |      |           | ●      |

## Arm trainer accessories

○ Basic equipment   ● Accessories   ◐ Model-dependent equipment

|                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                     |  | muvi | loop | gracie12 | layson |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|------|------|----------|--------|
| <p>507.050<br/><b>Pedal radius quick adjustment (arm trainer, pair)</b></p> <ul style="list-style-type: none"> <li>• For precise adjustment of the pedal radius</li> <li>• Steplessly or precisely adjustable in 4 steps (5.0 / 7.5 / 10.0 / 12.5 cm)</li> <li>• Recommended for joint stiffness (contractures) and severe spasticity</li> </ul> <p> </p>                                                                                                                                           |  | ●    |      |          |        |
| <p>599.100<br/><b>Arm/upper body trainer active and passive (retrofit kit)</b></p> <ul style="list-style-type: none"> <li>• Incl. swiveling handle bar for leg training</li> <li>• Can be swiveled into the training area for maximum legroom during arm/upper body training</li> <li>• Incl. symmetry training and motivation programs</li> <li>• Height adjustable without tools for an ideal training position</li> <li>• Pedal radius adjustable 2-fold (3.5 cm and 7.0 cm)</li> </ul> <p> </p> |  |      |      | ●        |        |
| <p>562.000<br/><b>Hand fixation by wrist cuff, size L (piece)</b></p> <ul style="list-style-type: none"> <li>• Easy and quick fixation of the paralyzed (weak) hand on the arm/upper body trainer, on handle or the various hand grips</li> <li>• Recommended for users with limited gripping function</li> </ul> <p> </p>                                                                                                                                                                          |  | ●    | ●    | ●        | ●      |
| <p>562.030<br/><b>Hand fixation by wrist cuff, size M (piece)</b></p>                                                                                                                                                                                                                                                                                                                                                                                                                               |  | ●    | ●    | ●        | ●      |
| <p>562.020<br/><b>Hand fixation by wrist cuff, size S (piece)</b></p>                                                                                                                                                                                                                                                                                                                                                                                                                               |  | ●    | ●    | ●        | ●      |

Basic equipment     Accessories     Model-dependent equipment
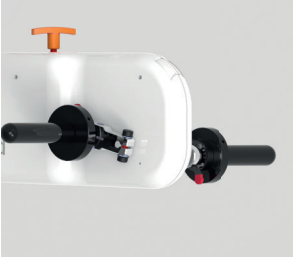
|                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                           | muvi                             | loop                             | gracile12 | layson                           |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------|----------------------------------|-----------|----------------------------------|
|  <p><b>265.100</b><br/><b>Soft Grip handles (pair)</b></p> <ul style="list-style-type: none"> <li>• Sturdy handles, soft padded and non-slip</li> </ul> <p><input type="radio"/> loop   <input type="radio"/> loop light   <input type="radio"/> loop parkinson   <input type="radio"/> loop kidz</p> <p><b>265.110</b><br/><b>Soft Grip handles with quick release system (pair)</b></p>                                                                                                                                                |                                  | <input checked="" type="radio"/> |           |                                  |
|  <p><b>265.120</b><br/><b>Plastic-coated handles with quick release system (pair)</b></p> <ul style="list-style-type: none"> <li>• Stable handles coated with disinfectable soft PVC</li> <li>• Easy to disinfect, recommended for use in rehabilitation facilities</li> </ul> <p><input type="radio"/> muvi   <input checked="" type="radio"/> loop   <input checked="" type="radio"/> loop light   <input checked="" type="radio"/> loop parkinson   <input checked="" type="radio"/> loop kidz   <input type="radio"/> loop prof</p> | <input checked="" type="radio"/> | <input checked="" type="radio"/> |           |                                  |
|  <p><b>261.524</b><br/><b>Plastic-coated handles with quick release system (pair)</b></p> <ul style="list-style-type: none"> <li>• Extended arm handle specially designed for lying patients</li> </ul> <p><input type="radio"/> layson.la   <input checked="" type="radio"/> layson kidz.la   <input type="radio"/> layson prof</p> <p><b>on request</b></p>                                                                                                                                                                          |                                  |                                  |           | <input checked="" type="radio"/> |



○ Basic equipment   ● Accessories   ◐ Model-dependent equipment

| muvi | loop | gracile12 | layson |
|------|------|-----------|--------|
|      |      |           | ◐      |
|      | ●    |           |        |
| ●    | ●    |           | ●      |
|      |      | ●         |        |



**261.525**  
**Plastic-coated pediatric handles with quick release system (pair)**  
 • Extended arm handle specially designed for children in supine position

● layson.la   ○ layson kidz.la   ● layson prof



on request



**555.000**  
**Tetra handles (pair)**  
 • For easy and independent insertion of the arms  
 • With forearm support, which can be adjusted flexibly  
 • Especially suitable for tetraplegics and partly for stroke patients








**555.200**  
**Tetra handles with quick release system (pair)**



**555.020**  
**Pediatric tetra handles (pair)**  
 • For easy and independent insertion of the arms  
 • With forearm support, which can be adjusted flexibly  
 • Especially suitable for tetraplegics and partly for stroke patients



○ Basic equipment   ● Accessories   ◐ Model-dependent equipment

|                                                                                                                                                                              |                                                                                                                                                                                                                                                                                                                                  | muvi | loop | gracile12 | layson |
|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------|------|-----------|--------|
|                                                                                             | <b>557.000</b><br><b>Vertical handles with hand rests (pair)</b> <ul style="list-style-type: none"> <li>• For users who can hold on to something but require a hand rest</li> <li>• With slip protection on the inside</li> </ul>                                                                                                |      | ●    |           |        |
|                                                                                                                                                                              | <br><b>557.200</b><br><b>Vertical handles with hand rests and quick release system (pair)</b>                                                                                                                                                   | ●    | ●    |           | ●      |
|                                                                                            | <b>557.020</b><br><b>Pediatric vertical handles with hand rests (pair)</b> <ul style="list-style-type: none"> <li>• For users who can hold on to something but require a hand rest</li> <li>• With slip protection on the inside</li> </ul>                                                                                      |      |      | ●         |        |
|                                                                                           | <b>372.000</b><br><b>Plastic-coated ergo handles (pair)</b> <ul style="list-style-type: none"> <li>• User can choose from 3 handle holding positions</li> <li>• Enables an individual hand position without equipment change</li> <li>• Recommended for use in rehabilitation facilities</li> <li>• Easy to disinfect</li> </ul> |      | ●    |           |        |
| <br><b>372.200</b><br><b>Plastic-coated ergo handles with quick release system (pair)</b> | ●                                                                                                                                                                                                                                                                                                                                | ●    |      | ●         |        |

○ Basic equipment   ● Accessories   ○ Model-dependent equipment

| muvi | loop | gracile12 | layson |
|------|------|-----------|--------|
| ●    | ●    |           | ●      |
|      |      | ●         |        |
|      |      | ●         |        |



356.200

**Plastic-coated forearm shells with disinfectable fastening straps and quick release system (pair)**

- Ergonomically shaped and padded, 28.0 cm long
- For fastening and guiding the arms in case of severe paralysis
- Lateral evasive movement of the forearms is possible
- Recommended for use in rehabilitation facilities (easy to disinfect)



556.024

**Pediatric forearm shells with arm cuffs, size S (pair)**

- Arm fixation and guidance in case of severe paralysis
- Lateral evasive movement of the forearms is possible
- Ergonomically shaped and padded, 21.0 cm long
- Standard with special cross handle for children
- Cuffs supplied as standard in size S



556.023

**Pediatric forearm shells with arm cuffs, size XS (pair)**



○ Basic equipment   ● Accessories   ◐ Model-dependent equipment



**556.006**  
**Forearm shells with arm cuffs, size L (pair)**  
 • Ergonomically shaped and padded, 28.0 cm long  
 • For fastening and guiding the arms in case of severe paralysis  
 • Lateral evasive movement of the forearms is possible  
 • With cross handle as standard  
 • Cuffs supplied as standard in size L

**556.003**  
**Forearm shells with arm cuffs, size XS (pair)**

**556.004**  
**Forearm shells with arm cuffs, size S (pair)**

**556.005**  
**Forearm shells with arm cuffs, size M (pair)**

**556.007**  
**Forearm shells with arm cuffs, size XL (pair)**

|         | muvi | loop | gracile12 | layson |
|---------|------|------|-----------|--------|
| 556.006 |      | ●    |           |        |
| 556.003 |      | ●    |           |        |
| 556.004 |      | ●    |           |        |
| 556.005 |      | ●    |           |        |
| 556.007 |      | ●    |           |        |

MUVI

LOOP

GRACILE12





LAYSON

ACCESSORIES

UPGRADES

## Other accessories

○ Basic equipment   ● Accessories   ◐ Model-dependent equipment

|                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                               |  | muvi | loop | gracie12 | layson |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|------|------|----------|--------|
|  <p>261.523<br/><b>Docking station for removable operating panel</b></p> <ul style="list-style-type: none"> <li>• Holder for attaching the removable operating panel</li> <li>• Plastic-coated for easier disinfection</li> <li>• Magnetic fixation</li> </ul>                                                                                                                              |  |      |      |          | ●      |
|  <p>100.000<br/><b>Wheelchair stabilizer</b></p> <ul style="list-style-type: none"> <li>• Recommended for severe spasticity</li> <li>• Supports the wheelchair: prevents tipping and slipping backwards</li> <li>• For all common wheelchairs with push handle</li> <li>• With padded support for the wheelchair push handles</li> <li>• Width of the handle support: 580 mm</li> </ul>  |  | ●    | ●    |          |        |
| <p>100.020<br/><b>Wheelchair stabilizer for children</b></p> <ul style="list-style-type: none"> <li>• Especially designed for children's wheelchairs with push handle</li> <li>• Width of the handle support: 330 mm</li> </ul>                                                                                                                                                                                                                                                                                                                               |  |      | ●    | ●        |        |

○ Basic equipment   ● Accessories   ◐ Model-dependent equipment

|  |                                                                                                                                                                                                                                                                                                               | muvi | loop | gracile12 | layson |
|--|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------|------|-----------|--------|
|  | <p>587.100<br/><b>Flat stand (flat steel)</b></p> <ul style="list-style-type: none"> <li>• For easy driving over with sports wheelchairs, electric wheelchairs and wheelchairs with rigid footrest</li> </ul>                                                                                                 |      |      | ●         |        |
|  |                                                                                                                                                                                                                                                                                                               |      |      |           |        |
|  | <p>308.100<br/><b>Belt stabilizer comfort</b></p> <ul style="list-style-type: none"> <li>• Incl. front stand</li> <li>• Incl. suspension rod and release hook</li> <li>• Easy to operate from a wheelchair without an assistant</li> <li>• Flexible application for different types of wheelchairs</li> </ul> | ●    |      |           |        |
|  | <p>265.005<br/><b>Belt stabilizer</b></p>                                                                                                                                                                                                                                                                     |      | ●    |           |        |









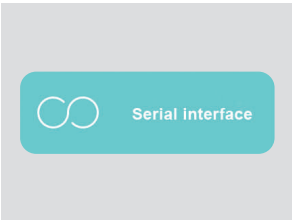



MOTOmed®  
PRODUCT  
CONFIGURATION

# MOTomed® product configuration

Product configurations **cannot** be retrofitted at a later date.

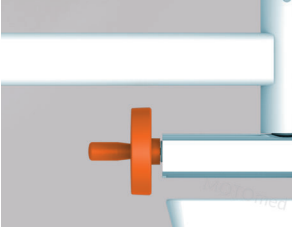Basic equipment     Optional product configuration     Model-dependent product configuration

|                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                          | muvi                  | loop                             | gracile12                        | layson                           |
|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------|----------------------------------|----------------------------------|----------------------------------|
|  <p><b>265.001</b><br/><b>Special stand with reduced stand height</b></p> <ul style="list-style-type: none"> <li>• For easier starting with sports wheelchairs, electric wheelchairs and wheelchairs with rigid footrest</li> <li>• Height of standard stand: 72 mm (mid-foot),<br/>Height of special stand: 30 mm</li> </ul> <p><input checked="" type="radio"/> loop</p>                                                                                                                                                             |                       | <input checked="" type="radio"/> |                                  |                                  |
|  <p><b>265.002</b><br/><b>Gas spring support for height adjustment</b></p> <ul style="list-style-type: none"> <li>• Infinitely height adjustable, no effort required</li> </ul> <p><input checked="" type="radio"/> loop   <input checked="" type="radio"/> loop light   <input checked="" type="radio"/> loop parkinson   <input checked="" type="radio"/> loop kidz   <input type="radio"/> loop prof</p>                                                                                                                         |                       | <input checked="" type="radio"/> |                                  |                                  |
|  <p><b>261.501</b><br/><b>Knee bending adjustment incl. adjustment wheel</b></p> <ul style="list-style-type: none"> <li>• To adjust the strength of knee flexion</li> <li>• For setting the ideal position without repositioning the patient</li> </ul> <p><input type="radio"/> layson (.la)   <input checked="" type="radio"/> layson (.l)   <input type="radio"/> layson kidz (.la)   <input checked="" type="radio"/> layson kidz (.l)</p> <p><input type="radio"/> layson dia   <input type="radio"/> layson prof</p>         |                       |                                  |                                  | <input checked="" type="radio"/> |
|  <p><b>261.520</b><br/><b>Removable operating panel</b></p> <ul style="list-style-type: none"> <li>• The user can operate the MOTomed independently</li> <li>• Cable with rewinder, length of unrolled cable approx. 2.9 m</li> </ul> <p><input checked="" type="radio"/> layson   <input checked="" type="radio"/> layson kidz   <input type="radio"/> layson dia   <input type="radio"/> layson prof</p>                                                                                                                         |                       |                                  |                                  | <input checked="" type="radio"/> |
|  <p><b>265.500</b><br/><b>Activation of serial interface</b></p> <ul style="list-style-type: none"> <li>• Activation of serial interface installed under the colour touch display</li> <li>• For transfer of device parameters and training data</li> </ul> <p><input checked="" type="radio"/> loop*   <input checked="" type="radio"/> loop parkinson   <input checked="" type="radio"/> loop kidz   <input checked="" type="radio"/> layson prof</p> <p><b>214.000</b><br/><b>Serial interface RS232 MOTomed gracile12</b></p>  | <input type="radio"/> | <input checked="" type="radio"/> | <input checked="" type="radio"/> | <input checked="" type="radio"/> |

\* not for MOTomed loop light models





## MOTomed® UPGRADES



### Digital intelligence

To meet new needs, users have a total of **61 upgrades** available to enhance their leg or arm/upper body trainer, arm/upper body trainer and leg trainer. The MOTomed loop can also be adapted to new indications via a software package (e.g. Parkinson's disease).

## MOTomed® upgrade bundles

Our motion trainers of the MOTomed loop edition can be adapted with upgraded bundles to meet changed requirements. Thanks to the use of different attachments and software packages, almost every MOTomed loop can be upgraded. All extensions can be found in the [upgrade table on page 60](#), or contact RECK (tel: +49 7374 18-85) or a specialist dealer you trust.

### Frequently used MOTomed loop upgrade bundles:



260.211

**MOTomed loop.l** >>> **MOTomed loop.la**

Included in the upgrade bundle: software package, Soft Grip handles (pair, incl. pedal cranks)



260.251

**MOTomed loop light.la** >>> **MOTomed loop.la**

Included in the upgrade bundle: software package, plastic-coated safety foot shells with fastening straps (pair, incl. pedal cranks)



260.235

**MOTomed loop.la** >>> **MOTomed loop p.la**

Included in the upgrade bundle: software package



260.295

**MOTomed loop kidz.a** >>> **MOTomed loop p.la**

Included in the upgrade bundle: software package, plastic-coated safety foot shells with fastening straps (pair, incl. pedal cranks)

# MOTOmed® loop upgrades

## Your model

|                          |                |                 |                |                      |                       |                      |  |
|--------------------------|----------------|-----------------|----------------|----------------------|-----------------------|----------------------|--|
| MOTOmed loop.l           |                | ✓<br>260.211    |                |                      | ✓<br>260.213          |                      |  |
| MOTOmed loop.la          | ✓<br>260.230   |                 | ✓<br>260.232   |                      | ✓<br>260.233          |                      |  |
| MOTOmed loop.a           |                | ✓<br>260.221    |                |                      | ✓<br>260.223          |                      |  |
| MOTOmed loop light.l     |                |                 |                |                      |                       |                      |  |
| MOTOmed loop light.la    | ✓<br>260.250   | ✓<br>260.251    | ✓<br>260.252   |                      |                       |                      |  |
| MOTOmed loop light.a     |                |                 |                |                      |                       |                      |  |
| MOTOmed loop p.l         | ✓<br>260.240   | ✓<br>260.241    |                |                      | ✓<br>260.243          |                      |  |
| MOTOmed loop p.la        | ✓<br>260.260   | ✓<br>260.261    | ✓<br>260.262   |                      | ✓<br>260.263          |                      |  |
| MOTOmed loop kidz.l      | ✓<br>260.270   | ✓<br>260.271    |                |                      | ✓<br>260.273          |                      |  |
| MOTOmed loop kidz.la     | ✓<br>260.280   | ✓<br>260.281    | ✓<br>260.282   |                      | ✓<br>260.283          |                      |  |
| MOTOmed loop kidz.a      |                | ✓<br>260.291    | ✓<br>260.292   |                      | ✓<br>260.293          |                      |  |
| MOTOmed loop.la prof     |                |                 |                |                      |                       |                      |  |
| <b>Desired upgrade →</b> | MOTOmed loop.l | MOTOmed loop.la | MOTOmed loop.a | MOTOmed loop light.l | MOTOmed loop light.la | MOTOmed loop light.a |  |



|  |                  |                   |                     |                      |                     |
|--|------------------|-------------------|---------------------|----------------------|---------------------|
|  | ✓<br>260.214     | ✓<br>260.215      | ✓<br>260.216        | ✓<br>260.217         |                     |
|  | ✓<br>260.234     | ✓<br>260.235      | ✓<br>260.236        | ✓<br>260.237         | ✓<br>260.238        |
|  |                  | ✓<br>260.225      |                     | ✓<br>260.227         | ✓<br>260.228        |
|  |                  |                   |                     |                      |                     |
|  | ✓<br>260.254     | ✓<br>260.255      | ✓<br>260.256        | ✓<br>260.257         | ✓<br>260.258        |
|  |                  |                   |                     |                      |                     |
|  |                  | ✓<br>260.245      | ✓<br>260.246        | ✓<br>260.247         |                     |
|  | ✓<br>260.264     |                   | ✓<br>260.266        | ✓<br>260.267         | ✓<br>260.268        |
|  | ✓<br>260.274     | ✓<br>260.275      |                     | ✓<br>260.277         |                     |
|  | ✓<br>260.284     | ✓<br>260.285      | ✓<br>260.286        |                      | ✓<br>260.288        |
|  |                  | ✓<br>260.295      |                     | ✓<br>260.297         |                     |
|  |                  | ✓<br>260.239      |                     |                      |                     |
|  | MOTOmed loop p.l | MOTOmed loop p.la | MOTOmed loop kidz.l | MOTOmed loop kidz.la | MOTOmed loop kidz.a |

Upgrade


For detailed information and pricing on individual upgrade bundles, please contact RECK (Tel: +49 7374 18-85) or your trusted dealer.

# MOTOmed Service



## Competent advice

The MOTOmed consulting team will do everything in its power to adapt the device to your individual needs to ensure you have the best possible training.

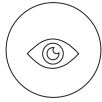

## Demonstration

The MOTOmed consulting team will gladly organize a demonstration together with our partner in your country.



## Rent

The MOTOmed consulting team will be pleased to inform you about the different options for purchase or rent. Please contact us for further information. We will be happy to advise you.



## Studies

On a scientific level, the effectiveness of MOTOmed movement therapy has been demonstrated in different clinical pictures. You are welcome to take a look at more than 30 studies and scientific research results.



## Testimonials

What experiences and successes have patients and facilities had using MOTOmed movement therapy? Give us a call. We will send you reports that are relevant to the corresponding indication free of charge.

**Inform yourself – [www.motomed.com](http://www.motomed.com)**


MADE  
IN  
GERMANY



We constantly adapt our products to the latest state of the art in science and technology. Therefore, there may be differences to the illustrated device and equipment. Misprints and errors excepted. Colour deviations may occur due to printing technology.

Copyright by RECK-Technik GmbH & Co. KG. All rights reserved. No part of this publication may be reproduced in any form or by any means without the written permission of RECK.

MOTOmed has also been tested in accordance with DIN EN 60601-1-11 and thus meets the special requirements for medical electrical equipment for care in domestic environments.

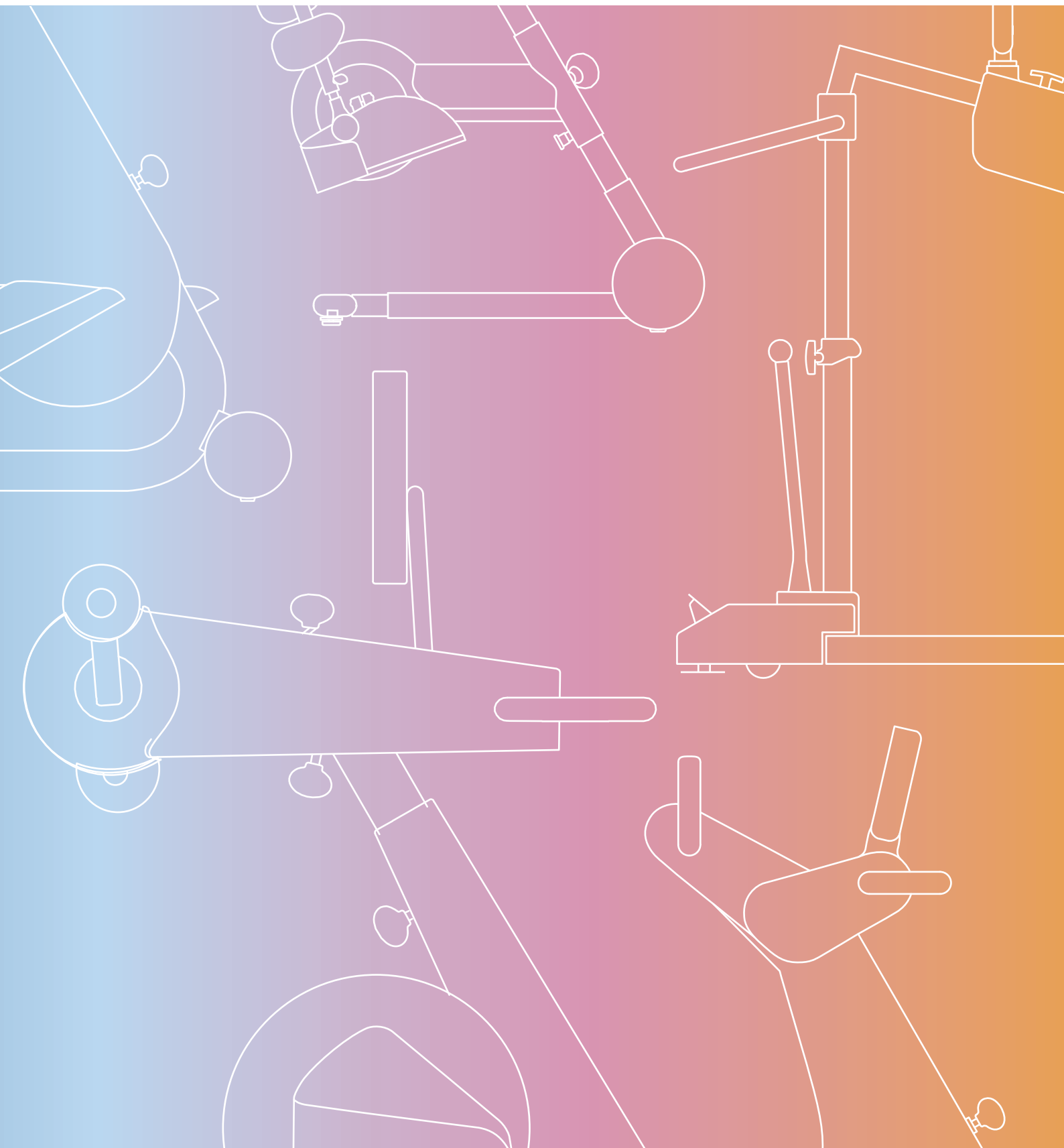

## **RECK**

RECK-Technik GmbH & Co. KG  
Reckstrasse 1-5  
88422 Betzenweiler  
GERMANY

Phone +49 7374 18-85  
Fax +49 7374 18-480  
info@motomed.com  
www.motomed.com

en  
IDN 100.019.430  
As of: 20200304