

## SOFT RAISED TOILET SEAT

Reference	Description
09/7300	Without lid
09/7301	With lid



### Advantages :

- Outstanding stability.
- Easy to use, quick fitting with no fixing kits.
- Softness of the foam.

### Instruction :

- Before installation, make sure the toilet bowl is well maintained.



For the model with a lid, do not remove the **Rehosoft...NG** from packaging by holding the lid.

- Installation : Simply place the raised toilet seat on the toilet bowl. The **Rehosoft...NG** fits perfectly into a large majority of bowl.
- The outstanding stability of the **Rehosoft...NG** provides an optimal safety.

### Description :

- Size : 38 wide x 40,5 long x 14 cm high / 15" x 16" x 6"
- User raising height: 11 cm / 4"
- Net weight without lid : 2,4 kg / 4,4 lbs, Net weight with lid: 3 kg / 6,6 lbs
- Max. user weight: 120 kg / 19 st

### Composition :

- Surface in Polyvinyl chloride (PVC),
- Injected Polyurethane foam,
- Polypropylene lid

### Cleaning instruction :

- Cleaning : Soapy hot water and all usual products.
- Disinfection : Chlorine dilution and disinfectant aerosol.

### Precautions for use :

- Do not immerse the rear of **Rehosoft...NG**.
- Avoid contact with coloured products as much as possible.
- PharmaOuest Industries cannot be held responsible for any damage related to misuse of the product.

### Packaging :

- By unit, in box : 43 x 44,5 x 19,5 cm / 17" x 17" x 7"
- Gross weight without lid: 2,8 kg / 4,4 lbs, Gross weight with lid: 3,4 kg / 6,6 lbs.