



● Ø 1-3mm | ○ Ø 2-6mm
 Power: ■ | Power: ■■

With better control of procedures the CryoPen M sets new standards for superb outcomes.

The CryoPen M dispenses a fine jet of nitrous oxide under high pressure which allows you to work with millimeter precision.

It offers the ideal solution for accurate, quick, effective stress free treatment of all skin imperfections.

Instrument

This instrument is well prepared for all treatment opportunities where delicate freezing power is required.

Cartridges

8g N₂O cartridges: 1 box containing 24 cartridges
1 carton containing 12 boxes

Once punctured the pressure will remain for weeks. Shelf life 36 months. Storage at room temperature.

CryoPen M set with 2 applicators

Blue/red dot applicator: Ø 1-3mm
flow time 135-155sec (8g)

White/red dot applicator: Ø 2-6mm
flow time 80-100sec (8g)

What kind of lesions?

Acrochordon, actinic keratosis and cheilitis, capillary haemangioma, cutaneous leishmaniasis, discoid lupus erythematosus, filiform, flat, vulgar, plantar and anogenital warts, rosacea, lichen sclerosus, granuloma annulare, molluscum contagiosum, seborrheic keratosis and viral lesions.

About the procedure

No special preparation is required. Treatment is applied for 2 to 25 seconds on each lesion. In larger areas circular or crossing movement is required to include all tissue.