Note:
The Head for the massage gun comes with a lithium-ion battery, please make sure to charge the battery before use with the included USB-A to USB-C cable.

Please read the instructions carefully before using the Massage Head.

Product specifications

Battery capacity	800 mAh
Battery type	Rechargeable lithium-ion battery, non-replaceable
Charging time	2.5 hours
Applicable charger	5V / 2A charger
Heat power	3.7W
Cool power	4.5W
Product size	9.1 x 4.5 x 4.5 cm 3.6 x 1.8 x 1.8 inch
Product weight	120 g 0.26 lb
Maximum temperature (Heat)	Level 1: 40°C Level 2: 45°C
Maximum temperature (Cool)	Level 1: 20°C Level 2: 15°C
Working time (Heat)	Level 1: 2.5 hours Level 2: 2 hours
Working time (Cool)	Level 1: 1 hour Level 2: 40 min

Precautions before use

To reduce the risk of malfunction, electric shock, injury, fire, death, and equipment damage, please read all the instructions and warnings in this manual carefully before using this product.

- Keep away from liquids or heat sources, and keep the vents free of dust.
- After charging is complete, please unplug the power cord.
- Don't use this product while charging.
- Please use this product according to the instructions.
- Before each use, carefully check whether the product can be used normally according to the operation methods of this manual.


WARNING

This product cannot be used under any of the following situations without the prior consent of the doctor: Pregnancy, diabetic complications (such as neuropathy or retinal injury), wearing a pacemaker, recent surgery, epilepsy or migraine, herniated disc, diastasis vertebrae, or spondylitis, recent joint replacement surgery or IUD, metal needle or steel plate.

Forbiddance

- It is forbidden for children under the age of 10 to use it. If you are injured, do not use this product. Please consult your doctor before using this product when your muscles are damaged.
- It is forbidden to use on the surface of non-dry, non-clean skin. You can use it across clothes, gently press and move on the skin, each part for about 60 seconds.
- It is forbidden to use when bruising, pain, or discomfort appears.
- It is forbidden to put any objects into the ventilation holes of the motor.
- It is forbidden to immerse the product in water.
- It is forbidden to disassemble this product in any way.
- Forbid fingers, hair, or other parts of the body near the axis and back of the massage head to prevent pinch and injury.

Product parts

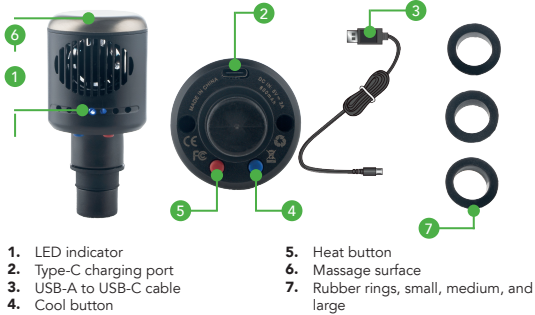- LED indicator
- Type-C charging port
- USB-A to USB-C cable
- Cool button
- Heat button
- Massage surface
- Rubber rings, small, medium, and large

Instructions

The button is divided into two buttons: blue and red.

- Blue is the cool button and red is the hot button. Long press the red button for 2 seconds to enter the first level and press it again to enter the second level. Long press the blue button for 2 seconds to enter the first level and press it again to enter the second level.
- Long press one of the buttons for 2 seconds to shut down the device.
- When the power is low, the LED indicator will flash red and then switches off. The red light will flash when charging and will shine steadily blue when fully charged.
- The LED indicator will shine blue for Cool level 1 and will shine more bright for Cool level 2.
- The LED indicator will shine red for Heat level 1 and will shine more bright for Heat level 2.
- The device will automatically turn off after 15 minutes of usage for safety.
- The massage surface can be cleaned with a damp cloth, but make sure to switch off the product and don't use any detergents.

Compatibility

Please scan below QR code to see which massage guns are compatible with this massage head:

Disclaimer

© 2023 Tunturi New Fitness BV
All rights reserved. The product and the manual are subject to change. Specifications can be changed without further notice.

Tunturi New Fitness B.V. / P.O. Box 60001 / 1320 AA Almere
The Netherlands / www.tunturi.com

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