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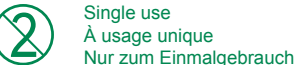
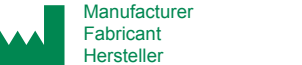
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
2797

PK7659080 10/2019

BARD® FLIP-FLO™ Catheter Valve Directions for use

Warning: This product should not be used without assessment of bladder function by an appropriate medical professional.

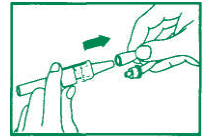
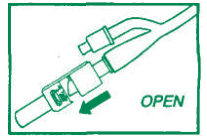
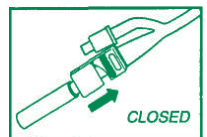
Indications

For use with an indwelling catheter.

Contraindications

- Reduced bladder capacity.
- Cognitive impairment.
- No bladder sensation.
- Insufficient manual dexterity to operate the FLIP-FLO™ valve.

Instructions

- Wash hands.
- Attach FLIP-FLO™ valve to catheter ensuring that you do not touch either the catheter end or the valve connector.
- Wash hands thoroughly before and after operating FLIP-FLO™ valve.
- Empty bladder as frequently as advised by your doctor or nurse.
- To open the FLIP-FLO™ valve, push lever down.
- Allow urine to drain into toilet or an appropriate receptacle.
- To close the FLIP-FLO™ valve, pull lever up.
- A night drainage bag may be attached to the flexible connector to allow continuous urine drainage overnight.

WARNING — leave FLIP-FLO™ valve in open position.
 It is recommended that the FLIP-FLO™ valve is changed every 5-7 days. Simply disconnect the valve from the catheter and discard. Attach a new valve following the above instructions.

Signature Manifest

Document Number: BAW7659080

Revision: 0

Title: Bard Flip-Flo Catheter Valve DFU

All dates and times are in Eastern Standard Time.

BX CR-21892 DC14160 IFUs / DFUs

Doc Control Check

Name/Signature	Title	Date	Meaning/Reason
Brooke Winfield (BWINFIELD)	Division Labeling Coordinator	04 Nov 2019, 08:44:13 AM	Approved

Assign Approvers/Modify

Name/Signature	Title	Date	Meaning/Reason
Brooke Winfield (BWINFIELD)	Division Labeling Coordinator		
Rebecca Harris (RHARRIS)	Supvr, Labeling Documentation	05 Nov 2019, 09:28:06 AM	Complete
Dee Swarns (DSWARNS)	Supervisor, Labeling	11 Nov 2019, 01:16:08 PM	Complete

Review/Approve

Name/Signature	Title	Date	Meaning/Reason
Dee Swarns (DSWARNS)	Supervisor, Labeling	11 Nov 2019, 01:16:25 PM	Approved
Brooke Winfield (BWINFIELD)	Division Labeling Coordinator	11 Nov 2019, 02:23:40 PM	Approved
Manuel Martinez (MMARTINEZ2)	Label Design Specialist	11 Nov 2019, 06:14:23 PM	Approved
Erick Davila (EDAVILA)	Label Design Specialist	12 Nov 2019, 10:01:53 AM	Approved
Zuilma Villela (ZVILLELA)	Engineer	13 Nov 2019, 03:24:56 PM	Approved

DC Final Review

Name/Signature	Title	Date	Meaning/Reason
Dee Swarns (DSWARNS)	Supervisor, Labeling	14 Nov 2019, 04:48:36 PM	Complete

CR Final Release

Name/Signature	Title	Date	Meaning/Reason
Dee Swarns (DSWARNS)	Supervisor, Labeling	14 Nov 2019, 04:48:57 PM	Approved