


Sofort-Kalt-Kompresse

Instant cold pack

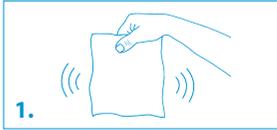

1.
Inhalt nach unten schütteln.
Shake contents to bottom of bag.



2.
Inneren Beutel fest umklammern
und zum Platzen bringen.
Locate inner pouch and squeeze
firmly to break.



3.
Beutelinhalt durch Schütteln des
Beutels mischen.
Shake well to mix contents.

Für schnelle Kühlung bei stumpfen Verletzungen, zur Vermeidung und Reduzierung von Schwellungen und zur Schmerzlinderung. Direkten Hautkontakt vermeiden. Zum Schutz vor Kälteverbrennung, Kompresse unbedingt in ein Tuch wickeln. Nur zur äußerlichen Anwendung. Nicht länger als 20 Minuten anwenden. Nicht benutzen bei Beschädigung des Beutels.

Gegenanzeige: Raynaud-Krankheit, Kälte-Allergie, periphere arterielle Verschlusskrankheit, rheumatoide Arthritis, Sensibilisierungsstörung der betroffenen Stelle, unbehandelte Hypertonie. Anwendung bei Herz-, Kreislaufkrankung und Diabetes Mellitus nach Rücksprach mit dem Arzt.

Warnung: Bei Verschlucken des Inhalts sofort einen Arzt konsultieren. Sollte die Lösung mit Augen, Haut oder offener Wunde in Berührung kommen, sofort gründlich mit Wasser abspülen und im Zweifel einen Arzt konsultieren. Vor Kindern sicher aufbewahren.

Inhalt: Urea, Wasser.

The product is for obtaining rapid cooling effect in cases of obtuse injuries, for the avoidance and reduction of swelling and for the easing of pain. Avoid direct skin contact. To prevent frostbite wrap pack in a soft cloth. For external use only. Do not apply for more than 20 minutes at a time. Discard if punctured or leaking.

Contraindications: Raynaud's disease, cold allergy, peripheral artery occlusive disease, rheumatoid arthritis, sensory impairment of the affected area, untreated hypertension. Patients suffering from cardiovascular disease and diabetes mellitus may only use the product following consultation with their doctor.

Warning: If swallowed seek urgent medical advice. If solution contacts eyes, skin or open wound flood with water and in case of doubt, seek medical advice. Keep out of reach of children.

Contents: Urea, Water.

REF 93.000.00.048

LOT



Medizinprodukt
Medical Device



P. J. Dahlhausen & Co. GmbH
Emil-Hoffmann-Straße 53 · 50996 Köln · Germany
Tel.: +49 (2236) 3913-0 · Fax: +49 (2236) 3913-109
info@dahlhausen.de · www.dahlhausen.de

Rev. 05/2020-10

UDI

(01)04034342073503



PZN - 07234273