



# Garment Measuring Guide

Official dealer **Huntleigh**




# Garment Measuring Guide

The following guide is intended as an aid to help the user select the correct size garment.

When determining the correct size garment:

- Remove any bandaging, stockings or bulky dressings
- Measure the points as shown on the diagram below
- DO NOT pull the measuring tape tight
- Make sure the garment extends beyond the area under treatment.

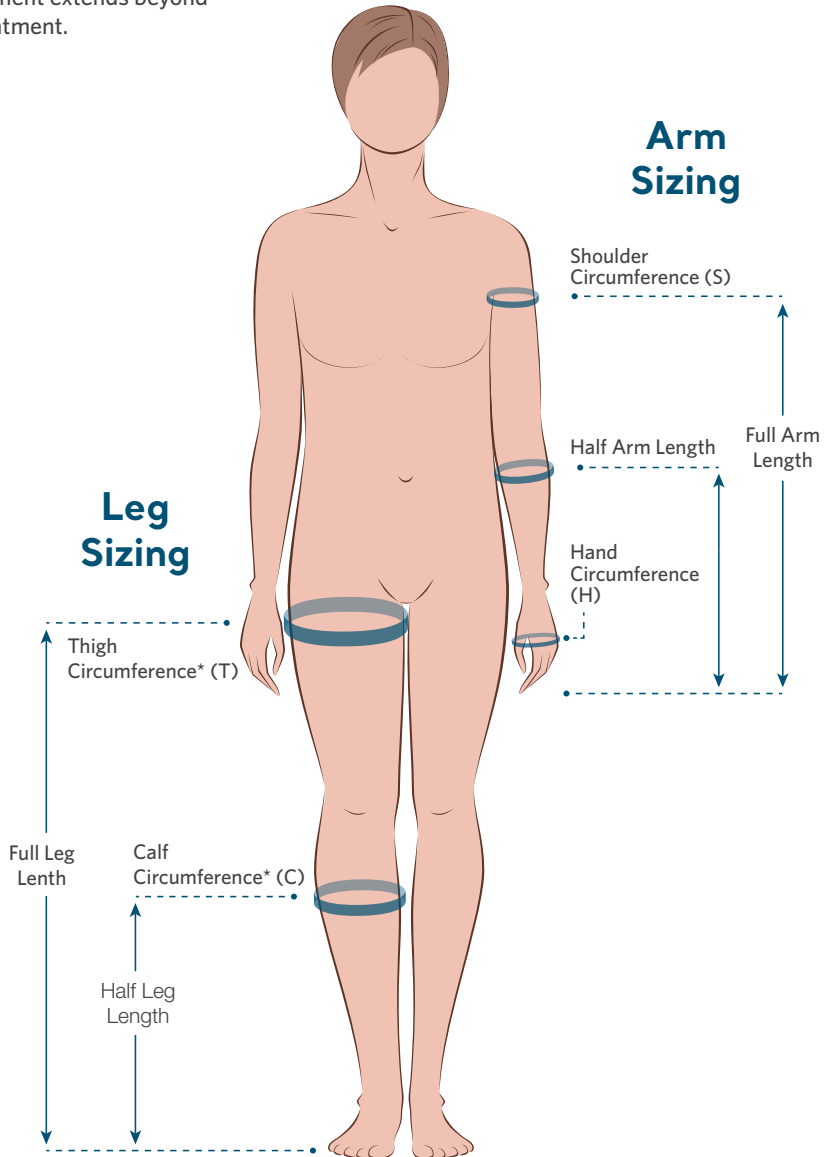

\*Measurements should be made at the widest point on the calf and thigh.

## Lymph Assist (Hydroven 12) Arm

Twelve Chamber Arm Garments



Order Code	Type	Length (L)	Shoulder Circumference (S)	Hand Circumference (H)
1533523	Full Arm	68cm	62cm	44cm
1533524	Full Arm	78cm	62cm	44cm

## Lymph Assist (Hydroven 12) Arm Insert

Twelve Chamber Arm Inserts



Order Code	Type	Length (L)	Additional Width at Shoulder	Additional Width at Hand
1533527	Full Arm	68cm	17cm	13cm
1533528	Full Arm	78cm	17cm	13cm

## Lymph Assist (Hydroven 12) Leg

Twelve Chamber Leg Garments



Order Code	Type	Length (L)	Thigh Circumference (T)	Calf Circumference (C)
1533519	Standard Leg	76cm	71cm	59cm
1533520	Standard Leg	84cm	71cm	59cm
1533521	Wide Leg	76cm	79cm	62cm
1533522	Wide Leg	84cm	79cm	62cm

## Lymph Assist (Hydroven 12) Leg Insert

Twelve Chamber Leg Inserts



Order Code	Type	Length (L)	Additional Width at Thigh	Additional Width at Foot
1533525	Leg Insert	76cm	19cm	14cm
1533526	Leg Insert	84cm	19cm	14cm

Official Huntleigh distributor



Arseus Medical  
Rijksweg 10 2880 Bornem - Belgium

0800 76 769  
info.hospital@arseus-medical.be  
www.arseus-medical.be

As a proud member of the Arjo family, we have been committed to supporting healthcare professionals in improving outcomes and enhancing patient wellbeing since 1979. We do this through our proven solutions for Vascular Assessment & Treatment and Fetal & Patient Monitoring. With innovation and customer satisfaction as our guiding principles, we strive for clinical excellence and improved performance, for life.

**Manufactured and distributed by Huntleigh Healthcare Ltd.**  
35 Partmanmoor Road, Cardiff, CF24 5HN, United Kingdom  
T: +44 (0)29 20485885 sales@huntleigh-diagnostics.co.uk  
www.huntleigh-diagnostics.com

Registered No: 942245 England & Wales. Registered Office:  
ArjoHuntleigh House, Houghton Hall Business Park, Houghton Regis, Bedfordshire, LU5 5XF  
©Huntleigh Healthcare Limited 2019



**A Member of the Arjo Family**  
® and ™ are trademarks of Huntleigh Technology Limited  
As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice.

AW: 1001043-2

**HUNTLEIGH**

LIT 779384/EN-5