Instructions for Use Gebruiksaanwijzing Mode d'emploi Gebrauchsanweisung

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English – Instructions for Use

Intended Use

The Fico Soft Fix nursing blanket is designed for professional use in hospitals, nursing homes, residential care facilities and other healthcare institutions, and may also be used in a supervised home-care setting.

The product is intended to temporarily and gently limit movement of agitated or care-dependent patients, with the aim of promoting comfort, safety and calmness, without active fixation, and always under the supervision of a healthcare professional or caregiver.

The **FR-TR versions** are specifically developed for environments with increased requirements regarding fire safety and tear resistance.

The **2-piece version** has been developed to increase the safety of both the patient and the care institution. Thanks to its separated construction and the anchoring of one component to the bed frame, the risk of pulling or tipping the mattress is significantly reduced. This makes the 2-piece version particularly suitable for more agitated patients and for care situations with an increased risk of falls or injury.

Product Description

Outer surface: Technical textile suitable for intensive professional use

Versions:

- Soft Fix nursing blanket
- Soft Fix nursing blanket FR-TR ((fire-retardant and tear-resistant))
- Soft Fix 2-piece nursing blanket FR-TR ((anchorable – fire-retardant and tear-resistant))

Characteristics:

- Soft, elastic and comfortable for the patient
- Durable and wear-resistant
- FR-TR version: fire-retardant and extra tear-resistant
- CE-marked Class I medical device in accordance with MDR (EU 2017/745)

Indications and Contraindications

Indications

The Fico nursing blanket is intended for use in professional care environments to support:

- Comfortable and safe covering of the patient
- Promotion of calmness and a sense of security in agitated, confused or cognitively impaired patients
- Limitation of picking and smearing behaviour
- Reduction of removal or damage to clothing
- Limitation of unwanted undressing or lying naked
- Reduction of manipulation, loosening or removal of IV lines, feeding tubes or other medical devices
- Support in limiting unwanted standing up or wandering
- Fall prevention as part of a broader care and safety policy

Contraindications

The Fico nursing blanket must not be used in the following situations:

- In very aggressive or physically strong patients, where there is an increased risk of product damage or injury
- In patients with biting behaviour or other destructive behaviour directed at textiles
- In obese patients, or when there is insufficient freedom of movement at the level of the legs, arms, neck or torso
- (in these situations, a customised solution may be considered)
- In patients or young persons where excessive freedom of movement occurs at the level of the neck and arms, resulting in an increased risk of entanglement or unsafe use
- (in these situations, a customised solution may be considered)
- When used on a mattress construction that is too high, such as alternating pressure mattresses placed on top of a standard mattress
- (this must always be taken into account; customised solutions are possible)
- When the product cannot be correctly applied or adapted to the individual patient

- If the product is damaged, worn or incomplete

Important note

The use of the nursing blanket must always be individually assessed and carried out within the framework of applicable care and safety guidelines. In case of doubt regarding suitability or application, consultation with the care team is required.

Instructions for Use

1. Use the nursing blanket only for patients for whom gentle limitation is medically justified
2. Apply the blanket in accordance with the care protocols of the institution
3. Ensure that the patient always retains sufficient freedom of movement
4. Regularly check the comfort and position of the patient
5. Use the product only under supervision of healthcare personnel
6. Ensure the product is correctly and completely applied (especially for the 2-piece version)
7. Keep identification and product labels attached for traceability

Preparation

Always wash the nursing blanket before first use, in accordance with the washing instructions. This is necessary to allow for initial material and dimensional shrinkage.

Installation and Removal for Cleaning

Fico Soft Fix nursing blanket & nursing blanket FR-TR

Suitable for mattresses with the following dimensions:

- Width: 85 – 90 cm
- Length: 190 – 210 cm
- Mattress thickness: 14 – 18 cm

Installing the nursing blanket

1. Remove all existing blankets and sheets from the bed.
2. It is recommended to place a mattress cover or protector over the mattress for hygiene and patient comfort (not supplied with the product).
3. Place the nursing blanket centrally on the bed, with the head opening at the top.
4. Slightly lift the top of the mattress and pull the tucking flap of the blanket around the mattress, similar to fitting a mattress cover.
5. Then slightly lift the bottom of the mattress and pull the lower tucking flap around the mattress.
6. Smooth the blanket so that the seams align correctly with the corners of the mattress.
(Steps 5 and 6 may be easier when performed by two persons.)
7. Fully open the long zipper to ensure correct positioning and later access.
8. Check that the nursing blanket fits securely but without excessive tension around the mattress.

Removal for cleaning

9. Fully close the central long zipper.
10. First release the lower flap, followed by the upper flap, slightly lifting the mattress if required.
11. Remove the nursing blanket from the mattress.
12. Wash the nursing blanket according to the prescribed washing and maintenance instructions.
13. After each wash, inspect for wear, damage or defective zippers before reuse.

Installation and Removal for Cleaning

Fico Soft Fix 2-piece nursing blanket FR-TR

Suitable for mattresses with the following dimensions:

- Width: 85 – 90 cm
- Length: 190 – 210 cm
- Mattress thickness: 14 – 18 cm

Installing the 2-piece nursing blanket FR-TR

Placement of the under-sheet (anchoring component)

1. Remove all existing blankets and sheets from the bed.
2. Place the black under-sheet underneath the mattress, with the buckles facing downward.
(Optionally, the mattress may be removed from the bed to facilitate this step.)
3. Securely anchor the buckles of the under-sheet to the bed frame.

⚠ Attention – adjustable beds:

Anchoring must be carried out exclusively to the moving parts of the bed frame.

Anchoring to fixed or non-moving parts may lead to excessive tension, tearing of the under-sheet or buckles, and unsafe use.

Incorrect anchoring increases the risk to both patient and caregiver.

Damage or defects resulting from incorrect installation or anchoring are not covered by warranty.

The under-sheet may remain permanently underneath the mattress and does not need to be washed.

4. Adjust the length of the straps so that they are firmly tightened.
5. Place the mattress back on top of the under-sheet.
6. It is recommended to place a mattress cover or protector over the mattress for hygiene and patient comfort (not supplied with the product).

Placement of the top cover

7. Place the top cover centrally on the mattress, with the head opening at the top.
8. Zip the top cover to the under-sheet at the foot end of the bed. The zipper runs completely around the mattress edge.
9. Smooth the top cover so that the seams align correctly with the corners of the mattress.
10. Fully open the long zipper to ensure correct positioning and later access.
11. Check that the nursing blanket fits securely but without excessive tension.

Removal for cleaning (2-piece system)

12. Fully close the central zipper of the top cover.
13. Unzip and detach the top cover from the under-sheet.
14. Remove the top cover from the mattress.
15. Wash only the top cover according to the prescribed washing and maintenance instructions.
16. After each wash, inspect the top cover for wear, damage or defective zippers before reuse.
17. The under-sheet remains under the mattress and is not washed.
18. Important Safety Notice
19. Incorrect installation or anchoring of the 2-piece FR-TR nursing blanket may negatively affect the function, safety and durability of the product and may create risks for the patient and the care institution.

Cleaning and Maintenance

General guidelines

- Clean and maintain the product in accordance with the internal hygiene and laundry protocols of the care institution.
- Wash the product before first use to stabilise the material.
- Wash after each use or in accordance with infection-prevention guidelines.
- Close all zippers before washing to prevent damage.
- Do not wash together with sharp or heavy objects.
- Do not use aggressive cleaning agents that may damage the textile or its protective properties.

Washing instructions (step-by-step)

1. Remove the nursing blanket (or only the top cover for the 2-piece version) from the bed.
2. Inspect the product for visible damage before washing.
3. Fully close all zippers.
4. Machine wash at a maximum of 60 °C.
5. Use a mild, standard detergent.
6. Do not use bleach or chlorine-based products.
7. Tumble dry at low to moderate temperature (max. 40 °C).
8. Allow the product to cool and dry completely before reuse.
9. After each wash, inspect for:
 - wear
 - tears
 - damaged zippers or seams
10. Do not use the product if damage is detected; contact the supplier in time to discuss repair options.

Important notes

Incorrect cleaning or drying may result in:

- reduced service life
- deterioration of protective properties
- loss of product functionality
- Damage resulting from non-compliant washing or maintenance is not covered by warranty.

Warnings and Precautions

Not intended as an active fixation device

Do not use without medical indication or without supervision

Do not use in patients with an increased risk of entrapment or respiratory problems

Discontinue use immediately in case of discomfort, stress or skin irritation

Do not use damaged products

Always follow applicable guidelines regarding measures restricting freedom

Manufacturer

Fico Belgium bv

www.ficobelgium.eu

info@ficobelgium.eu

CE marking applied since: 2024