

Because one pair of hands is often not enough



The HUWA-SAN[®] FOGGER
disinfects everywhere - quickly and effectively.

Huwa-San[®]
A sustainable way of disinfection


Roam Technology
YOUR PARTNER IN HYGIENE CONTROL

What is Huwa-San[®]?

Huwa-San is a disinfectant based on silver stabilized **hydrogen peroxide** (H_2O_2). This is a colourless, odourless liquid. After it reacts, there is only water (H_2O) and oxygen (O_2) left, so no traces remain.

Huwa-San is a recognized biocide, proven to be effective against viruses, bacteria, spores and fungi, among others. This is the perfect solution in the fight against coronavirus.

Roam Technology distributes high-quality disinfectants based on this **clinically proven technology** to, among others, the healthcare, food and travel sectors.



**Intensive sanitizing,
without the
labour-intensive work!**



Why the Huwa-San[®] Fogger?

The portable Huwa-San Fogger takes Huwa-San's technology to the next level. This is because the new nebulizer allows you to thoroughly disinfect an entire room extremely effectively from top to bottom, without anyone being present.

The Huwa-San Fogger is **a huge leap forward in terms of saving time and improving quality!**

You simply start the device, and the Huwa-San disinfectant is atomized throughout the room at record speed.


Huwa-San[®]
FOGGER



**Stop the spread
of infections**

This is how easy it is to use the **Huwa-San[®] Fogger**

1

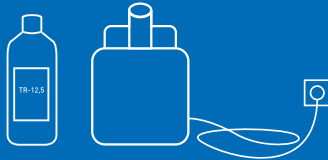

Take the portable Huwa-San Fogger to the room to be disinfected, together with a bottle of Huwa-San TR-12.5 disinfectant. Plug the Fogger's cable into an electrical socket.

2



Unscrew the cap on the bottle, attach the bottle to the nozzle, and snap the nozzle onto the Huwa-San Fogger.

3



Enter the room's volume on the display so that the device knows how much product to dispense. Turn on the Huwa-San Fogger and leave the room, making sure that the room is completely sealed.

4



After the atomizing has finished, leave the hydrogen peroxide to do its work. You can then re-enter the room and welcome your patients, passengers, guests or sports club members in a completely safe environment.

Use biocides safely. Read the label and product information before use.

Key Benefits



Clinically
proven efficacy



Time-saving
(not labour-intensive)



Antiviral and
antibacterial



Effective
against coronavirus



Odourless and
colourless, no residues



Non-corrosive



Discover our full range of surface disinfectants

Only need to disinfect a small area?
Then use our Huwa-San TR-3 range to
disinfect all kinds of surfaces.
Discover our Huwa-San TR-3 Spray
and our Huwa-San TR-3 refill pack.



Huwa-San TR-3 Spray
(500 ml)

Huwa-San TR-3
refill pack
(1 l)

Contact our distributors!

Are you interested in the added value of the
Huwa-San Fogger? Then make an appointment
with one of our distributors.
They will be happy to tell you more about how to
buy the Huwa-San Fogger, and about ordering
and delivery of the Huwa-San disinfectant bottles.

www.roamtechnology.com

Huwa-San®
A sustainable way of disinfection


Roam Technology
YOUR PARTNER IN HYGIENE CONTROL

Roam Technology Belgium
Geleenlaan 24 • I.Z. Poort Genk 6835
B-3600 Genk • Belgium
Tel: +32 (0)89 44 00 42
info@roamtechnology.com