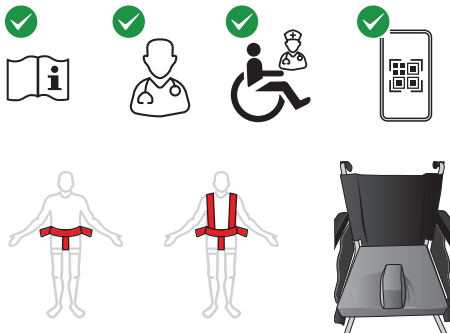
REF	Désignation / Omschrijving Bezeichnung / Designation	Mesures / Maten / Maßnahmen / Measurements
RFC26000	WAIST STRAP SOFT	
RFC26100	WAIST BELT SOFT	
RFC26200	WAIST AND PELVIC BELT SOFT	
RFC26300	WAIST AND SHOULDER BELT SOFT	
RFC26400	WAIST / SHOULDER AND PELVIC BELT SOFT	
RFS19050	Extender nylon buckle 2.0	150cm
RFS20050	Extender nylon buckle 4.0	150cm

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Indications

1. RENOL products are used for a medical purpose to ensure the safety and security of the patient in the chair.
2. RENOL products are only used under medical prescription.
3. The products must be used by trained staff following the RENOL's instructions for safe use and according to the establishment's safety rules.

Precautions of use

4. To avoid any accident and in order to ensure the person's comfort and safety, it is essential to read the instruction leaflets carefully and understand them before operating Renol products. All users follow the instructions and procedures, patients' safety depends on using belts and the locking system correctly.
5. Ensure that a RENOL product is not damaged before use, all products must be clean and in good condition.
6. We recommend monitoring the immobilised patient frequently; restless and combative patients must be under constant observation.
7. Ensure that RENOL products do not come into contact with injured skin, do not obstruct breathing nor impede blood pressure.
8. Always use the correct size: too small or too large may compromise safety or comfort. Measure the patient's waist size, select the appropriate RENOL product.
9. Remove all sharp objects from the patient (jewellery, watches, glasses, etc.).
10. Only use products RENOL on suitable support. Never use RENOL products as a safety belt in vehicles.
11. When supporting or immobilising the patient in a chair, don't forget the patient's ability to turn over.
12. Connect RENOL products to a solid part of the chair. The straps must be secured; they cannot slide or change position; the fastening point of the strap on the frame equipment must not damage the product (sharp edge, etc.).
13. When immobilizing the patient in the chair with the RFC26100 and RFC26300 belt, we recommend the use of a RENOL Abduction cushion (RCV07300 – RCV07400) to prevent forward sliding and possible suffocation of the patient with his strap.

Maintenance and materials

14. Follow the washing instructions marked on the products.
 15. Detergents: avoid using chlorine.
 16. Temperature: maximum temperature 95°C.
 17. Drying: delicate cycle at low temperature, about 5% shrinkage possible.
 18. Material: 100% Terylene (polyester).
 19. The raw materials are treated with fire retardant and anti bacterial.
- All our products are CE-approved – Class I medical devices.
For more information, contact your support specialist, your distributor or www.renolcare.com