
Mölnlycke®

Mesalt®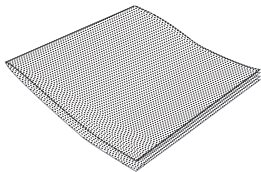

CE 2797


Manufacturer / Fabricant


Mölnlycke Health Care AB
Gamlestadvägen 3C, Box 13080
SE-402 52 Göteborg, Sweden

www.molnlycke.com

Mesalt® 5 x 5cm/2 x 2in • 7.5 x 7.5cm/3 x 3in • 10 x 10cm/4 x 4in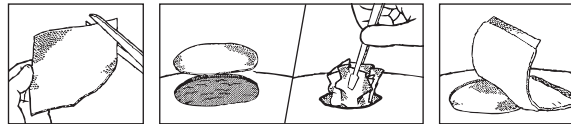

Mesalt® 2 x 100cm / 0.8in x 1.1yd

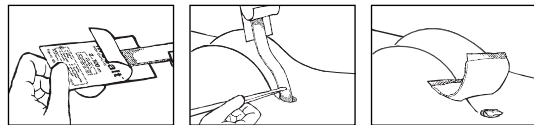www.molnlycke.com/symbols

Single use
Usage unique

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Mesalt®

Cleansing Dressing with 20% Sodium Chloride

en

Product description

Mesalt consists of a soft viscose/polyester nonwoven material impregnated with 20% w/w sodium chloride. Mesalt absorbs wound exudate and the sodium chloride is released. Mesalt becomes hypertonic, thereby exudate, bacteria and necrotic material is drawn into the dressing, facilitating the natural wound healing process.

Product material content: viscose/polyester, 20% w/w sodium chloride.

Indications for use

Management of moderate to heavily exuding non-infected/infected wounds such as pressure ulcers, foot ulcers and deep wound management e.g. cavity wounds and fistulas.

Precaution(s)

- Do not use Mesalt on patient and/or user with known hypersensitivity to the materials/components of the product.
- Mesalt should not be used on dry wounds.
- Mesalt should not be allowed to come into direct contact with exposed bone or tendon.
- Infected wounds should be evaluated on a regular basis and treated appropriately.
- Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.
- Sterile. Do not use if sterile barrier is damaged or opened prior to use. Do not re-sterilise.

Instructions for use

Mesalt can be used by lay persons under supervision of health care professionals.

Application:

1. Clean the wound area and debride of any excessive necrotic tissue or eschar.
2. Dry the surrounding skin, but do not dry the surface of the wound.
3. Where clinically indicated, a water-resistant cream or ointment can be used on the surrounding skin.

4. Choose the appropriate Mesalt dressing size. For shallow wounds do not allow Mesalt to overlap the wound edges. For deep wounds, loosely fill the wound.
5. Apply Mesalt dry into the wound
6. Depending on the level of exudate, cover with an appropriate secondary dressing.
7. Where clinically indicated, compression therapy may be used in conjunction with Mesalt.

en

Dressing change and removal

Mesalt should be changed when saturated (at least once a day), or according to good clinical practice. If the condition of the wound deteriorates, discontinue use and consult a healthcare professional.

Special storage conditions and handling conditions

Mesalt should be stored in dry conditions (below 60% RH) and below 25°C (77°F).

Disposal should be handled according to local environmental procedures.

Other information

For EU-customers, further information on the safety and clinical performance of Mesalt please refer to the 'Summary of Safety and Clinical Performance' for Mesalt. The document is published in the European Database on Medical Devices, EUDAMED, <https://ec.europa.eu/tools/eudamed>.

If any serious incident has occurred in relation to the use of Mesalt, you should report to Mölnlycke Health Care and to your local competent authority.

Mesalt is a registered trademark of Mölnlycke Health Care AB.

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