


REF	Désignation / Omschrijving Bezeichnung / Designation	Mesures / Maten / Maßnahmen / Measurements	REF	Désignation / Omschrijving Bezeichnung / Designation
RFC29200	WAIST AND PELVIC BELT LOCK			
RFC29400	WAIST/SHOULDER AND PELVIC BELT LOCK			
RFK01111	MAGNET KEY (M)			
RFK01222	MAGNET BUTTON + PIN (M)			
RFK01201	MAGNET PIN			
RFK01202	MAGNET BUTTON			
RFK00100	STEEL KEY (RL)			
RFK00433	STEEL BUTTON + PIN (RL)			
RFK00300	STEEL PIN			
RFK00400	STEEL BUTTON			
REF + M =	BELT + MAGNET LOCKS			
REF + RL =	BELT + STEEL LOCKS			


REF + M =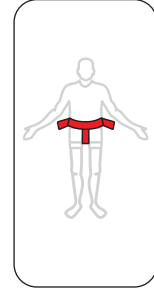

REF + RL =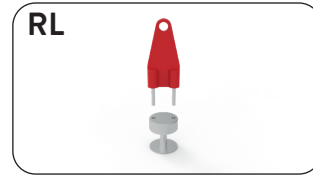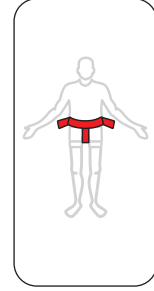

- Associer ceinture abdominale et ceinture épaules toujours avec une ceinture pelvienne
- Bijbehorende Taillegordel en schoudergordel altijd met bekkengordel
- Zugehöriger Hüftgurt und Schultergurt immer mit einem Beckengurt
- Associate waist belt and shoulder belt always with a pelvic belt

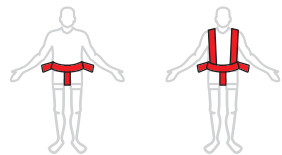Lire la notice (IFU) / Lezen IFU / Lesen IFU / Read IFU

Regarder la Video / Bekijk Video / Schau Video / Watch Video

Utilisation à la chaise / Gebruik in stoel / Stuhl benutzen / Using in chair

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Indications

- RENOL products are used for a medical purpose to ensure the safety and security of the patient in the chair.
- RENOL products are only used under medical prescription.
- The products must be used by trained staff following the RENOL's instructions for safe use and according to the establishment's safety rules.

Precautions of use

- To avoid any accident and in order to ensure the person's comfort and safety, it is essential to read the instruction leaflets carefully and understand them before operating belts. All users follow the instructions and procedures, patients' safety depends on using belts correctly.
- Ensure that a RENOL product is not damaged before use, all products must be clean and in good condition.
- We recommend monitoring the immobilised patient frequently; restless and combative patients must be under constant observation.
- Ensure that RENOL products do not come into contact with injured skin, do not obstruct breathing nor impede blood pressure.
- Always use the correct size: too small or too large may compromise safety or comfort. Measure the patient's waist size, select the appropriate RENOL product.
- Remove all sharp objects from the patient (jewellery, watches, glasses, etc.).
- Only use products RENOL on suitable support. Never use RENOL products as a safety belt in vehicles.
- When supporting or immobilising the patient in a chair, don't forget the patient's ability to turn over.
- Connect RENOL products to a solid part of the chair. The straps must be secured; they cannot slide or change position; the fastening point of the strap must not damage the product (sharp edge, etc.).
- When using the abdominal belt, always use the pelvic belt RFP22000 unless otherwise specified by the prescribing doctor.
- Ensure that the RENOL key, part number: RFK00100 (RL) and RFK01111 (M) is always available. Use only RENOL locks.
- In an emergency, if the key is not available or problem with lock system, cut the strap close to the frame.
- Before use, check operation of the magnet (M) lock system, check correct operation of the RFK01202 Button with the key RFK01111 and pin RFK01101. We recommend replacing the Button RFK01202 after 2 years. The lifetime may be shorter or longer, depending on use.
- Before use, check operation of the mechanical lock system (RL), check correct operation of the RFK00400 Button with the key RFK00100 and long pin RFK00300. We recommend replacing the Button RFK00400 every 3 years. The lifetime may be shorter or longer, depending on use.
- The magnetic key (M) must be placed 10 cm minimum from a pacemaker or any other medical system that could be disturbed by a magnet.

Maintenance and materials

- Follow the washing instructions marked on the products.
 - Close Velcro fastenings before washing and drying to prevent them wearing.
 - Detergents: avoid using chlorine.
 - Temperature: maximum temperature 95°C.
 - Drying: delicate cycle at low temperature, about 5% shrinkage possible.
 - Remove all locking systems before washing and drying.
 - Never immerse the magnetic (M) and mechanical closures (RL) in a liquid.
 - Material: 100% Terylene (polyester).
 - The raw materials are treated with fire retardant and anti bacterial.
 - 2 years warranty, this warranty does not cover damage resulting from normal wear and tear.
 - All our products are CE-approved - Class I medical devices.
- For more information, contact your support specialist, your distributor or www.renolcare.com